


VITA Linearguide 3D-MASTER®

The systematic and precise way to achieve the correct tooth shade

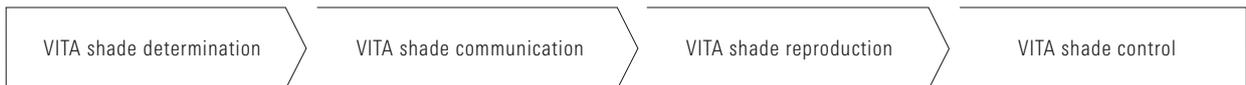

Date of issue: 2023-03

VITA – perfect match.

VITA

Product description

With the VITA Linearguide 3D-MASTER you can determine the correct tooth shade quickly and precisely. The ergonomic design and the systematic arrangement of the VITA Linearguide 3D-MASTER enables quick determination of the suitable 3D-MASTER shade.

The VITA Linearguide 3D-MASTER facilitates the change-over from traditional shade determination systems to precise shade determination and shade communication in the VITA SYSTEM 3D-MASTER. For more information visit www.vita-zahnfabrik.com

VITA Linearguide 3D-MASTER® Simply efficient



Remove the VITA Valueguide 3D-MASTER from the opened VITA Linearguide 3D-MASTER.



Use the VITA Valueguide 3D-MASTER to make an initial choice by comparing the shade samples with the natural tooth. Determine which degree of lightness from 0 to 5 matches the tooth shade.



Based on your initial choice, take the corresponding VITA Chroma / Hueguide 3D-MASTER (0/1, 2, 3, 4 or 5) out of the plastic box and determine the chroma and the hue.



You have determined the suitable tooth shade quickly and reliably in just two steps.

Benefits

- A fast and precise way to achieve the correct tooth shade in just two steps
- Systematic arrangement and modern design for a simple, self-explanatory application
- Clear organization of the shade samples in terms of lightness, chroma and hue
- 26 natural and systematically arranged tooth shades and three additional shades for tooth whitening (VITA Bleached Shades)
- Possibility of creating intermediate shades by mixing

Benefit from the advantages of the VITA SYTEM 3D-MASTER and the ability to achieve reliable and systematic determination of all tooth shades.

Designation	Prod. No.
VITA Linearguide 3D-MASTER	B363

VITA