VITAVM®LC

Verblendkunststoff

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VITAVM[®]LC

Veneering resin

Please read the instructions for use before using the product.

The VITA VM LC product range includes systematically matched components for fixed and removable restorations for extraoral use. VITA VM LC flow is the low viscosity flow material of the product family.

Intended purpose

VITA VM LC products are veneer resin materials for dental prosthesis.

Patient target group

No restriction

Intended user

Dental professionals only: dentists and dental technicians (Rx only)

Indications

- Full and partial veneering of metal frameworks: crowns, bridges, telescopic crowns, implant suprastructures
- Inlays, veneers

Application areas:

- Individualization and layering of VITA ENAMIC
- Veneering partially yttrium-stabilized ZrO₂ frameworks (CTE 10.0 - 10.5 · 10⁻⁶ · K⁻¹), such as VITA YZ SOLUTIONS
- Individualization of VITA acrylic teeth
- Reproduction of gingival areas
- Veneering of removable and partially removable dentures (according to the manufacturer's information) made of polyether ether ketone (PEEK) with a ceramic filler content of up to 20%, such as BioHPP/Bredent - PEEK OPTIMA LT1 polymer, e.g., Juvora, InnoBlanc Medical

Long-term temporaries:

- Individualizing and layering of long-term temporaries made from VITA CAD-Temp
- Metal-free crowns and three-unit anterior bridges made from VITA VM LC

Contraindications

- Occlusal dysfunctions or parafunctions, such as bruxism
- All alloys and resin framework materials may be used for frameworks, which are suitable for veneering with composite, according to the manufacturer's information.

Product application/control

Workflow:

Please adhere to the following workflow when using this product:

1. Framework preparation
2. Surface conditioning if required (e.g., VITA VM LC PRIMER I+II, observe manufacturer's information)
3. Application of opaque and subsequent polymerization, according to polymerization information*
4. Individual application of dentine, incisal and effect materials (intermediate polymerization after application of a material is recommended, final polymerization is required for layer thicknesses of 2 mm and more, observing the polymerization information *)
5. Application of VITA VM LC GEL is recommended to avoid an inhibition layer during final polymerization
6. Final polymerization is carried out according to the polymerization information *

*Polymerization information

When using light-curing composites, a good polymerization result strongly depends on the power of the unit in use. To be able to provide reliable recommendations, VITA has extensively tested the most common light curing devices on the market in combination with VITA VM[®]LC. Polymerization times and information can be found here:

Recommended product combinations

For recommended product combinations with the VITA system components and corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to Instructions for Use I Full Version.

Storage/disposal

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data

Physical properties of the BASE DENTINE and incisal materials

	Unit of measure	VM LC	VM LC flow
Flexural Strength	MPa	approx. 90	approx. 130
Water absorption	µg/mm ³	< 40	< 40
Solubility	µg/mm ³	< 5	< 5

Chemical composition of the BASE DENTINE and incisal materials

Components	Wt%	
	VITAVM [®] LC	VITAVM [®] LC flow
(Meth-)acrylates	39 – 45	32 – 41
Highly dispersed silicon dioxide	41 – 52	–
Mixed oxide from silicone dioxide and zirconia	–	55 – 68
Copolymer	7 – 13	–
Initiators and stabilizers	< 2	< 3
Pigments	< 1	< 1

Safety information

When working with the product, wear suitable safety goggles/face protection, gloves and safety clothing.

Product safety

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at https://www.vita-zahnfabrik.com/product_safety
The safety data sheets can be downloaded at www.vita-zahnfabrik.com or requested by fax at (+49) 7761 -562 -233.

Hazardous substances

VITA VM LC PRE OPAQUE, MODELLING LIQUID,

Danger

H317 May cause an allergic skin reaction.
H319 Causes serious eye irritation.
H412 Harmful to aquatic life with long lasting effects

VITA VM LC OPAQUE PASTE

Danger

H315 Causes skin irritation.
H317 May cause an allergic skin reaction.
H319 Causes serious eye irritation.
H412 Harmful to aquatic life with long lasting effects

VITA VM LC CHROMA PLUS , VITA VM LC PAINT, VITA VM LC flow

Danger

H317 May cause an allergic skin reaction.
H412 Harmful to aquatic life with long lasting effects.

VITA VM LC OPAQUE LIQUID,

Danger

H225 Highly flammable liquid and vapour.
H315 Causes skin irritation.
H317 May cause an allergic skin reaction.
H335 May cause respiratory irritation

VITA VM LC PRIMER

Danger

PRIMER I

H225 Highly flammable liquid and vapour.
 H315 Causes skin irritation.
 H319 Causes serious eye irritation.
 H336 May cause drowsiness or dizziness.

PRIMER II

H225 Highly flammable liquid and vapour.
 H315 Causes skin irritation.
 H317 May cause an allergic skin reaction.
 H335 May cause respiratory irritation
 H412 Harmful to aquatic life with long lasting effects.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of this information: 2024-01

After the publication of this information for use, any previous versions become obsolete. The current version in each case is available at www.vita-zahnfabrik.com

VITA Zahnfabrik has been certified and the following products bear the mark:

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VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen (Germany)
Zweigniederlassung Basel c/o Perrig AG, Max Kämpf-Platz 1, 4058 Basel
VITA Zahnfabrik · H. Rauter GmbH & Co. KG