## Gebrauchsinformation

### VITA ADIVA® C-PRIME Einkomponenten-Silanhaftvermittler

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### Instructions for use

### VITA ADIVA® C-PRIME Single-component silane bonding agent

VITA ADIVA C-PRIME is a single-component silane bonding agent that creates a chemical bond between restorations made of ceramic as well as hybrid ceramic (VITA ENAMIC) and adhesive composite, thus strengthening the adhesive bond. VITA ADIVA C-PRIME can be used with all adhesive composites formulated on the basis of methacrylate (e.g. VITA ADIVA F-CEM).

#### Composition

Solution of methacrylate silane in ethanol.

#### Indication

For chemical bonding of sintered, pressed, and CAD/CAM-fabricated restorations made of feldspar ceramic (e.g. VITABLOCS), hybrid ceramic (VITA ENAMIC), and glass ceramic (e.g. VITA SUPRINITY PC) to adhesive composite. In addition, VITA ADIVA C-PRIME can also be used for the intraoral repair of ceramic restorations with composites.

#### Contraindication

For the adhesive bonding of restorations made of silicate-free zirconium oxide ceramic (e.g. VITA YZ), silanization is ineffective.

#### Use

### 1. Bonding of restorations

Place VITA ADIVA C-PRIME on a mixing palette and quickly apply a thin layer on a restoration that has been etched with 5% hydrofluoric acid gel (e.g. VITA ADIVA CERA-ETCH), using a disposable brush or microbrush. Next, carefully disperse with a water- and oil-free air stream until the surface is no longer moist and appears glossy.

Surfaces treated in this way are now suitable for bonding with adhesives and composites on the basis of methacrylates (e.g. VITA ADIVA F-CEM), which should be applied within **30 min** after the application of VITA ADIVA C-PRIME.

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## EN

**Note:** Contamination of the silanized adhesive surface with saliva or blood prevents a reliable bonding. Clean contaminated surfaces again with 5% ceramic etching gel (e.g. VITA ADIVA CERA-ETCH) for **15 s**, thoroughly rinse, and dry. Repeat the silanization afterwards.

### 2. Intraoral repairs of restorations

Grind surfaces to be repaired with a diamond grinder (40 µm) or with an intraoral sandblaster. Next, rinse with water and dry with a water- and oil-free air stream.

If the repair can be carried out extraorally, etching can also be done with 5% hydrofluoric acid (e.g. VITA ADIVA CERA-ETCH), if necessary. This creates a better bond between the ceramic and the composite.

**Caution: Hydrofluoric acid is highly corrosive and poisonous! Avoid all body contact (e.g., hands)! Observe the instructions for use.**

After etching, thoroughly rinse the restoration with water or clean it in an ultrasonic bath. **Never** use hydrofluoric acid intraorally!

### Use of VITA ADIVA C-PRIME

#### See 1.

#### Important:

VITA ADIVA C-PRIME can interfere with the adhesive bonds to the dentine. For this reason, always use fine brushes or microbrushes for applying the silane bonding agent, and limit the application exclusively to the restoration. Never let VITA ADIVA C-PRIME get on the adjacent tooth substance.

#### Storage information

Do not store above 25 °C (77°F)! Do not use after the expiry date.

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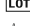
 Symbol for „storage temperature“

#### Forms of delivery

VITA ADIVA C-PRIME, dropper bottle of 5 ml each.

#### Guarantee

Harvard Dental International GmbH guarantees that this product is free of material and manufacturing defects. Harvard Dental assumes no further liability, nor any implicit guarantee concerning marketability or suitability for a particular purpose. The user is responsible for the application and the proper use of the product. If defects occur on the product within the warranty period, the customer's only claim and the only obligation of Harvard Dental is the repair or replacement of the Harvard Dental International product.

#### Limitation of liability

Insofar as exclusion of liability is legally allowed, Harvard Dental International GmbH bears no liability for losses or damage to this product, regardless of whether direct, indirect, special, concomitant, or consequent damage is concerned, regardless of the legal basis, including guarantee, contract, negligence, or premeditation.

**Always store out of reach of children!**  
**For dental use only!**

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