# VITA

VITA Zahnfabrik · H. Rauter GmbH & Co. KG  
Spitalgasse 3 · 79713 Bad Säckingen  
Germany · Allemagne · Alemania · Germania  
Tel. +49 (0) 7761/562-0  
Fax +49 (0) 7761/562-299  
www.vita-zahnfabrik.com  
info@vita-zahnfabrik.com

Manufactured for VITA

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### Instructions for use

EN

**VITA VIONIC** BASE is a millable blank made of a high-quality, industrially polymerized acrylic polymer that features shade stability for the CAD/CAM fabrication of denture bases.

#### Additional information

For the fabrication of denture bases in the **VITA VIONIC** System. **VITA VIONIC** BASE are acrylic blanks made of PMMA polymethyl methacrylate for processing in dental milling systems.

#### System-specific indications

**VITA VIONIC** BASE is intended exclusively for the fabrication of full dentures with the approved **VITA** denture teeth and adhesive systems (**VITA VIONIC BOND**) in the **VITA VIONIC** System.

#### System-specific contraindication

**VITA VIONIC** BASE and the **VITA VIONIC** system components are not approved for the fabrication of full dentures with denture teeth from third party providers.

#### Finishing / polishing

Please read the current **VITA VIONIC SOLUTIONS** working instructions for the processing of **VITA VIONIC** BASE and system components (www.vita-zahnfabrik.com).

#### Processing

Please read the current **VITA VIONIC SOLUTIONS** working instructions for the processing of **VITA VIONIC** BASE and system components (www.vita-zahnfabrik.com).

#### Relining and repair

Denture bases made of **VITA VIONIC** BASE can be relined and repaired with a standard PMMA cold-curing resin.

Manufacturer recommendation: cold-curing resin FuturaGen (Schütz Dental GmbH) in the shades orange, transparent pink and opaque pink.

#### Additional safety information

##### Side effects

Undesirable side effects of the medical device and system components are extremely rare when properly processed and used.

Immune responses (e.g., allergies) or local discomfort can occur. Should you be aware of unwanted side effects - even those you are unsure of - please immediately notify **VITA Zahnfabrik H. Rauter GmbH & Co. KG**, Spitalgasse 3, 79713 Bad Säckingen.

#### Contraindications / interactions

In the event of hypersensitivity of the patient to any of the components, this product and the system components must not be used, or must only be used under the strict supervision of the attending physician / dentist. In these cases, the composition of the medical device supplied by us is available on request. Known cross-reactions or interactions of the medical device with other materials in the mouth, must be considered by the dentist.

#### Note

Please give the abovementioned information to the attending dentist. When processing, pay attention to the existing safety data sheets.

#### Labeling

Symbol for “date of manufacture”

Symbol for “Protect from light”

Symbol for “batch number”

The batch number (LOT) is printed on the **VITA VIONIC** BASE blank.

“Please read the instructions for use”

#### Delivery form

**VITA VIONIC** BASE is available in dimensions Ø 98.4 / h 30 mm and Ø 98.4 / h 26 mm, each in the three shades Light-Pink, Orange-Pink and Deep-Pink.

VITA VIONIC® BASE		
Shade	Size	Product number
Light-pink	Ø 98.4 / h 30 mm	ADLP9830
Orange-pink	Ø 98.4 / h 30 mm	ADOP9830
Deep-pink	Ø 98.4 / h 30 mm	ADDP9830
Light-pink	Ø 98.4 / h 26 mm	ADLP9826
Orange-pink	Ø 98.4 / h 26 mm	ADOP9826
Deep-pink	Ø 98.4 / h 26 mm	ADDP9826

**VITA VIONIC System Components** for the bonding of **VITAPAN EXCELL®** und **VITAPAN®** LINGOFORM DD FRAME teeth and approved **VITA** tooth lines

#### VITA VIONIC® BOND

Description	Prod. No.
VITA VIONIC BOND KIT	AD9VBK

#### Disposal

Waste from **VITA VIONIC** BASE can be disposed of in household waste.

#### Warranty

Our technical application notes, whether oral, written or practical, are based on our own experience and can therefore only be considered as guidelines.

#### Service life

The maximum service life is printed on the label. Do not use after the expiration date.

#### Physical properties

Composition: > 98% PMMA,  
< 2% inorganic pigments

Vickers hardness HV50: 14.2  
Flexural strength: 91 MPa  
E-modulus: 2650 MPa  
Water absorption: 26.8 µg/mm<sup>3</sup>  
Solubility in water: 1.6 µg/mm<sup>3</sup> residual monomers: < 1.0%  
Shelf life: 10 years

The technical data are averages and may fluctuate slightly between delivered batches.

#### Additional information

Detailed information on the processing of **VITA VIONIC** BASE can be found in the processing instructions for **VITA VIONIC SOLUTIONS**.

Please observe instructions for use!

The safety data sheet can be downloaded from [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) or requested by fax at +49 (0) 77 61 / 562-233.

The product bears the mark

**CE 0297** VITA VIONIC BASE

The product has been developed for use in the dental sector and must be processed in accordance with the instructions for use. The information and illustrations provided do not constitute a guarantee of features and are not binding.

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