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### VITA ADIVA® OXY-PREVENT

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## Instructions for use

### VITA ADIVA® OXY-PREVENT

Please read the instructions for use before using the product.

### Intended use

VITA ADIVA OXY-PREVENT is an oxygen-impermeable, translucent glycerine gel designed to prevent the oxygen inhibition layer from forming during the processing of composites. It can be removed easily with water spray after application and is well-suited to use as a try-in paste.

### Patient target group

No restrictions.

### Indication

- For application to the cementation joint of all-ceramic, hybrid ceramic or composite restorations *before* the polymerization of the composite cement.
- For masking composite fillings before polymerization.
- As a try-in paste for trying in ceramic or composite restorations to evaluate the shade effect.

### Contraindication

None known.

### Product application/control

1. Application during the luting of all-ceramic, hybrid ceramic or composite restorations:  
Use restorations with composite cement (e.g., VITA ADIVA F-CEM). Remove excess. Apply VITA ADIVA OXY-PREVENT to the composite joint using an application cannula. Light-cure the entire composite cement with VITA ADIVA OXY-PREVENT. Remove VITA ADIVA OXY-PREVENT with water spray under suction. Finish and polish the restoration.
2. Use with direct composite fillings:  
Apply VITA ADIVA OXY-PREVENT to the modeled composite filling using an application cannula. Light-cure the entire composite using VITA OXY-PREVENT. Remove VITA ADIVA OXY-PREVENT with water spray under suction. Finish and polish the restoration.
3. Use as a try-in paste for trying in ceramic and composite restorations:  
Apply VITA ADIVA OXY-PREVENT in the cavity or on the restoration using the application cannula. Try in to evaluate the shade and fit. Remove VITA ADIVA OXY-PREVENT with water spray. Once this is done, dry the restoration with an air blower.

## Recommended product combinations

VITA ADIVA OXY-PREVENT is applied using single-use application cannulas for VITA ADIVA OXY-PREVENT.

### Storage/disposal

Seal tightly and store at room temperature between 10 °C and 23 °C.

The products labeled with a pictogram for hazardous substances must be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems.

### Technical data

Chemical composition	
Components	Wt%
Water	58 - 68 %
Glycerol	30 - 34 %
Hydroxypropyl methylcellulose	<1 %
Sodium benzoate	<1 %
Potassium sorbate	<1 %

Physical data: Not required.

### General risks/reporting incidents

Information on the general risks associated with dental treatments and on reporting serious incidents in connection with medical devices can be found at [https://www.vita-zahnfabrik.com/service\\_risks](https://www.vita-zahnfabrik.com/service_risks)

Safety data sheets can be downloaded at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) or requested by fax from (+49) 7761-562-233.

### Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability for damage incurred as a result of using the product in conjunction with materials or equipment provided by other manufacturers that is not compatible or not authorized for use with our product. The VITA Modulbox is not a required component of the product. Date of issue of this information: 10/20 After the publication of this information for use, any previous versions become obsolete. The current version can be found under [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com)

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VITA Zahnfabrik has been certified and the following product bears the marks:

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