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DE

VITA classical A1 – D4®

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Instructions for use

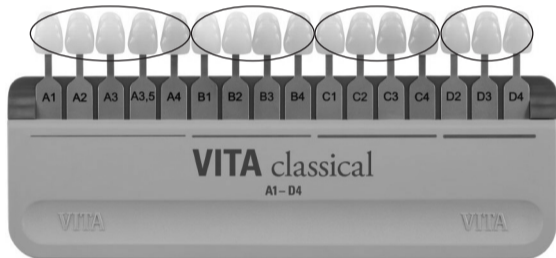
VITA classical A1 – D4®

Please read the instructions for use before using the product.

Structure

The VITA classical shade family is divided into the following categories:

A1. A2. A3. A3.5. A4 B1. B2. B3. B4 C1. C2. C3. C4 D2. D3. D4
Reddish-brownish reddish-yellowish grayish reddish-grayish

Procedure

- The shade guide should be held with arm bent, directly in front of the patient's tooth (distance of approx. 25 - 30 cm).
- Decide quickly on the shade by comparing the shade samples of groups A – D with the natural tooth.
- Disinfect the shade guide after each shade determination.

Tips on determining tooth shade.

- Determine the tooth shade before preparation, since after preparation the tooth appears lighter due to dehydration.
- Determine the tooth shade preferably in natural daylight or using standardized daylight lamps (5500K – 6500K) and not under typical indoor lighting conditions.
- The surrounding area should be as color neutral as possible. Remove influencing lipstick or cosmetics and cover up strong-colored clothes with a gray cape.
- Hold the shade sample tooth as close to the patient's tooth as possible.

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EN

- Only use the middle area of the shade sample tooth for selecting the shade.
- Make your choice quickly and accept your first decision, since the eyes begin to tire after approx. 5-7 seconds.

Hygiene tips and instructions for care

The acrylic parts of VITA classical A1 – D4 shade guide are made of a high-quality, easy-care and temperature resistant material that can be surface disinfected. We recommend customary, mild spray disinfectant or disinfectant wipes. According to the manufacturers' product information these disinfectants are effective against HIV and HBV, and are available from most dental dealers. Please follow the respective manufacturer's instructions for use. Disinfectants containing phenol, compounds with phenylphenol groups or methyl ethyl ketone will damage the acrylic parts.

The entire shade guide can be autoclaved at a temperature of 132 °C/270 °F.

Note

The acrylic parts of the VITA classical A1 – D4 shade guide must be protected from intense sunlight.

VITA BLEACHED SHADES

With the additional VITA Bleached Shades, you can correctly determine the shades of bleached teeth. The bleach shades OM1, OM2, OM3 are taken from the VITA 3D-MASTER SYSTEM. If you have selected the VITA Bleached Shades as an extension set, please attach the adapter to the VITA classical A1–D4 shade guide from the left side.

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