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VITACOLL®

Instructions for use

VITACOLL® bonding agent for denture teeth

Please read the instructions for use before using the product.

Intended use

VITACOLL is a liquid bonding agent that creates a reinforced bond between PMMA-based denture teeth and denture base materials.

Patient target group

No restrictions

Indication

For partial and full dentures. Recommended application for hot polymers; mandatory application for autopolymers / cold polymers. Today's autopolymers can generally be used with both the casting and packing techniques.

Contraindication

VITACOLL may only be used as a bonding agent for PMMA-based denture teeth.

Product application/control

Preparation of the denture teeth before transfer to denture base acrylic

1. Clean the teeth with boiling water and remove all wax and contaminants
2. Roughen the basal and cervical-circular tooth areas by powder blasting (corundum particle grain size 100-150 µm, air pressure 2-3 bar) or by using a milling machine; round off sharp edges and avoid hole retentions.

Use of VITACOLL

4. Apply the VITACOLL bonding agent liquid to the roughened tooth surfaces using a brush
5. Allow it to react for five minutes. Dampened surfaces should have a homogeneous silk matte gloss; if not, the process must be repeated starting from step four.
6. After the reaction time, the denture teeth should be fixed in the denture base acrylic within 10 minutes. If the 10 minutes are exceeded, the adhesive effect of VITACOLL will be reduced and the application of VITACOLL has to be repeated starting from step four.

Detailed information on processing can be found in the document "Instructions for Use | Full Version" at www.vita-zahnfabrik.com/vitacoll

Recommended product combinations

Recommended application for hot polymers; mandatory application for autopolymers / cold polymers.

Storage/disposal

Ensure that the bottle is closed tightly and store it at a refrigerated temperature of 5°C (41°F) to 10°C (50°F).

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems.

Technical data

Chemical Composition	Wt %
butanone CAS no. 78-93-3	50
methyl methacrylate CAS no. 80-62-6	50

Safety information

Close the bottle immediately after use with the original cap.

General risks / reporting incidents

Information on the general risks associated with dental treatments and on reporting serious incidents in connection with medical devices can be found at www.vita-zahnfabrik.com/service_risks

Hazardous substances

DANGER!

H225 Highly flammable liquid and vapor.

H315 Causes skin irritation.

H319 Causes serious eye irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

H336 May cause drowsiness and dizziness.

Safety data sheets can be downloaded at www.vita-zahnfabrik.com/vitacoll or requested by fax from (+49) 7761-562-233.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not a required component of the product. Date of issue of this information: 04.21

After the publication of this information for use any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com

VITA Zahnfabrik has been certified, and the following products bear the mark:

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