

VITA ADIVA® FULL-ADHESIVE LUTING



Workflow for full-adhesive luting of zirconia crowns
 Workflow für volladhäsive Befestigung von Zirkondioxid-Kronen

1. Tooth Conditioning

2. Restoration Conditioning

3. Luting

1 2


3 4

5 6

1. Tooth Conditioning
Konditionierung Zahn

1

VITA ADIVA®
TOOTH-ETCH

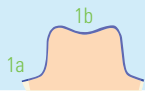

30s

20s

20s

20s



2

VITA ADIVA®
T-BOND I + II

1:1

5-10s

massage in

30s

gentle

15s

20s

massage in

30s

gentle

15s

20s

For „Restoration Conditioning“/ „Luting“ please turn over!
 Für „Konditionierung Restauration“/„Befestigung“ bitte wenden!

VITA ADIVA® FULL-ADHESIVE LUTING



1. Tooth Conditioning

2. Restoration Conditioning

3. Luting

Workflow for full-adhesive luting of zirconia crowns
 Workflow für volladhäsive Befestigung von Zirkondioxid-Kronen

1 2

3 4

5 6

2. Restoration Conditioning /
 Konditionierung Restauration

3. Luting /
 Befestigung

