# VITA MultiPump

Gebrauchsanweisung  
Instructions for use  
Mode d'emploi  
Istruzioni d'uso  
Instrucciones de uso

VITA shade determination

VITA shade communication

VITA shade reproduction

VITA shade control

Date of issue: 2022-02

VITA – perfect match.

**VITA**

## Information

**DE**

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Information

EN


In order to put the VITA MultiPump into operation, a VITA vacuum pump (prod. no. **DVPU220, DVPU110 or DVPU100**) must be available.

The VITA MultiPump is delivered with two USB cables as well as vacuum tube material for connecting two furnaces. In order to operate up to four furnaces, the accessory set (Prod. No. D61004) is required.

The VITA MultiPump can only be operated using a VITA vPad excellence with a serial number higher than **2320152050** as well as all models in the VITA VACUMAT 6000 M and VITA VACUMAT 6000 MP series, in combination with a VITA vacuum pump.

Please note that in order to be able to operate the VITA MultiPump, software update **150707** or a later version must be installed on all VITA vPads and dental furnaces from the series listed above.

Please also note that it is possible to operate the VITA MultiPump with just one or up to four VITA vPad excellence devices. The following diagram provides an example of how four dental furnaces from the above series can be operated with one VITA vacuum pump, one VITA vPad excellence and one VITA MultiPump.

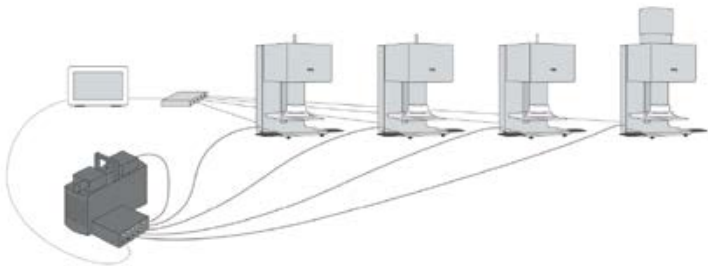**Note:**

**Before connecting the tubes and cable, please remove the corresponding stoppers in each case from the VITA MultiPump. Please leave the stoppers in the connections that you do not currently wish to use in order to prevent these connections from becoming contaminated.**

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


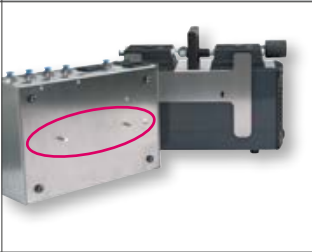
Pos	Action	Picture
<p><b>1</b></p>	<p><b>Furnace check</b></p> <p>If you have purchased your VITA MultiPump together with one or more dental furnaces from the above series, you can ignore the following note.</p> <p>If you have purchased your VITA MultiPump separately and you would like to operate it with existing VITA dental furnaces from the above series, please check that your furnaces are leak-tight. To do so, connect the VITA vacuum pump without the VITA MultiPump to the VITA furnaces to be tested.</p> <p>In your VITA vPad excellence, go to the “Settings” tab and under “Furnace settings”, select “Leak test.” Perform the leak test. Once your VITA dental furnace successfully passes this leak test, you can use the VITA MultiPump without any limitations. In the unlikely event that your VITA dental furnace is not sufficiently leak-tight, please contact the VITA service department as specified below for details of the next steps.</p>	
<p><b>2</b></p>	<p>Once you have unpacked the contents of the VITA MultiPump packaging, you will first need the VITA vacuum pump. The aim is to first attach the VITA MultiPump to the VITA vacuum pump using the bracket provided for this purpose.</p>	
<p><b>3</b></p>	<p>To do so, use the hex key provided to remove the screws on the handle of the VITA vacuum pump.</p>	

Pos	Action	Picture
4	Detach the handle and store it safely together with the two screws. You will need everything together again in order to reattach the handle.	 A photograph showing the VITA MultiPump unit with its handle removed. The handle and two screws are placed on the surface next to the pump.
5	Take the VITA MultiPump bracket and place it on the pump in such a way that the mounting holes are positioned directly above the screw holes of the VITA vacuum pump handle.	 A photograph showing the VITA MultiPump bracket being placed onto the pump housing. The mounting holes of the bracket are aligned with the screw holes of the handle.
6	Now reattach the VITA vacuum pump handle using the screws provided for this purpose. To do so, use the hex key provided.	 A photograph showing the VITA vacuum pump handle being reattached to the pump housing using the provided screws and a hex key.
7	Now take the VITA MultiPump.	 A photograph showing the VITA MultiPump unit and its power supply unit. The pump is on the left and the power supply is on the right.
8	Place the VITA MultiPump in an upright position so that you have access to the screw thread on the underside of the housing.	 A photograph showing the VITA MultiPump unit in an upright position, highlighting the screw thread on the underside of the housing.

Pos	Action	Picture
9	Attach the pins provided on the underside of the VITA MultiPump housing.	
10	Now use the pins to hang the VITA MultiPump into the mounting holes provided for this purpose on the attached bracket.	 
11	<p>Using a sharp knife or similar, cut a section approximately 25 cm in length off the vacuum tube provided. Ensure that you cut the tube off as vertically as possible.</p> <p>Now connect the VITA vacuum pump to the VITA MultiPump using the cut section of vacuum tube.</p> <p>While applying a little pressure to the connection, insert the vacuum tube until it slips into the quick coupling with a “jerk”.</p>	 

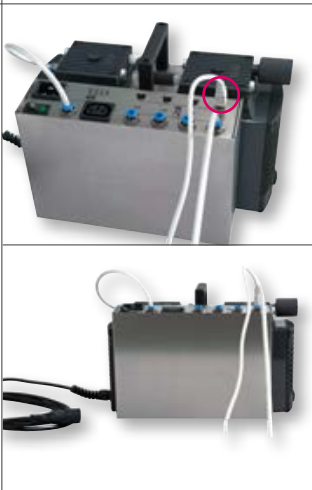Pos	Action	Picture
12	<p>The description below explains how you can connect one dental furnace from the above series as well as one VITA vPad excellence to the MultiPump. Proceed in the same way for all additional dental furnaces and / or VITA vPad excellence devices.</p>	
13	<p>Using a sharp knife or similar, cut a sufficiently long section off the vacuum tube. Ensure that you cut the tube off as vertically as possible.</p> <p>Connect the first VITA dental furnace to the VITA MultiPump using the vacuum tube provided.</p> <p>While applying a little pressure to the connection, insert the vacuum tube until it slips into the quick coupling with a "jerk".</p> <p>All vacuum tubes must hang "loosely". In other words, they should not be pulled tight following insertion in the MultiPump and the dental furnace.</p>	
14	<p>Connect the VITA vPad excellence to the VITA MultiPump using the USB cable provided.</p>	

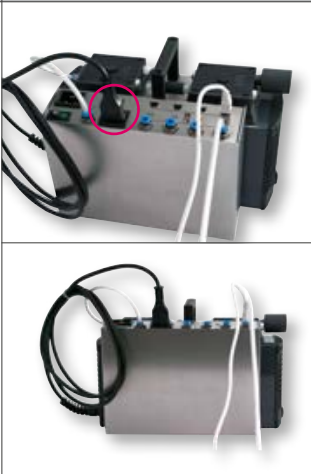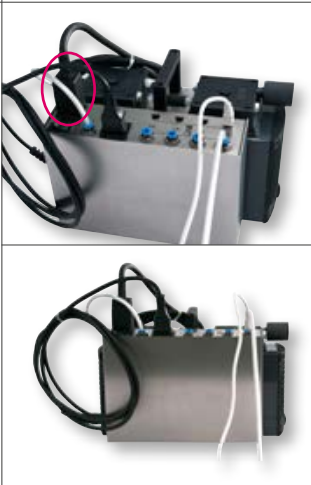Pos	Action	Picture
15	Connect the VITA vacuum pump power cable to the VITA MultiPump.	
16	Now insert the VITA MultiPump power cable into the VITA MultiPump, however, do not connect it to the power source yet.	
17	You have now connected all cables and tubes necessary for operating one dental furnace and one VITA vPad excellence with the VITA MultiPump. The following illustration shows you again which cables you should have connected.	

Pos	Action	Picture
18		
19	<p>If you have connected and attached everything correctly, you can now connect the VITA MultiPump power cable as well as your VITA dental furnace to the power source.</p> <p>You can now switch on the VITA MultiPump using the main switch. Do not switch on the VITA dental furnace until the VITA MultiPump has already been switched on; otherwise, the VITA MultiPump cannot log onto the VITA vPad.</p> <p>The VITA vPad excellence will switch on and boot as usual.</p>	

Pos	Action
-----	--------


**20 Connecting additional VITA dental furnaces and VITA vPads**

Please note:


- Furnace positioning on the VITA vPad excellence must always begin with 1
- Always connect the VITA dental furnaces and VITA vPad to the VITA MultiPump beginning with step 1
- The USB cable of the VITA vPad is always inserted at the location on the VITA MultiPump where the vacuum tube of the corresponding VITA dental furnace is located.

Example: If the first VITA vPad uses vacuum connections 1+2 for two VITA dental furnaces, then the VITA dental furnaces of the second VITA vPad must be connected to vacuum connections 3+4 and the USB cable of the second VITA vPad connected to USB port 3 of the VITA MultiPump.


Three connection examples are shown below.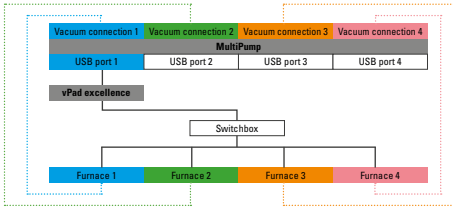

One VITA vPad excellence with four VITA dental furnaces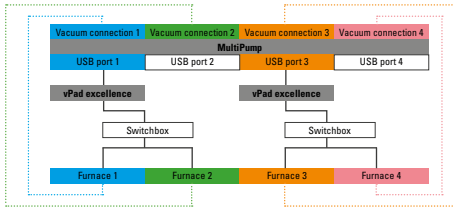

Two VITA vPad excellence devices, each with two VITA dental furnaces

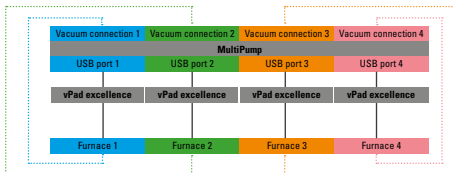Four VITA vPad excellence devices, each with one VITA dental furnace

Pos	Action
21	<p>Notes when using the system</p> <ul style="list-style-type: none"> <li>• Evacuation of multiple VITA dental furnaces at the same time is not possible. When vacuum firing programs are started almost simultaneously in multiple VITA dental furnaces, these will be evacuated one after another.</li> <li>• Essentially the following applies: The VITA dental furnace that has reached the corresponding program start temperature is always the one that will start evacuation.</li> </ul>

	Error messages	Correcting faults
1	<ul style="list-style-type: none"> <li>• <b>No connection to the MultiPump.</b></li> </ul> <p>The status display lights up continuously in red and an audio signal is heard. The VITA vPad excellence has lost its connection to the VITA MultiPump.</p>	<p>Check that the connections between the VITA vPad and VITA MultiPump are properly seated. This error message can only be removed by restarting the VITA dental furnace.</p>
2	<ul style="list-style-type: none"> <li>• <b>Vacuum was not established.</b></li> </ul> <p>There is no communication between the VITA vPad and VITA MultiPump.</p>	<p>Check whether the VITA MultiPump is connected to an energy source. Check whether the VITA MultiPump is switched on. Check that the connections between the VITA vPad and VITA MultiPump are properly seated. Restart the VITA dental furnace after the previous steps have been completed.</p>

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
With the unique VITA SYSTEM 3D-MASTER, all natural tooth shades can be systematically determined and perfectly reproduced.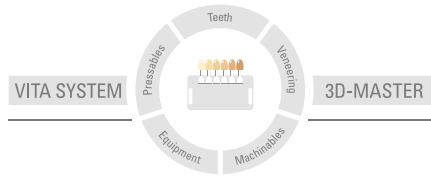

**Please note:** Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of this information: 2022-02

After the publication of this information for use any previous versions become obsolete. The current version can be found at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com)

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VITA MultiPump is CE marked within the meaning of EC Directive 2006/95/EC, 2004/108/EC and 2011/65/EC.

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VITA Zahnfabrik H. Rauter GmbH & Co.KG