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VITAPAN EXCELL® DD FRAME
Denture teeth

VITAPAN® LINGOFORM DD FRAME
Denture teeth

Please read the instructions for use before using the product.

Denture teeth for the digital prosthetics workflow for CAM modification using the Amann Girrbach Full Denture System (FDS).

Intended purpose

VITAPAN EXCELL DD FRAME and VITAPAN LINGOFORM DD FRAME are resin artificial teeth for dental treatments.

Patient target group

No restrictions

Intended user:

Dental professionals only: dentists and dental technicians

Indication

The VITAPAN EXCELL DD FRAME and VITAPAN LINGOFORM DD FRAME denture tooth blanks are machined for the following products using the appropriate Amann Girrbach mills:

- Full dentures
- Partial dentures

Contraindication

VITAPAN EXCELL DD FRAME and VITAPAN LINGOFORM DD FRAME are not suitable for CNC machining in milling machines that are not part of the Amann Girrbach Full Denture System (FDS).

Product application/control

Please adhere to the following workflow when using this product:

1. Clamp the tooth frame in place,
2. CAM modification,
3. Blast the machined surfaces,
4. Remove the prosthetic teeth from the frame,
5. Clean the tooth blanks and remove any residual wax.

Detailed information on working with this product can be found in the Instructions for Use I Full Version document at www.vita-zahnfabrik.com/vionifcframe.

Recommended product combinations

For recommended product combinations with the VITA system components and corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to Instructions for Use I Full Version.

Storage/disposal

The prosthetic teeth should be stored away from light. They can be disposed of in domestic waste. The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data

Chemical composition	Wt%
Polymethyl methacrylate CAS no. 9011-14-7	84 - 86
Silicon dioxide CAS no. 14808-60-7	14 - 15
Pigments	< 1

Physical properties	Value
Appearance according to ISO 22112	Complies with standard
Dimension of teeth according to ISO 22112	Complies with standard
Color and blending of shades according to ISO 22112	Complies with standard
Free of porosity and other defects according to ISO 22112	Complies with standard
Surface finish according to ISO 22112	Complies with standard
Bonding to denture base polymers according to ISO 22112	Complies with standard
Resistance to blanching, distortion, and crazing according to ISO 22112	Complies with standard
Shade stability according to ISO 22112	Complies with standard
Dimensional stability according to ISO 22112	Complies with standard

Safety note

When working with the product, wear suitable safety goggles/face protection and light respiratory protection.

Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks, and (if applicable) short clinical safety and performance reports (SSCPs) can be found at https://www.vita-zahnfabrik.com/product_safety.

The safety data sheets can be downloaded at www.vita-zahnfabrik.com or requested by fax at (+49) 7761-562-233.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of this information for use: 2022-02 After the publication of this information for use, any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com.

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VITA Zahnfabrik H. Rauter GmbH & Co.KG
Zweigniederlassung Basel c/o Perrig AG