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## Instructions for use

### VITACOLL® bonding agent for denture teeth

Please read the instructions for use before using the product.

VITACOLL is a liquid bonding agent that creates a reinforced bond between PMMA-based denture teeth and denture base materials.

### Intended purpose

VITACOLL is an accessory for VITA MRP teeth. VITA MRP (Microfiller Reinforced Polymer matrix) teeth are acrylic denture teeth used for dental treatments.

### Patient target group

No restrictions

### Intended user

Dental professionals only: dentists and dental technicians

### Indication

The use of VITACOLL liquid bonding agent is recommended for heat curing polymers resins and is a mandatory requirement in the case of self-curing / cold polymer resins. Today's self-curing polymer resins can generally be used with both casting and packing techniques. In order to always achieve a reinforced bond between acrylic teeth and base material, the teeth should be treated with VITACOLL.

### Contraindication

VITACOLL may only be used as a bonding agent for PMMA-based acrylic teeth.

### Product application/control

**Preparation of the denture teeth** before transfer to denture base acrylic:

1. Clean the teeth with boiling water and remove all wax and contaminants.
2. Roughen the basal and cervical-circular tooth areas by powder blasting (corundum particle grain size 100-150 µm, air pressure 2-3 bar) or by using a milling machine; round off sharp edges and avoid hole retentions.

### Use of VITACOLL

4. Apply the VITACOLL bonding agent liquid to the roughened tooth surfaces using a brush.
5. Allow it to react for five minutes. Dampened surfaces should have a homogeneous silk matte gloss; if not, the process must be repeated starting from step four.
6. After the reaction time, the denture teeth should be fixed in the denture base acrylic within 10 minutes. If the 10 minutes are exceeded, the adhesive effect of VITACOLL will be reduced and the application of VITACOLL has to be repeated starting from step four.

Detailed information on processing can be found in the document "Step-by-Step Instructions" at [www.vita-zahnfabrik.com/vitacoll](http://www.vita-zahnfabrik.com/vitacoll).

### Recommended product combinations

Recommended application for hot polymers; mandatory application for autopolymers / cold polymers.

### Storage/disposal

Ensure that the bottle is closed tightly and store it at a refrigerated temperature of 5°C (41°F) to 10°C (50°F).

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling

systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

### Technical data

Chemical Composition	Wt %
Butanone	50
CAS no. 78-93-3	
Methyl methacrylate	50
CAS no. 80-62-6	

### Safety information

In case of handling in an open environment, use equipment with local exhaust ventilation.

### Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at [https://www.vita-zahnfabrik.com/product\\_safety](https://www.vita-zahnfabrik.com/product_safety).

### Hazardous substances

- DANGER!
- H225 Highly flammable liquid and vapor.
  - H315 Causes skin irritation.
  - H319 Causes serious eye irritation.
  - H317 May cause an allergic skin reaction.
  - H335 May cause respiratory irritation.
  - H336 May cause drowsiness and dizziness.
  - P280 Wear protective gloves/protective clothing/eye and face protection.
  - P405 Store in a locked location.

Safety data sheets can be downloaded at [www.vita-zahnfabrik.com/vitacoll](http://www.vita-zahnfabrik.com/vitacoll) or requested by fax from (+49) 7761-562-233.

### Disclaimer

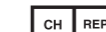
Please note: Our products must be used in accordance with the instructions for use. We do not accept liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of these instructions for use: 2022-03

After the publication of this information for use, any previous versions become obsolete. The current versions can be found at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com).

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Release Date & sign.:  
 Correction required

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