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DE

VITAFOL® H Silikon-Isoliermaterial

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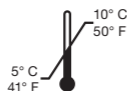
EN

VITAFOL® H silicone insulation material

Please read the instructions for use before using the product.

Intended purpose

VITAFOL H contains silicone insulation material for the protection of acrylic and ceramic teeth during denture fabrication and can be used on wax models prior to embedding.

Patient target group

No restrictions

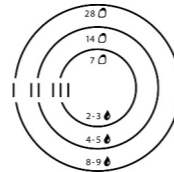
Intended user

Dental professionals only: dentists and dental technicians

Product application/control

Please adhere to the following workflow when working with this product:

- Apply the paste and the hardener liquid to the dosing block and mix for approx. one minute using a spatula.

Paste quantity:

For 28 teeth = outer circle I
For 14 teeth = middle circle II
For 7 teeth = inner circle III

Hardener liquid:

For 28 teeth (circle I) = 8-9 drops
For 14 teeth (circle II) = 4-5 drops
For 7 teeth (circle III) = 2-3 drops

- Apply the mixture to individual areas that require insulation or spread it over the entire wax model and allow it to flow into the interdental spaces.
- While the insulation film is still damp, sprinkle the bonding crystals over the surface evenly.
- Allow to cure for approx. 8-12 minutes at room temperature, then embed the model in the cuvette with plaster as usual.
- After the plaster has set, separate the cuvettes and extract the wax.
- While the divestment process, teeth and areas that have been treated with VITAFOL H come out of the cuvette without any plaster on them.

Recommended product combinations

VITAFOL H consists of VITAFOL H paste, VITAFOL H hardener and VITAFOL H bonding crystals, and should only be used in this combination.

Storage/disposal

VITAFOL H paste and VITAFOL H hardener must be stored at a refrigerator temperature of between 5°C (41°F) and 10°C (50°F).

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Safety information

- After removal, immediately seal the tube and bottle properly.
- If the hardener liquid thickens up, this is due to a failure to observe correct product usage.

- The hardener liquid may become slightly cloudy or mottled; this will not affect its performance.
- The volume of the hardener bottle matches the volume of the paste tube.
- The tube is made of flexible plastic and allows easy kneading of the paste after long periods without use.

Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at https://www.vita-zahnfabrik.com/product_safety.

Hazardous substances

VITAFOL H hardener

IMPORTANT!


- | | |
|-----------|---|
| H226 | Flammable liquid and vapor. |
| H319 | Causes serious eye irritation. |
| H332 | Harmful if inhaled. |
| H335 | May cause respiratory irritation. |
| H361d | May cause harm to unborn children in pregnant women. |
| H372 | Prolonged or repeated exposure will cause damage organs. |
| P210 | Keep away from heat, hot surfaces, sparks, open flames and other sources of ignition. No smoking. |
| P260 | Do not breathe in dust/fume/gas/mist/vapors/spray. |
| P280 | Wear protective gloves/protective clothing/eye and face protection. |
| P337+P313 | In case of persistent eye irritation, seek medical advice or consult a doctor. |

Safety data sheets can be downloaded at www.vita-zahnfabrik.com/vitafolh or requested by fax from (+49) 7761-562-233.

Disclaimer

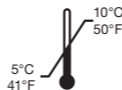Please note: Our products must be used in accordance with the instructions for use. We do not accept liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of this information for use: 2022-03
After the publication of this information for use, any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com.
VITA Zahnfabrik has been certified and the following products bear the mark:

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