

The following SSCP (Summary of Safety and Clinical Performance) is applicable to the following product(s):

VITA AKZENT PLUS®, VITA INTERNO®

### Table of content

Identification of the device .....	2
Indications, Intended Purpose and Target populations.....	2
Device description .....	4
a) Description of the medical device(s).....	4
b) Previous generations of the medical device(s) .....	4
c) Accessories / other products which are intended to be used with the medical device(s) .....	4
Possible therapeutic or diagnostic alternatives.....	6
Reference to harmonized standards and CS applied .....	7
Summary of clinical data.....	9
a) Clinical studies of the medical device(s).....	9
b) Clinical evaluation.....	9
c) Post market clinical follow-up .....	10
d) Conclusion of clinical performance and safety of the medical device(s).....	10
Suggested profile and training of users .....	11
Information on residual risks, undesirable effects and warnings and precautions ....	11
a) Residual Risks.....	11
b) Undesirable effects.....	11
c) Warnings and precautions.....	11

### Revision History

Version	Changes
001	Initial version

### Identification of the device



Device trade name	VITA AKZENT PLUS®, VITA INTERNO®
Manufacturer	VITA Zahnfabrik H. Rauter GmbH & Co. KG Spitalgasse 3 D-79713 Bad Säckingen
Manufacturers SRN	DE-MF-000005906
BASIC-UDI-DI	++J017BB3PS
Medical device nomenclature (EMDN)	Q010699 - MATERIALS FOR THE PREPARATION OF CUSTOM-MADE DENTAL DEVICES – OTHER
Class of device	Ila
Year of first CE certificate	VITA AKZENT PLUS: 2012
	VITA INTERNO: 1997
Notified Body including identification no.	DEKRA Certification GmbH, identification no.: 0124

### Indications, Intended Purpose and Target populations

Intended purpose	VITA AKZENT PLUS and VITA INTERNO products are material-kits for porcelain/ceramic restorations for dental treatments.
Indication	<p>VITA AKZENT PLUS:</p> <ul style="list-style-type: none"> <li>• for internal shade modification and external characterization</li> </ul> <p>Compatible with all dental ceramic materials:</p> <ul style="list-style-type: none"> <li>• CAD/CAM materials (e.g. VITABLOCKS, VITA YZ SOLUTIONS)</li> <li>• All-ceramics (such as VITA LUMEX AC)</li> <li>• Metal ceramics (such as VITA VM 13)</li> <li>• Press ceramics (such as VITA PM 9)</li> </ul>

	<ul style="list-style-type: none"><li>• Suitable for the firing temperatures from 750 °C to 950 °C</li><li>• CTE independent</li></ul> <p>VITA INTERNO: The VITA INTERNO stains can be used with all types of ceramic techniques in combination with the following products: VITA VM 9, VITA VM 11, VITA VM 13, VMK Master, VITA VMK 95 and VITA LUMEX AC.</p>
Contraindication	<p>VITA AKZENT PLUS: None known.</p> <p>VITA INTERNO: None known.</p>
Intended user	Dental technician, Dentist, Rx only

### Device description

#### a) Description of the medical device(s)

With VITA AKZENT Plus stains, users can easily and efficiently optimise the colour of all types of dental ceramic materials, regardless of the CTE of the restoration. No matter whether you colour your restoration internally or superficially characterise and finely glaze it: With the fluorescent VITA AKZENT Plus stains, this can be done easily and without much effort. The VITA AKZENT Plus stains are available as powders and ready-to-use pastes depending on the processing preference, habit and field of application.

The glazing BODY STAINS and GLAZE are additionally available as a spray, which ensures a large-area, uniform surface treatment without scattering loss through even application. The new, special spray head stands for targeted application.

The anomalies occurring in the natural tooth are the colour model for the VITA INTERNO shades. With these materials, natural colour effects can be recreated and ceramic work can be perfectly matched in colour to the natural model.

Especially the colour effects occurring in the incisal area of natural teeth can be reproduced naturally with VITA INTERNO.

They can be used either pure or mixed into Base Dentine, Opaque Dentine, Dentine, Enamel or the translucent materials.

#### b) Previous generations of the medical device(s)

Veneering ceramics and their characterization material have been on the market in all possible variations for many decades. Special predecessor products of VITA AKZENT PLUS and VITA INTERNO do not exist in this form.

#### c) Accessories / other products which are intended to be used with the medical device(s)

*The following products can, but do not have to be used with the product. For a detailed description of how to use the products in combination, please refer to the product's instructions for use.*

#### Accessories for VITA AKZENT PLUS:

Name of accessory (manufacturer)	Short description
VITA AKZENT PLUS mixing plate	<ul style="list-style-type: none"> <li>– 22 trough spaces for mixing all powder stains and glaze materials</li> <li>– one large hollow trough as a liquid reservoir, with small ring-shaped sponge</li> </ul>

VITA AKZENT PLUS shade sample disc	<ul style="list-style-type: none"> <li>– disc with printed shade samples to assess the shade effect of the stains after firing</li> <li style="padding-left: 20px;">– 14 EFFECT STAINS</li> <li style="padding-left: 20px;">– 5 BODY STAINS</li> </ul>
VITA AKZENT PLUS CHROMA STAINS Shade Guide	<ul style="list-style-type: none"> <li>– Shade guide with printed shade samples to assess the stains after firing</li> <li style="padding-left: 20px;">– 4 CHROMA STAINS classical A1-D4</li> <li style="padding-left: 20px;">– 5 CHROMA STAINS 3D-MASTER</li> </ul>
VITABLOCS Mark II	Recommended substructure

### Accessories for VITA INTERNO:

Name of accessory (manufacturer)	Short description
VITA VM 9	Recommended combination product
VITA VM 13	Recommended combination product
VITA VMK MASTER	Recommended combination product
VITA VMK 95	Recommended combination product

### Possible therapeutic or diagnostic alternatives

Diagnostic/therapeutic alternative with conditions of use	Possible benefit/advantage and possible risks/disadvantages as far as known
VITA AKZENT PLUS and VITA INTERNO are characterisation materials indicated for the staining of dental restorations. They have no therapeutic benefit and do not have to be used in any case. They are only intended to enhance the naturalness and aesthetics of the dental restoration.	There is no risk other than not using these materials, except that the aesthetics of the restoration may be minimised.

### Reference to harmonized standards and CS applied

Common specifications are not used for the products VITA AKZENT PLUS and VITA INTERNO. The following standards are applied for these products at VITA:

- *DIN EN 1641 10:2009 Dentistry – Medical devices for dentistry – Materials;*
- *DIN EN 22248 02:1993 Packaging Complete filled transport packages, Vertical impact test by dropping*
- *DIN EN 62366 08:2021 Medical devices - Application 01 usability engineering to medical devices*
- *DIN EN ISO 10993-1 05:2021 Biological evaluation of medical devices – Part 1: Evaluation and testing within a risk management system*
- *DIN EN ISO 10993-10 10: 2014 Biological evaluation of medical devices – Part 10: Tests for irritation and skin sensitization*
- *DIN EN ISO 10993-11 09:2018 Biological evaluation of medical devices – Part 11: Tests for systemic toxicity*
- *DIN EN ISO 10993-12 08:2021 Biological evaluation of medical devices – Part 12: Sample preparation and reference materials*
- *DIN EN ISO 10993-13 11:2010 Biological evaluation of medical devices – Part 13: Identification and quantification of degradation products from polymeric medical devices*
- *DIN EN ISO 10993-14 08:2009 Biological evaluation of medical devices Part 14: Identification and quantification of degradation products from ceramics*
- *DIN EN ISO 10993-16 02:2018 Biological evaluation of medical devices – Part 16: Toxicokinetic study design for degradation products and leachables*
- *DIN EN ISO 10993-17 08:2009 Biological evaluation of medical devices Part 17: Establishment of allowable limits for leachable substances (*
- *DIN EN ISO 10993-18 03:2021 Biological evaluation of medical devices Part 18: Chemical characterization of materials*
- *DIN EN ISO 10993-2 10:2006 Biological evaluation of medical devices – Part 2: Animal welfare requirements*
- *DIN EN ISO 10993-3 02:2015 Biological evaluation of medical devices – Part 3: Tests for genotoxicity, carcinogenicity and reproductive toxicity*
- *DIN EN ISO 10993-4 12: 2017 Biological evaluation of medical devices– Part 4: Selection of tests for interactions with blood*
- *DIN EN ISO 10993-5 10:2009 Biological evaluation of medical devices – Part 5: Tests for in vitro cytotoxicity*
- *DIN EN ISO 10993-6 09:2017 Biological evaluation of medical devices – Part 6: Tests for local effects after implantation*
- *DIN EN ISO 10993-9 03:2022 Biological evaluation of medical devices – Part 9: Framework for identification and quantification of potential degradation products*
- *DIN EN ISO 13485 12:2021 Medical devices – Quality management systems – Requirements for regulatory purposes*
- *DIN EN ISO 14971 04:2022 Medical devices – Application of risk management to medical devices*
- *DIN EN ISO 15223-1 02:2022 Medical devices – Symbols to be used with information to be supplied by the manufacturer – Part 1: General requirements*
- *DIN EN ISO 20417 03:2022 Information to be supplied by the manufacturer of medical devices;*
- *DIN EN ISO 4180 03:2020 Packaging – Complete filled transport packages – General rules for compilation of performance test schedules*
- *DIN EN ISO 7405 03:2019 Dentistry – Evaluation of biocompatibility of medical devices used in dentistry*
- *ISO 10993-1 10:2018 Biological evaluation of medical devices – Part 1: Evaluation and testing within a risk management process*

- *ISO 13485 03:2016 Medical devices – Quality management systems: Requirements for regulatory purposes*
- *ISO 2206 04:1987 Packaging – Complete filled transport packages – Identification of parts when testing*
- *ISO 7405 10:2018 Dentistry – Evaluation of biocompatibility of medical devices used in dentistry*
- *ISO TR 24971 06:2020 Medical devices – Guidance on the application of ISO 14971*
- *ISO 109931 Devices-Guidance FDA 2020*
- *MEDDEV 2\_7\_1\_rev4\_en 06:2016*



### Summary of clinical data

#### a) Clinical studies of the medical device(s)

According to Article 61 European Medical Device Regulation, (EU) 2017/745 (MDR) the requirement to perform clinical investigations pursuant to paragraph 4 shall not apply to implantable devices and class III devices:

- which have been lawfully placed on the market or put into service in accordance with Directive 90/385/EEC or Directive 93/42/EEC and for which the clinical evaluation is based on sufficient clinical data, and
- is in compliance with the relevant product-specific CS for the clinical evaluation of that kind of device, where such a CS is available; or
- that are sutures, staples, dental fillings, dental braces, tooth crowns, screws, wedges, plates, wires, pins, clips or connectors for which the clinical evaluation is based on sufficient clinical data and is in compliance with the relevant product-specific CS, where such a CS is available.

The products under evaluation (VITA AKZENT PLUS, VITA INTERNO) have been lawfully placed on the market and put into service in accordance with Directive 93/42/EEC and the clinical evaluation of these products is based on sufficient clinical data, and is in compliance with the relevant product-specific CS for the clinical evaluation of that kind of device. Therefore, clinical investigations are not necessary to proof clinical safety and clinical evaluation is performed.

#### b) Clinical evaluation

*Extract from the final summary of the clinical evaluation “Veneering Ceramics and Stains VITA Zahnfabrik H. Rauter GmbH & Co. KG” at novineon CRO GmbH November 2021/Rev.4: [...]*

On the basis of the documentation provided by VITA and current scientific literature, we conclude that the potential risks associated with the application of Veneering Ceramics and Stains are acceptable residual risks for the patient and the user. The main clinical risks, chipping of the restauration surface and decrease in color matching are described in dental in the scientific literature. Furthermore, VITA recognized also the occupational health risks of the materials for dental technicians. On the basis of the relevant scientific literature, we conclude that the risks associated with the use of dental ceramics of this kind are well-documented in the published literature, thus being known to dental physicians and adequately trained dental technicians. When complying with all warnings and precautions, the Veneering Ceramics and Stains have an acceptable benefit/risk profile. [...]

### c) Post market clinical follow-up

PMCF studies may not be required when the medium/long-term safety and clinical performance are already known from previous use of the device or where other appropriate post-market surveillance activities would provide sufficient data to address the risks.

The indication and treatments of VITA dental medical devices are well-known clinical procedures. The basic principles of using dental material are the same since the beginning of the 20th century.

The VITA post market monitoring collects clinically relevant data to an extent that fulfills the requirements of the European Medical Device Regulation, (EU) 2017/745 (MDR) in order to adequately assess and confirm the safety of medical devices.

Therefore, no PMCF studies are required for VITA AKZENT PLUS and VITA INTERNO and therefore, also not part of this SSCP.

### d) Conclusion of clinical performance and safety of the medical device(s)

VITA always ensures the clinical safety of its products, even after they have been placed on the market, by constantly updating the clinical evaluation of its medical products and monitoring them on the market in accordance with the requirements of the European Medical Device Regulation, (EU) 2017/745 (MDR) and according to MEDDEV 2.7/1 revision 4.

For VITA AKZENT PLUS and VITA INTERNO it can be said that the clinical evaluation of the products clearly states the clinical safety and performance. There is no doubt about the safety and reliability of the products. They can be used safely in the manner communicated by VITA with regard to indications, contraindications, compliance with safety instructions and residual risks.

### Suggested profile and training of users

VITA dental products are designed for use by professional users. This specification is made clear by the labeling of VITA products with the symbol "Rx only". The specialist users are dentists and dental technicians who have excellent prior knowledge in the use of our products due to their many years professional training and/or university education. Follow-up training is the responsibility of the expert users and is offered by VITA specifically for VITA products. This guarantees safe handling of VITA products at every point in the application process.

### Information on residual risks, undesirable effects and warnings and precautions

#### a) Residual Risks

Information on product-specific residual risks can be found on the VITA homepage at [Downloadcenter. Produktsicherheit \(vita-zahnfabrik.com\)](https://www.vita-zahnfabrik.com)

Please be aware, that these are possible complications and residual risks of the dental product group in general, and not specific to VITA materials.

These risks must be communicated to the patient by trained personnel (e.g., dental professionals).

#### b) Undesirable effects

There are no known undesirable effects for the products.

#### c) Warnings and precautions

- **Extremely flammable aerosol**

Spray-on ceramic glaze material. For dental applications only. Not for intraoral use. Shake well before use. Pressurized container: do not puncture or burn. Protect from direct sunlight and temperatures above 50 C°. Do not pierce or burn even after use. Do not spray into flames or onto glowing objects. Keep away from ignition sources. - No smoking. Keep away from heat / sparks / open flame / hot surfaces. **IMPORTANT: VITA AKZENT PLUS glaze sprays must not be used in combination with spray-on opaque materials.**



- **In case of formation of dust, use an extraction system or wear a face mask.**



*These warnings and precautions can also be found in the corresponding instructions for use of the product(s).*