VITA Polish Cera

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VITA Polish Cera

Please read the instructions for use before using the product.

Intended use

VITA Polish Cera is a diamond polishing paste for extraoral and final high-gloss polishing of restorations made of dental ceramic materials: feldspar ceramics, glass ceramics and zirconia ceramics.

Product application/control

Workflow:

VITABLOCS feldspar ceramics (e.g., VITA LUMEX AC and VITABLOCS):

Pre-polishing with flexible diamond-coated discs (e.g., Sof-Lex, 3M) or fine-grained diamond finishing instruments.

Then apply polishing paste using a soft goat hair or bison polishing brush.

Speed 5,000 - 10,000 min⁻¹.

Final high-gloss polishing with a dry cotton buff.

Maintain recommended speed and work with moderate contact pressure to avoid excessive heat development.

Lithium disilicate ceramic (e.g., VITA AMBRIA), lithium silicate ceramic (e.g., VITA SUPRINITY PC), zirconia ceramic (e.g., VITA YZ):

For optimum pre- and high-gloss polishing, we recommend using the polishers of the VITA SUPRINITY Polishing Set technical for the handpiece, Prod. No. ELSPSETT, or the VITA SUPRINITY Polishing Set clinical for the contra-angle, Prod. No. ELSPSETC, as the first polishing step.

Then apply polishing paste using a soft goat hair or bison polishing brush.

Speed 5,000 - 10,000 min⁻¹.

Final high-gloss polishing with a dry cotton buff.

Maintain recommended speed and work with moderate contact pressure to avoid excessive heat development.

Recommended product combinations

VITA Polish Cera polishing paste is very suitable in connection with polishing of VITA SUPRINITY PC, VITA AMBRIA and VITA YZ in conjunction with the polishers of the VITA SUPRINITY Polishing Sets clinical and technical.

Disposal

Residues of polishing paste can be disposed of with household waste.

Safety information

For extraoral use only! Wear personal protective equipment (e.g., eye protection). We recommend the use of a separate polishing brush for each patient.

Technical data

Composition

Mixture of fatty acids and inorganic abrasive substances.

Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at www.zahnfabrik.com/service_risks.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications.

We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Publication of these working instructions:

2022-09

After the publication of these working instructions, any previous versions become obsolete. The current version can be found under www.vita-zahnfabrik.com

VITA Zahnfabrik · H. Rauter GmbH & Co. KG
Spitalgasse 3 · 79713 Bad Säckingen · Germany
Phone +49 (0) 7761 / 562-0
Fax +49 (0) 7761 / 562-299
www.vita-zahnfabrik.com
info@vita-zahnfabrik.com

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