VITABLOCS®

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VITABLOCS®

VITABLOCS are industrially manufactured, fine-structure feldspar ceramic blocks used to fabricate inlays, onlays, veneers and crowns with various CAD/CAM systems.

Intended purpose

VITABLOCS are ceramic materials for dental treatments.

Patient target group

No restrictions.

Intended user

Dental professionals only: dentists and dental technicians

Indication

Indication	Type of material	VITABLOCS Mark II	VITABLOCS Triluxe forte	VITABLOCS RealLife
 Inlay		●	○	○
 Onlay		●	○	○
 Table top		●	○	○
 Veneer		○	●	●
 Endo-crown*		○	○	○
 Anterior crown		○	●	●
 Posterior crown		○	○	○
 Veneer structure for the VITA Rapid Layer Technology		●	●	—

● recommended ○ possible * molars only

Contraindication

General

- in cases of inadequate oral hygiene
- for insufficient preparation results
- in cases of insufficient hard tooth substance
- in cases of insufficient space

Hyperfunction

- Restorations made of VITABLOCS are contraindicated for patients diagnosed with excessive occlusal function, in particular, those who grind and clench their teeth. The use of VITABLOCS restorations for devitalized teeth of patients with hyperfunctions is absolutely contraindicated.

Endocrowns - premolars

- Due to the small adhesive surface and the small root diameters, endocrowns for premolars are contraindicated.

Bridges

- Since VITABLOCS consist of a fine-structure feldspar ceramic with a limited strength of approx. 140 MPa, this material is not suitable for the fabrication of monolithic (mono-ceramic) bridges.

All-ceramic substructures

- VITABLOCS are not suitable for the fabrication of all-ceramic substructures. In addition, VITA LUMEX AC must only be used for individualization and not for full veneers of copings made from these materials.

Product application/control

Preparation and ceramic layer thicknesses

Inlays

Occlusal: at least 1.5 mm
Area of the isthmus: at least 1.5 mm

Onlays

Occlusal: at least 1.5 mm
Area of cusps: at least 2.0 mm

Veneers

Incisal: 0.5 - 0.7 mm
Labial: 0.5 mm
Cervical: 0.2 - 0.3 mm

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Incisal: at least 1.5 mm
Circumferential: at least 1.0 mm
Crown margin: 1.0 mm

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Area of the cusps: 1.5 mm - 2.0 mm
Occlusal: at least 1.5 mm
Circumferential: at least 1.0 mm
Crown margin: 1.0 mm

Processing

Select the program for VITABLOCS. Fix VITABLOCS according to the instructions of the manufacturer of the CAD/CAM system and start the milling process. Diamond-coated milling tools are required for processing.

Finishing and polishing

- Prior to cementation, proximal areas are polished outside the mouth, for example, with VITA Polish Cera diamond polishing paste.
- Finishing/smoothing the outer and occlusal surface of the restoration using flexible discs (for example, Sof-Lex discs, 3M Espe) in descending order of grit size and fine-grit finishing diamonds, while exerting little pressure and ensuring sufficient cooling with water (observe manufacturer's instructions).
- High-gloss polishing with diamond polishing paste (for example, VITA Polish Cera).

Adhesive bonding

- Light- or dual-curing composites, such as VITA ADIVA F-CEM, and an adhesive system should be used for adhesive bonding (cementation) of restorations made of VITABLOCS.

Conditioning tooth substance:

- Etch the tooth substance with VITA ADIVA TOOTH-ETCH (phosphoric acid gel, 35%) for 20 sec.
- Application of an adhesive system (such as VITA ADIVA T-BOND).

Conditioning the restoration:

- Apply VITA ADIVA CERA-ETCH (hydrofluoric acid gel, 5%) to the inner surfaces.
- Etching time: 60 sec.
- Completely remove any remaining acid by using water spray (60 sec) or clean in the ultrasonic bath. Then dry for 20 sec.
- Apply silane (for example, VITA ADIVA C-PRIME) to the etched surfaces and then dry carefully.
- Luting composite (for example, VITA ADIVA F-CEM) is applied thinly and the restoration is carefully placed in situ.
- Light curing

Detailed information on processing can be found in the document "Instructions for Use | full version" at www.vita-zahnfabrik.com/vitablocs

Recommended product combinations

For recommended product combinations with the VITA system components and corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to the Instructions for Use | full version.

Storage/disposal

The VITABLOCS must be stored in a dry place. They can be disposed of with household waste.

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data**Physical properties**

Properties	Unit of measure	Value
CTE (25–500°C)	10^{-6} K^{-1}	9.4 ± 0.1
Density	g/cm^3	2.4 ± 0.5
Flexural strength (Schwickerath) (ISO 6872)	MPa	136 ± 20
Transformation range	°C	780-790

Chemical composition

Oxides	% by weight
SiO ₂	55–70
Al ₂ O ₃	20–24
Na ₂ O	6–10
K ₂ O	4–8
CaO	<1
TiO ₂	<1
Pigments	<1

Safety information

Please refer to the corresponding safety data sheets for detailed information. The corresponding safety data sheets can be downloaded at www.vita-zahnfabrik.com/sds.

When working with the product, wear suitable safety goggles/face protection, gloves and safety clothing. In case of formation of dust, use an extraction system or wear a face mask.

Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at https://www.vita-zahnfabrik.com/product_safety

**Disclaimer**


Please note: Our products must be used in accordance with the instructions for use. We do not accept liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of these instructions for use: 2022-06
After the publication of these instructions for use, any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com.

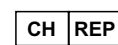VITA Zahnfabrik has been certified and the following product bears the CE mark: VITABLOCS®

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VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen (Germany)
Zweigniederlassung Basel c/o Perrig AG, Max Kämpf-Platz 1, 4058 Basel
VITA Zahnfabrik · H. Rauter GmbH & Co. KG