VITA ENAMIC®

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VITA ENAMIC®

VITA ENAMIC is a hybrid dental ceramic featuring a dual ceramic-polymer network structure.

Intended purpose

VITA ENAMIC products are ceramic materials for dental treatments.

Patient target group

No restrictions.

Intended user

Dental professionals only: dentists and dental technicians

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Contraindication

- Bridge restorations
- Free-end restorations
- Parafunction (for example, bruxism)

Product application/control

Preparation and ceramic layer thicknesses

Inlays

Occlusal: at least 1.0 mm

In the area of the isthmus: at least 1.0 mm

Onlays

Occlusal: at least 1.0 mm

Veneers

Incisal: at least 0.3 mm

Labial.: at least 0.3 mm

Cervical: at least 0.2 mm

Table tops

Occlusal: at least 1.0 mm

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Incisal: at least 1.0 mm

Circumferential: at least 0.8 mm

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Occlusal: at least 1.0 mm

Circumferential: at least 0.8 mm

Processing

Select the program for VITA ENAMIC. Fix the VITA ENAMIC block according to the instructions of the manufacturer of the CAD/CAM system and start the milling process. Diamond-coated milling tools are required for processing.

Finishing and polishing

- We recommend the use of the polishers of the VITA ENAMIC Polishing Sets technical or clinical for optimal pre- and high-gloss polishing.
- Then apply VITA Polish Hybrid polishing paste using a soft goat-hair brush.

- Speed 5,000 - 12,000 min-1.
- Final high-gloss polishing with a dry cotton buff.
- Maintain recommended speed and work with moderate contact pressure to avoid excessive heat development.

Adhesive bonding

- Light- or dual-curing composites, such as VITA ADIVA F-CEM, and an adhesive system should be used for adhesive bonding (cementation) of restorations made of VITA ENAMIC.

Conditioning tooth substance:

- Etch the tooth substance with VITA ADIVA TOOTH-ETCH (phosphoric acid gel, 35%) for 20 sec.
- Application of an adhesive system (such as VITA ADIVA T-BOND).

Conditioning the restoration:

- Apply VITA ADIVA CERA-ETCH (hydrofluoric acid gel, 5%) to the inner surfaces.
- Etching time: 60 sec.
- Completely remove any remaining acid by using water spray (60 sec) or clean in the ultrasonic bath. Then dry for 20 sec.
- Apply silane (for example, VITA ADIVA C-PRIME) to the etched surfaces and then dry carefully.
- Luting composite (for example, VITA ADIVA F-CEM) is applied thinly and the restoration is carefully placed in situ.
- Light curing

Detailed information on processing can be found in the document "Instructions for Use | full version" at www.vita-zahnfabrik.com/enamic

Recommended product combinations

For recommended product combinations with the VITA system components and corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to the Instructions for Use | full version.

Storage/disposal

They can be disposed of with household waste.

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data

Physical properties

| Properties | Unit of measure | Value |
|------------------------------|-----------------|---------|
| Flexural strength (ISO 6872) | MPa | 150–160 |
| Fracture toughness | MPa√m | 1.5 |
| Modulus | GPa | 30 |
| Weibull modulus | – | 20 |
| Hardness | GPa | 2.5 |

Chemical composition of the fine-structure feldspar ceramic network

| Oxides | % by weight |
|--------------------------------|-------------|
| SiO ₂ | 58-63 |
| Al ₂ O ₃ | 20-23 |
| Na ₂ O | 6-11 |
| K ₂ O | 4-6 |
| B ₂ O ₃ | 0.5-2 |
| CaO | <1 |
| TiO ₂ | <1 |

Chemical composition of the polymer network

The polymer network consists of methacrylate polymer.

Safety information

Please refer to the corresponding safety data sheets for detailed information. The corresponding safety data sheets can be downloaded at www.vita-zahnfabrik.com/sds.

**Product reliability**

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks, and (if applicable) short clinical safety and performance reports (SSCPs) can be found at https://www.vita-zahnfabrik.com/product_safety

**Disclaimer**

Please note: Our products must be used in accordance with the instructions for use. We do not accept liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of these instructions for use: 2022-06

After the publication of these instructions for use, any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com.

VITA Zahnfabrik has been certified and the following product bears the CE mark: VITA ENAMIC®

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VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen (Germany)
Zweigniederlassung Basel c/o Perrig AG, Max Kämpf-Platz 1, 4058 Basel
VITA Zahnfabrik · H. Rauter GmbH & Co. KG