# VITA CAD-Temp®

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# VITA CAD-Temp®

VITA CAD-Temp is a composite material made of acrylate polymer for the fabrication of long-term temporary restorations.

## Intended purpose

VITA CAD-Temp products are polymer-based temporary crown and bridge materials for dental treatments.

## Patient target group

No restrictions.

## Intended user

Dental professionals only: dentists and dental technicians

## Indication

VITA CAD-Temp is used for the fabrication of multi-unit, fully or partially anatomical long-term temporary bridge restorations with a span of up to two pontics and a clinical wearing period of up to three years. For the requirements of CAD/CAM systems, please refer to the information provided by the manufacturer of the respective system.

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 recommended \* Only terminal bridges are recommended for wearing periods of more than six months.

## Contraindication

- Permanent restorations with a clinical wearing period of more than three years.
- In patients who are allergic to PMMA.

## Product application/control

### Layer thicknesses

*The following geometries or minimum wall thicknesses must be adhered to:*

Connector areas:

Anterior bridges  
with one pontic 12 mm<sup>2</sup>  
with two pontics 12 mm<sup>2</sup>

Posterior bridges  
with one pontic 12 mm<sup>2</sup>  
with two pontics 16 mm<sup>2</sup>

Minimum wall thickness  
occlusal: 1.5 mm in the central fissure  
Circumferential: 0.8 mm

### Processing

Select the program for VITA CAD-Temp. Fix the VITA CAD-Temp blank according to the instructions of the manufacturer of the CAD/CAM system and start the milling process.

Milling tools are required for processing.

### Finishing and polishing

- Prepolishing using suitable silicone polishers and a small goat-hair brush.
- High-gloss polishing using acrylic polishing agents, such as VITA Polish Hybrid.
- Avoid generation of excessive heat.

### Bonding

- Sandblast the inner surfaces of the restoration with Al<sub>2</sub>O<sub>3</sub> (50-100 µm, pressure of 1–2 bar) or roughen with a coarse diamond bur.
- Fix on the tooth structure with temporary cements/bonding materials.
- If the definitive restoration is to be adhesively fixed, eugenol-free fixing materials must be used.
- Please observe the instructions for use and indications of the respective manufacturers.

Detailed information on processing can be found in the document "Instructions for Use | full version" at [www.vita-zahnfabrik.com/cad-temp](http://www.vita-zahnfabrik.com/cad-temp)

## Recommended product combinations

For recommended product combinations with the VITA system components and corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to the Instructions for Use | full version.

## Storage/disposal

They can be disposed of with household waste.

The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

## Technical data

### Physical properties

Properties	Unit of measure	Value
Flexural strength	MPa (Nmm <sup>-2</sup> )	> 80
Modulus of elasticity	MPa (Nmm <sup>-2</sup> )	Approx. 2800
Softening temperature (DSC)	°C	approx. 118
Inorganic filler content	Wt%	approx. 14
Water absorption	complies with EN ISO 10477 Polymer-based crown and bridge materials	
Solubility	complies with EN ISO 10477 Polymer-based crown and bridge materials	
Shade stability	complies with EN ISO 22112 Artificial teeth for dental prostheses	

### Chemical composition

Component	% by weight
Polymethyl methacrylate	82 - 86
SiO <sub>2</sub>	14 - 17
Pigments	< 1

### Safety information

Please refer to the corresponding safety data sheets for detailed information. Safety data sheets can be downloaded at [www.vita-zahnfabrik.com/sds](http://www.vita-zahnfabrik.com/sds).

### Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks, and (if applicable) short clinical safety and performance reports (SSCPs) can be found at [https://www.vita-zahnfabrik.com/product\\_safety](https://www.vita-zahnfabrik.com/product_safety)

**Disclaimer**

Please note: Our products must be used in accordance with the instructions for use. We do not accept liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of these instructions for use: 2022-06

After the publication of these instructions for use, any previous versions become obsolete. The current version can be found at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com).

VITA Zahnfabrik has been certified and the following product bears the CE mark:

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VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen (Germany)
Zweigniederlassung Basel c/o Perrig AG, Max Kämpf-Platz 1, 4058 Basel
VITA Zahnfabrik · H. Rauter GmbH & Co. KG