VITA – perfect match.

VITA

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### VITA VM<sup>®</sup> 11 Verblendkeramik

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## Instructions for use

### VITA VM<sup>®</sup> 11 veneering material

Please read the instructions for use before using the product.

VITA VM 11 is a low-fusing, fine-structure feldspar ceramic that has been specially developed for individualizing restorations made of zirconia-reinforced lithium silicate ceramic (ZLS).

### Intended purpose

VITA VM11 products are ceramic materials for dental treatments.

### Patient target group

No restrictions

### Intended user

Dental professionals only: dentists and dental technicians (Rx only).

### Indication

- For individualizing restorations made of zirconia reinforced lithium silicate glass ceramic (ZLS), especially for VITA SUPRINITY PC

### Materials:

- VITA SUPRINITY PC zirconia-reinforced lithium silicate glass ceramic, WAK (25-500°C), approx. 11.9 – 12.3 x 10<sup>-9</sup> x K<sup>-1</sup>

### Contraindication

- For substructures not within the recommended CTE range
- For parafunctions (e.g., bruxism)
- If minimum layer thicknesses of the ceramic can not be adhered to
- In cases of inadequate oral hygiene

### Product application/control

#### Workflow:

Please adhere to the following workflow when using this product:

- Substructure preparation (e.g., processing with rotary instruments, crystallization)
- DENTINE application
- ENAMEL application
- First dentine firing
- Shape corrections
- Characterization/glaze firing

#### Firing:

Predry. °C	→ min.	↗ °C/min.	approx. temp. °C*	→ min.	Vac.
400	6:00	55	1. dentine firing 800	1:00	on
	4:00	80	Glaze firing with AKZENT PLUS 800		–

\* As examples, the firing temperatures given represent the highest and lowest recommended firing cycle of the veneering ceramic firing. If necessary, additional firing programs can be found in the detailed instructions for use.

The specified firing temperatures are only reference values. The firing results are influenced by various parameters such as model, performance and age of the firing unit used.

Detailed information on processing and firing can be found in the document "Working Instructions VITA VM11" at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com)

### Recommended product combinations

For recommended product combinations with appropriate VITA system components, corresponding processing recommendations and restrictions regarding the

combination possibilities of the product, please refer to the instructions for use | full version.

### Storage/disposal

They can be disposed of with household waste. The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

### Technical data

Physical properties	Unit of measure	Value
3-point flexural strength	MPa	approx. 100
Solubility in acids	µg/cm <sup>2</sup>	approx. 8
Coefficient of thermal expansion (CTE) (25-500°C)	10 <sup>-6</sup> K <sup>-1</sup>	11.2 – 11.6

Chemical composition	Wt%
SiO <sub>2</sub>	62 – 65
Al <sub>2</sub> O <sub>3</sub>	8.5 – 12
K <sub>2</sub> O	9 – 12
Na <sub>2</sub> O	5 – 7.5
B <sub>2</sub> O <sub>3</sub>	4 – 6
CaO	1 – 2
ZrO <sub>2</sub>	< 1

### Safety information

While work is in progress, wear suitable safety goggles/face protection, gloves and safety clothing.

### Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at [https://www.vita-zahnfabrik.com/product\\_safety](https://www.vita-zahnfabrik.com/product_safety)

Safety data sheets can be downloaded at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) or requested by fax at (+49) 7761-562-233.

### Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product.

Date of issue of these instructions for use: 2022-09  
After the publication of these instructions for use, any previous versions become obsolete. The current version can be found at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com)

VITA Zahnfabrik has been certified, and the following products bear the mark: