
Systemkompatibilität

Stand: 2021-11

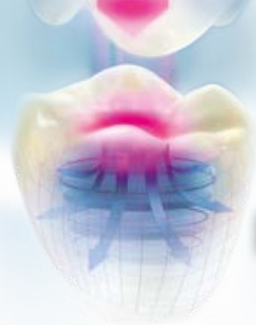VITA – perfect match.

VITA

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VITA – perfect match.

VITA

For more information, visit www.vita-enamic.de

VITA Zahnfabrik has been certified in accordance with the Medical Device Directive and the following product bears

the mark **CE 0124** : VITA ENAMIC®

System and software requirements

VITA ENAMIC:

- **CEREC /inLab MC XL, MC X5 (Dentsply Sirona)**
A **VITA Material Service Pack CEREC SW 4.0.x** can be downloaded for **CEREC software versions > 4.0 and < 4.2** from the Sirona website at www.dentsplysirona.com/de/CERECdownloads. For **inLab software versions > 4.0 and < 4.2**, a **VITA Material Service Pack inLab SW 4.0.x** can be downloaded from the Sirona website at www.dentsplysirona.com/de/inLabdownloads.

To mill VITA ENAMIC also in the small CEREC or inLab milling systems and with previous software versions (< 4.0), please select VITABLOCS Mark II, I14 or I10.

VITA ENAMIC *multiColor*:

Hardware requirements:

CEREC MC, MC X and MC XL Premium Package, Primemill, inLab MC XL and MC X5.

Software requirements:

CEREC 4.5 or higher, CEREC Premium (CAM) 4.5 or higher, inLab (CAM) 16.1 or higher.

- **Milling additive - Dentatec** (relating to a full tank)
 1. CEREC and inLab / 50 ml
 2. CEREC MC, MC X, MC XL, Primemill and inLab MC XL / 75 ml
 3. inLab MC X5/250 ml
- **Ceramill Motion II (Amann Girschbach AG)**
VITA ENAMIC is processed with Amann Girschbach systems using software release Ceramill Mind version 2.1.9 (XP) version 2.3.4 (Win7) and Ceramill Match 2 version 2.1.5 (Win7) or later.

- **KaVo ARCTICA/Everest (KaVo Dental GmbH)**
VITA ENAMIC is available in KaVo multiCAD, software version 2.8.0 or higher and can be processed using the KaVo ARCTICA Engine, version 2.5.9 or higher and using the KaVo Everest Engine, version 9.3 or higher. More information is available at: www.kavo-everest.com
- **IOS Technologies (Glidewell Laboratories)**
VITA ENAMIC can be processed in the TS150 milling system using the latest version.
- **PlanMill (D4D Technologies LLC)**
VITA ENAMIC can be processed in the PlanMill 30S / 40S system using the latest software version.

UNIVERSAL SOLUTIONS*

VITA ENAMIC blocks with universal holder system are available for the following CAD/CAM systems:

- CORITEC Series (imes-icore GmbH)
- CS 3000 (Carestream Inc.)
- DMG ULTRASONIC Series (DMG Mori AG)
- Impression K-/S-Modelle N4 / R5 / S1 / S2 / Z4 (vhf camfacture AG)
- MILLING UNIT M Series (Zirkonzahn S.r.l.)
- Organical Desktop Series (R+K CAD/CAM Technologie GmbH & Co. KG)
- DGS SHAPE DWX Series (DGS SHAPE Corporation)
- Röders RXD Series (Röders GmbH)
- Zfx Inhouse5x
- Straumann CARES C series (Institute Straumann AG)

*The CAD/CAM system partner has been validated by VITA Zahnfabrik for processing VITA ENAMIC.

Characterization of the shade

For shade characterization (staining) of VITA ENAMIC restorations, the VITA AKZENT LC stains are available.*

Individualization

The materials of the VITA VM LC CREATIVE Kit flow Edition, Prod. No. CVLFCFK are available for individualizing (layering technique) restorations made from VITA ENAMIC. Please observe the corresponding working instructions.

Polishing

In order to successfully polish VITA ENAMIC, special 2-stage polishing sets were developed. Perform high-gloss final polishing at the lowest speed and without cooling using water.

- ⚠ **Note:** Restorations made from VITA ENAMIC should not be fired during processing under any circumstances. Shade characterization and individualization are performed by way of polymerization.


🏢 VITA Zahnfabrik H. Rauter GmbH & Co. KG
Spitalgasse 3 | 79704 Bad Säckingen | Germany
Tel. +49 (0) 7761/ 562-0 | Fax +49 (0) 7761/ 562-299
Technical Hotline: Tel. +49 (0) 7761/ 562-222 | Fax +49 (0) 7761/562-446
www.vita-zahnfabrik.com | info@vita-zahnfabrik.com
📘 facebook.com/vita.zahnfabrik

VITA AKZENT LC Kit


Prod. No. CVLFCFK /
VITA VM LC CREATIVE KIT flow Edition

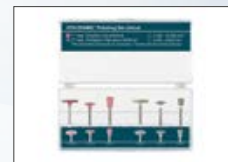Prod. No. EENPSETCV1 / VITA ENAMIC
Polishing Set clinical/for the contra-angle

Prod. No. EENPSETTV1 / VITA ENAMIC
Polishing Set technical/for the handpiece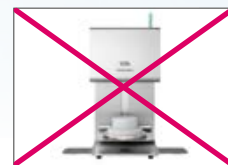

* Availability may vary by country.

VITA – perfect match.

VITA