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VITA – perfect match.

VITA

Thank you for your purchase –
and for trusting VITA SUPRINITY PC!

Further detailed instructions for use
(working instructions no 1951)
are available from www.vita-suprinity.com

Software and system requirements

• CEREC® / inLab® (Dentsply Sirona)

VITA SUPRINITY PC can be processed with Sirona's CEREC or inLab MC XL systems, software version V 4.2 or higher and with the inLab MC X5 system, inLab software version V 4.3 or later and with CEREC Primemill.

- Users of a Sirona system that does not match these system configurations should select lithium disilicate ceramic from the material menu to perform processing.

• ARCTICA® Engine / Everest® Engine (KaVo)

VITA SUPRINITY PC is available in KaVo multiCAD, software version 2.8.0 or higher and can be processed using the KaVo ARCTICA Engine, version 2.5.9 or higher and using the KaVo Everest Engine, version 9.3 or higher.

• PlanMill® 40 (E4D Technologies)

VITA SUPRINITY PC is available for PlanMill 40 from software release 5.8 and higher.

• Ceramill® Motion 2 (AmannGirrbach)

VITA SUPRINITY PC is processed with the Amann Girrbach Ceramill Motion 2 system using software version Ceramill Mind v2.1.9 (XP), v2.3.4 (Win7) and Ceramill Match2 2.1.5 (Win7) or higher.

• UNIVERSAL solutions*

VITA offers VITA SUPRINITY PC with universal holder system for the following CAD/CAM systems:

- Core3d i Serie (Core3d Centres International N.V.)
- CORiTEC Serie (imes-icore GmbH)
- CS 3000 (Carestream Inc.)
- DMG ULTRASONIC Serie (DMG Mori AG)
- Impression K-/S models (vhf camfacture AG)
- MILLING UNIT M Serie (Zirkonzahn S.r.l.)
- Röders RXD Serie (Röders GmbH)
- Roland DWX (Roland Digital Group)
- ZENOTEC select (Wieland Dental +Technik GmbH & Co. KG)
- Zfx Inhouse 5x (Zfx GmbH)

*) The CAD/CAM system partner has been validated by VITA Zahnfabrik for processing VITA SUPRINITY PC.

Crystallization

Predr. °C	→ min	↗ min	↘ °C/min	Temp. approx. °C	→ min	Vac min	↘ °C
400	4.00	7.49	55	830	8.00	8.00	600*

* The long cooling must always be carried out with the furnace chamber closed. Crystallization firing parameters for Programat furnaces see VITA SUPRINITY PC working instructions no 1951.

Shade characterization

For the shade characterization of restorations made of VITA SUPRINITY PC (staining technique) the VITA AKZENT Plus stains can be used. Please follow the corresponding working instructions (no 1925).

Individualization

For the individualization of VITA SUPRINITY PC restorations in the cut-back technique, the porcelains of the VITA VM 11 CREATIVE KIT can be used.

Further information is available in the VITA SUPRINITY PC working instructions.

 VITA Zahnfabrik H. Rauter GmbH & Co.KG
Spitalgasse 3 | 79713 Bad Säckingen | Germany
Tel. +49 (0) 7761/ 562-0 | Fax +49 (0) 7761/ 562-299
Hotline: Tel. +49 (0) 7761/ 562-222 | Fax +49 (0) 7761/ 562-446
www.vita-zahnfabrik.com | info@vita-zahnfabrik.com
 facebook.com/vita.zahnfabrik

VITA VACUMAT
6000 M



VITA AKZENT Plus

VITA VM 11

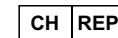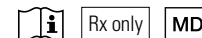VITA Zahnfabrik is certified according to the Medical Device Directive and the following product bears the mark

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VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen (Germany)
Zweigniederlassung Basel c/o Perrig AG, Max Kämpf-Platz 1, 4058 Basel