VITA YZ[®] MULTI TRANSLUCENT

CAD/CAM-BLOCK-MATERIAL

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VITA YZ[®] MULTI TRANSLUCENT

CAD/CAM BLOCK MATERIAL

VITA YZ MULTI TRANSLUCENT are zirconia blanks for reliable shade reproduction. They can be used for the production of fully and partially veneered reconstructions and monolithic bridge restorations in the anterior and posterior tooth regions.

VITA YZ MULTI TRANSLUCENT is part of the VITA YZ SOLUTIONS product group.

Intended purpose

VITA YZ SOLUTIONS products are ceramic materials for dental treatments.

Patient target group

No restrictions

Intended user

Dental professionals only: dentist and dental technician (Rx only).

Indication

VITA YZ MULTI TRANSLUCENT is indicated for:

- fully anatomical anterior and posterior crowns
- fully anatomical 4-unit anterior and posterior bridges
- fully and partially veneered single-tooth and up to 4-unit bridge substructures in the anterior and posterior tooth regions,
- Onlays
- Inlays
- Veneers

Contraindication

VITA YZ MULTI TRANSLUCENT is contraindicated for:

- more than two bridge units
- more than one cantilever bridge unit
- patients with parafunctions, in particular for bruxism
- inadequate oral hygiene
- insufficient preparation results
- insufficient hard tooth substance
- patients who have known allergies or sensitivities to the chemical ingredients of the material
- conventional or self-adhesive insertion of inlays, onlays, veneers

Product application/control

Workflow:

1. Preparation, shade determination
2. Intraoral scan or taking the impression / fabricating the model
3. CAD-Process (design)
4. CAM process (milling/grinding)
5. Reworking (removal/grinding of the restoration)
6. Manual shading technique or immersion technique (optional)
7. Sintering
8. Individualization/characterization/glazing/polishing
9. Bonding

Design parameters:

All information refers to sintered restorations	Minimum wall thickness in mm/ connector cross-sections		
	incisal occlusal axial	0.6 0.6 0.5	-
Crowns, inlays, onlays and veneers in the anterior and posterior regions	incisal axial	0.6 0.6	9 mm ²
Fully anatomical 3-unit anterior bridges	occlusal axial	0.7 0.6	12 mm ²

Note:

- The minimum wall thicknesses refer to fully sintered restorations.

Sintering parameters

A chairside furnace with speed sintering function is recommended for the sintering process. The sintering programs can be found in the manufacturer's recommendations for the respective firing device.

Note: Please observe the maximum restoration size, depending on the firing chamber size of your firing device.

Recommended product combinations

For recommended product combinations with the VITA system components, corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to the Instructions for Use | full version

Storage/disposal

Store the VITA YZ discs and materials in the original packaging and in a dry place. The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data

Property	Unit	Value
3-point flexural strength (ISO 6872)	MPa	850 MPa (incisal) - 1200 MPa (cervical)
Coefficient of thermal expansion (20-500 °C)	10 ⁻⁶ K ⁻¹	approx. 10.2
Chemical solubility (ISO 6872)	µg/cm ²	< 20
Density after sinter firing	g/cm ³	approx. 6.05

Chemical composition

Component	Wt%
ZrO ₂	86-93
Y ₂ O ₃	6-10
HfO ₂	1-3
Al ₂ O ₃	0-1
Pigments	0-2

Safety note

- While work is in progress, wear suitable safety goggles/face protection
- Only perform work under an extraction unit
- Wear protective gloves

Information on bonding of VITA YZ restorations in the patient's mouth can be found at www.vita-zahnfabrik.com/adviva.

Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at www.vita-zahnfabrik.com/product_safety. The safety data sheets can be downloaded at www.vita-zahnfabrik.com or requested by fax at (+49) 7761-562-233.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application. We cannot accept any liability if the products used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of these instructions for use: 2023-11


After the publication of this information for use, any previous versions become obsolete. The current version can be found at www.vita-zahnfabrik.com.

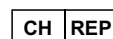VITA Zahnfabrik has been certified and the following products bear the mark:

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CH REP VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen (Germany)
Zweigniederlassung Basel c/o Perrig AG, Max Kämpf-Platz 1, 4058 Basel
VITA Zahnfabrik · H. Rauter GmbH & Co. KG