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Please read the instructions for use before using the product.

VITA VIONIC DENT DISC multiColor are acrylic-based, composite milling blanks with an inorganic filler that are used for the CAD/CAM fabrication of denture teeth for removable and partially removable dentures.

Intended purpose

VITA VIONIC® DENT DISC multiColor products are resin artificial teeth for dental treatments.

Patient target group

No restrictions.

Intended user

Dental professionals only: dentists and dental technicians (Rx only).

Indication

Indication range:

- Full and partial dentures

Contraindication

- Fixed restorations
- In patients with allergies or sensitivities to the ingredients

Workflow

Please adhere to the following workflow when working with this product:

1. Impression taking / intraoral scan, bite registration if necessary
2. Where required, fabrication of a model and scanning / impression scanning
3. CAD process (design)
4. CAM process (milling)
5. Reworking (removal, grinding of the restoration)
6. Conditioning of the bonding surface (recommendation: sandblasting with aluminium oxide 50 µm, 2 - 3 bar. Blow clean with oil-free pressurized air)
7. Bonding, e.g., using the VITA VIONIC BOND Kit (observe the instructions for use)
8. Individualization/characterization/polishing

Recommended product combinations

For recommended product combinations with the VITA system components and corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to Instructions for Use | Full Version.

Storage/disposal

Disposal via household waste. Recyclable waste (such as packaging, paper and plastics) must be disposed of via appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data

Physical properties	Unit of measure	Value
3-point flexural strength	MPa	approx. 80
Flexural modulus	MPa	approx. 2800

Chemical composition	Wt%
Silicon dioxide	14 - 15
Polymethyl methacrylate copolymer	84 - 86
Pigments	< 1

Safety information

- While work is in progress, wear suitable safety goggles / face protection
- Only perform work under an extraction unit

Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at https://www.vita-zahnfabrik.com/product_safety.

The safety data sheets can be downloaded at www.vita-zahnfabrik.com or requested by fax at (+49) 7761-562-233.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of this information: 2024-04.

After the publication of this information for use any previous versions become obsolete. The current version can be found under www.vita-zahnfabrik.com.

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VITA Zahnfabrik has been certified and the following products bear the mark:

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VITA VIONIC® DENT DISC multiColor

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Instructions for use | short version

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