VITAVM[®]CC

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VITAVM®CC

Please read the instructions for use before using the product.

VITA VM CC is a filler-free, cold-curing polymer resin, for extra-oral fabrication of temporary restorations and repairs.

Intended purpose

VITA VM CC products are veneering resins for dental restorations.

Patient target group

No restrictions.

Intended user

Dental professionals only: dentists and dental technicians (Rx only).

Indication

- Fabrication of temporary crowns and bridges
- Bonding of VITA denture teeth, e.g., on model casts
- Repairs
- Coverage of attachments for combined work

Contraindication

- Permanent veneers
- In case of parafunction
- Not recommended for bridge restorations without a substructure

Product application/control

- Mixing: The powder and liquid are mixed in a silicone bowl. VITA VM CC must be mixed in liquid form. Depending on the room temperature, a smaller or larger amount of LIQUID is used for this purpose.
- Once the material has been mixed evenly, it has a doughy consistency and can be processed following an appropriate expansion time (approx. one minute). The processing time span is approx. four to five minutes, depending on the room temperature.

Mixing ratio: A large measuring spoon of VITA VM CC powder (approx. 0.23 g) is mixed with four drops (approx. 0.13 g) of VITA VM CC LIQUID. (reference value)

Recommended product combinations

For recommended product combinations with appropriate VITA system components, corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to the instructions for use | full version.

Storage/disposal

They can be disposed of with household waste. The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data

Physical properties	Unit of measure	Value
3-point flexural strength	MPa	83
Modulus	MPa	2100
Water absorption	µg/mm ³	18
Solubility	µg/mm ³	2

Chemical composition	Wt%
PMMA	97 – 99.5
Dibenzoyl peroxide	0.5 - 2
Pigments	< 1

Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at https://www.vita-zahnfabrik.com/product_safety

The safety data sheets can be downloaded at www.vita-zahnfabrik.com or requested by fax at (+49) 7761-562-233.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application.

We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product.

Date of issue of this information for Use: 2024-01

After the publication of this information for use, any previous versions become obsolete. The current version can be found under www.vita-zahnfabrik.com

Hazardous substances

VITA VM CC

H412 Harmful to aquatic life with long lasting effects

VITA VM CC liquid

Danger

H225 Highly flammable liquid and vapour

H315 Causes skin irritation.

H317 May cause an allergic skin reaction.

H335 May cause respiratory irritation.

Marking

VITA Zahnfabrik has been certified, and the following product bears the marks VITAVM®CC.

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VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen (Germany)
Zweigniederlassung Basel c/o Perrig AG, Max Kämpf-Platz 1, 4058 Basel
VITA Zahnfabrik · H. Rauter GmbH & Co. KG