

VITA AKZENT® Plus FLUOGLAZE LT SPRAY

Quickstart Guide



CTE-independent, highly fluorescent glaze spray

VITA AKZENT® Plus FLUOGLAZE LT SPRAY – FACTS & FEATURES

VITA AKZENT Plus FLUOGLAZE LT SPRAY is:

- a CTE-independent, highly fluorescent glaze spray for all standard* veneering ceramics, monolithic zirconia, silicate or feldspar ceramics
- suitable for achieving a natural fluorescence, especially for monolithic, all-ceramic restorations without auto-fluorescence
- suitable for sealing the ceramic surface

When using VITA AKZENT Plus FLUOGLAZE LT SPRAY, you will benefit from:

- simple enhanced fluorescence
- fast and reliable glazing
- highly esthetic results through natural fluorescence

SYSTEM COMPONENTS



*) all VITA veneering ceramics, VITA YZ, VITA SUPRINITY PC, VITABLOCS, VITA AMBRIA.

VITA AKZENT® Plus FLUOGLAZE LT SPRAY

Using VITA AKZENT Plus FLUOGLAZE LT SPRAY



VITA AKZENT Plus FLUOGLAZE LT SPRAY order:

- Clean the restoration
- Shake spray can well immediately before use
- A distance of approx. 10 cm between the spray head and the surface to be sprayed is required
- Spray from all sides to apply a uniform layer
- Place the restoration on a firing tray for stains firing and glaze firing, and fire according to the firing parameters



> Optimal layer thickness



> Not enough applied



> Too much applied

Please note:

- If the gloss is insufficient, the process can be repeated.

Firing parameters

	Pre-dry. °C	→ min.	↗ min.	↗ °C/min.	approx. temp. °C	→ min.	VAC min.	↘ °C
GLAZE LT SPRAY	400	6.00	4.45	80	740-850*	1.00	-	-

* Firing temperature depends on the veneering ceramic or substructure material used. Observe the manufacturer's instructions


1 Creation of natural fluorescence

2 Example of a natural crown and monolithic zirconia front tooth crowns

natural crowns



monolithic
zirconia crowns

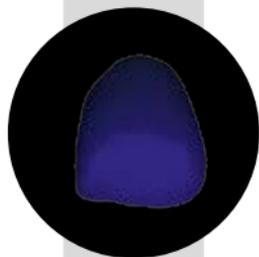

> no auto-
fluorescence




> fluorescence
too weak

with competitor's fluorescent



> streaky due to
use of paste

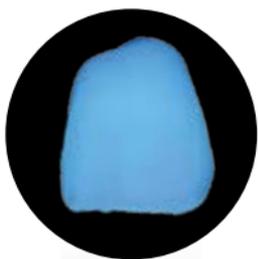

agent



> fluorescence
too high



with VITA AKZENT **Plus FLUOGLAZE LT**



**Perfectly matched and homogeneously
applied fluorescent and glaze agent**
> VITA AKZENT **Plus FLUOGLAZE LT**

VITA AKZENT® Plus – Product portfolio

VITA AKZENT Plus EFFECT STAINS

- highly chromatic and opaque stains
- to reproduce individual shades and to adapt the shade to the characteristics of a natural shade
- when applied thickly, they cover the underlying base shade completely

VITA AKZENT Plus BODY STAINS

- translucent finishing materials that have the effect of a thin color filter on the base material
- for color changes and color shifts towards reddish, yellowish, orange, brownish or gray-greenish

VITA AKZENT Plus CHROMA STAINS

- glazing stains for modifying the chromaticity of the base material within a shade group

VITA AKZENT Plus GLAZE

- Glaze material for a brilliant, glasslike, homogeneous and dense surface after firing

VITA AKZENT Plus GLAZE LT

- LT = Low Temperature: low-melting glaze material for ceramics with firing temperatures below 800°C


VITA AKZENT Plus FLUOGLAZE LT

- CTE-independent, highly fluorescent glaze spray to achieve natural fluorescence

VITA AKZENT Plus Finishing Agent

- for a natural surface texture with a matte finish

VITA AKZENT® Plus – Available shades

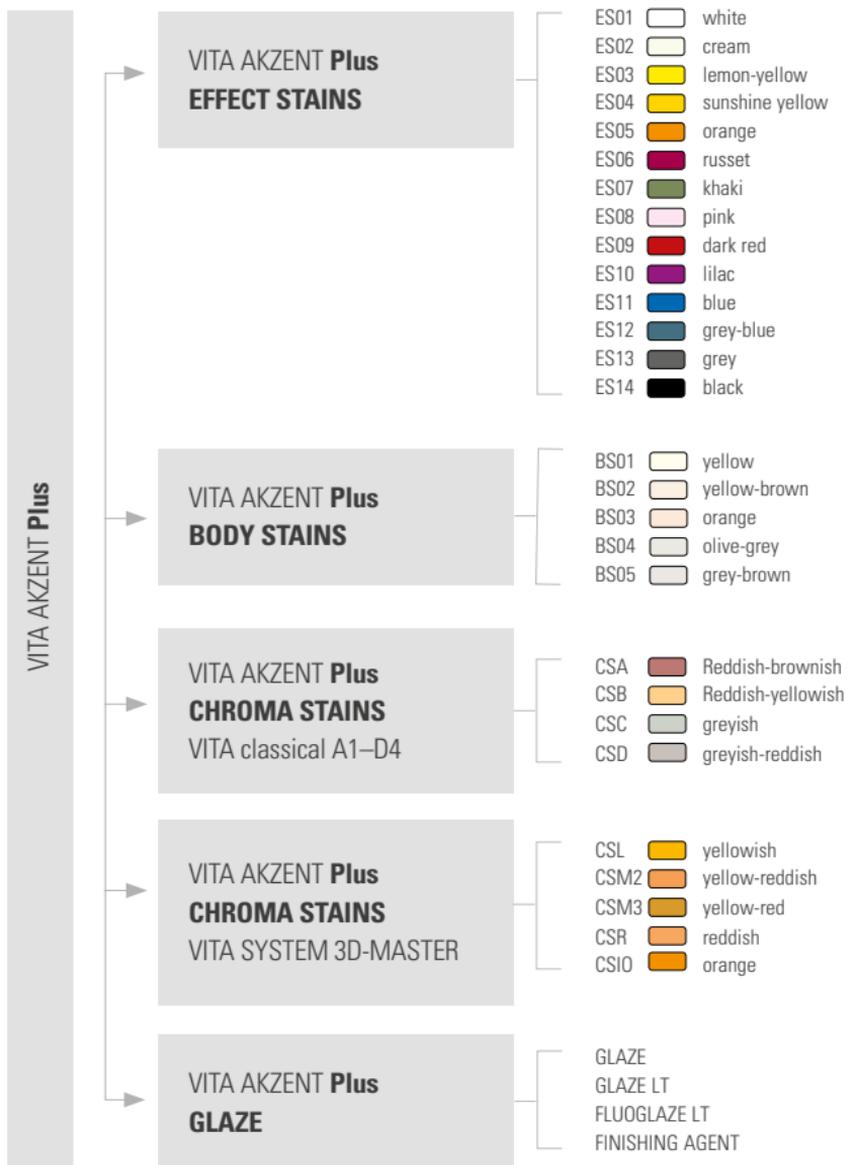



TECHNICAL HOTLINE

If you have technical questions concerning VITA product solutions, you can contact the Technical Service.

- ▶ **Phone +49 (0) 7761 / 56 22 22**
Email info@vita-zahnfabrik.com
8 a.m. to 5 p.m CET
- ▶ Additional international contact information can be found at **www.vita-zahnfabrik.com/contacts**



ONLINE SUPPORT

Questions about processing and product solutions? You can find detailed answers online in the form of **FAQs, videos and electronic tutorials.**

More detailed information can also be found in the comprehensive VITA AKZENT Instructions for Use.

- ▶ **www.vita-zahnfabrik.com**
- ▶ **www.facebook.com/vita.zahnfabrik**

CE 0124 MD Rx Only



 VITA Zahnfabrik H. Rauter GmbH & Co.KG
Spitalgasse 3 · 79713 Bad Säckingen · Germany
Tel. +49(0)7761/562-0 · Fax +49(0)7761/562-299
Hotline: Tel. +49(0)7761/562-222 · Fax +49(0)7761/562-446
www.vita-zahnfabrik.com · info@vita-zahnfabrik.com