VITA AKZENT® Plus

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VITA AKZENT® Plus

Please read the instructions for use before using the product.
VITA AKZENT Plus materials are fluorescent ceramic stain materials that can be used for internal coloring during layering, or external stain-glazing and glazing, regardless of the CTE.

Intended purpose

VITA AKZENT Plus products are auxiliary materials for porcelain / ceramic restorations for dental treatments.

Patient target group

No restrictions.

Intended user

Dental professionals only: dentists and dental technicians (Rx only).

Indication

For internal shade modifications and external characterization.

Compatible with all dental ceramic materials:

- CAD/CAM materials (such as VITABLOCS, VITA YZ SOLUTIONS)
- All-ceramics (such as VITA LUMEX AC)
- Metal ceramics (such as VITA VM 13)
- Press ceramics (such as VITA AMBRIA)

Suitable for firing temperatures from 750 °C to 950 °C.

CTE-independent.

Contraindication

None known.

Product application/control

VITA AKZENT Plus ceramic stain materials are placed unmixed in minute quantities on the unsintered veneering ceramic, or mixed in or applied to the pre-sintered veneering ceramic for surface characterization. The VITA AKZENT Plus materials are then coated with ceramic material and sintered. The firing parameters depend on the ceramic materials in use; see working instructions of the individual veneering ceramics.

Recommended product combinations

For recommended product combinations with appropriate VITA system components, corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to the instructions for use | full version.

Storage/disposal

They can be disposed of with household waste. The products labelled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data

Stain powder	
Chemical composition	Wt%
SiO ₂	45 - 75
Al ₂ O ₃	2 - 12
Na ₂ O	2 - 12
K ₂ O	2 - 12
CaO	≤ 6
B ₂ O ₃	8 - 14
BaO	≤ 2
SnO ₂	≤ 6
ZrO ₂	0 - 14
ZnO	≤ 2
TiO ₂	≤ 6
MgO	< 1
Fe ₂ O ₃	< 2
Y ₂ O ₃	< 1

Stain powder	
Chemical composition	Wt%
Cr ₂ O ₃	< 1.5
Additional components	< 1

Chemical composition	Wt%		
	Pastes	Sprays	Powders
Stain powder	37 - 67	6 - 15	100
Isobutane	-	60 - 85	-
Ethanol	-	7 - 30	-
Butanediol	23 - 45	-	-
Glycerin	7 - 15	-	-
Additional components	3 - 6,5	< 3	-

Hazard statements

VITA AKZENT Plus
GLAZE SPRAY, GLAZE LT SPRAY, FLUOGLAZE LT SPRAY
For dental applications only. Not for intraoral use.

Danger

H222 Extremely flammable aerosol.
H229 Pressurised container: May burst if heated.

Precautionary statement

VITA AKZENT Plus
GLAZE SPRAY, GLAZE LT SPRAY, FLUOGLAZE LT SPRAY
P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
P211 Do not spray on an open flame or other ignition source.
P251 Do not pierce or burn, even after use.
P410+P412 Protect from sunlight. Do not expose to temperatures exceeding 50°C/122 °F.

Safety at work and health protection

In case of formation of dust, use an extraction system or wear a face mask.

Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks, and (if applicable) short clinical safety and performance reports (SSCPs) can be found at
https://www.vita-zahnfabrik.com/product_safety

The safety data sheets can be downloaded at
www.vita-zahnfabrik.com or requested by fax at (+49) 7761-562-233.

Disclaimer

Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application.

We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product.

Date of issue of this information for Use: 2024-03

After the publication of this information for use, any previous versions become obsolete. The current version can be found under www.vita-zahnfabrik.com

Marking

VITA Zahnfabrik has been certified, and the following product bears the marks
VITA AKZENT®Plus.

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CH REP VITA Zahnfabrik H. Rauter GmbH & Co.KG, Bad Säckingen (Germany)
Zweigniederlassung Basel c/o Perrig AG, Max Kämpf-Platz 1, 4058 Basel
VITA Zahnfabrik · H. Rauter GmbH & Co. KG