VITA YZ® HT SHADE LIQUID
VITA YZ® ST SHADE LIQUID
VITA YZ® XT SHADE LIQUID
VITA YZ® EFFECT LIQUID

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Instructions for use | short version

Please read the instructions for use before using the product.

Intended purpose:

Use the corresponding VITA YZ SHADE LIQUIDs (HT/ST/XT) and EFFECT LIQUIDs for manual coloring of the milled restorations.

Patient target group:

No restrictions

Intended user

Dental professionals only: dentists and dental technicians (Rx only)

Indication:

For complete or partial coloring of VITA YZ SOLUTIONS restorations.

Contraindication:

Substructure materials with unsuitable CTE values and unsuitable material properties

Product application

Workflow:

Please observe the following workflow during processing: 1. Preparation of substructures, 2. Coloring using the brush or dipping technique, 3. Drying of substructures, 4. Sintering of substructures.

Use:

VITA YZ EFFECT LIQUID Stabilizer: After some time, the zirconia may become blotchy during coloring. In this case, add 5% of the stabilizer to the SHADE LIQUID.
 VITA YZ EFFECT SHADE LIQUID Indicator (adding shade pigments to the mixture): Add 1-2 drops of indicator to every cm of SHADE LIQUID in the bottle.

Number of brush strokes required to achieve the target shade:

Tooth shade	Color of liquid	Number of brush strokes*			EFFECT LIQUID
		Neck	Body	Incisal	
VITA classical A1-D4					
A1	A1	4	3	2	Chroma A Chroma B Chroma C Chroma D Brown Orange Blue Grey
A2	A2	4	3	2	
A3	A3	4	3	2	
A3.5	A3.5	4	3	2	
A4	A4	4	3	2	
B1	B1	4	3	2	
B2	B2	4	3	2	
B3	B3	4	3	2	
B4	B4	4	3	2	
C1	C1	4	3	2	
C2	C2	4	3	2	
C3	C3	4	3	2	
C4	C4	4	3	2	
D2	D2	4	3	2	
D3	D3	4	3	2	
D4	D4	4	3	2	
VITA SYSTEM 3D-MASTER (only available as VITA YZ HT SHADE LIQUID)					
1M1	1M1	3	2	1	Blue Grey
1M2	1M2	3	2	1	
2L1.5	2L1.5	3	2	1	
2M2	2M2	3	2	1	
3M2	3M2	3	2	1	
3M3	3M3	3	2	1	
4M2	4M2	3	2	1	

* All information applies to VITA YZ SOLUTIONS White materials. These are reference values that may vary depending on brush pressure, handling and the amount of liquid.

Pre-dry parameters

Programs	%	TO °C	↑ min.	↑ °C/min.	T1 °C	→ min.	↓ °C	%
Pre-dry	50	25	7:21	17	150	30:00	-	50

Sintering parameters

Programs	Lift position %	Start temp. °C	↑ min.	↑ °C/min.	T °C	→ min.	↓ °C	Lift position %
YZ HT Universal	100	25	83:49	17	1450	120:00	200	100
YZ ST Universal	100	25	188:08	8	1530	120:00	200	100
YZ XT Universal	100	25	356:15	4	1450	120:00	200	100

Recommended product combinations:

For recommended product combinations with the VITA system components, corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to the Instructions for Use | full version 10446.

Storage/disposal:

dark and cool: < 25°C/77 °F. Do not use the liquids after the expiration date.

The products labeled with pictograms for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as paper and plastic) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

Technical data:

Chemical composition
 VITA YZ XT, ST, HT SHADE LIQUIDs

Components	WT [%]
Water CAS No. 7732-18-5	Rest
Polyethylene glycol CAS No. 25322-68-3	< 20
Chrome (III) – nitrate nonahydrate CAS No.7789-02-8	< 1
Iron (III) – nitrate nonahydrate CAS No.7782-61-8	< 10
Erbium nitrate – hydrate CAS No. 100641-14-3	< 40
Manganese (II) – nitrate hexahydrate CAS No. 15710-66-4	< 0.5
Praseodymium(III) nitrate hexahydrate CAS No. 15878-77-0	< 1
Pigments (E102, E151, E133, E127, E110, E155)	< 1

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