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## Instructions for use EN

### VITA VIONIC BOND® bonding system

Please read the instructions for use before using the product.

VITA VIONIC BOND is a self-curing, two-component bonding system (BOND I and II) for permanently bonding VITA denture teeth from the VITA VIONIC product series in the corresponding cavities of prosthetic bases made of compatible denture base materials.

#### Intended purpose

VITA VIONIC BOND products are materials for cured denture base resin for dental treatments.

#### Patient target group

No restrictions

#### Intended user

Dental professionals only: dentists and dental technicians

#### Indication

For permanently bonding VITA denture teeth from the VITA VIONIC product series in the corresponding cavities of prosthetic bases made of compatible denture base materials; see [www.vita-zahnfabrik.com/vionibond](http://www.vita-zahnfabrik.com/vionibond).

#### Contraindication

- For bonding prefabricated teeth that are not part of the VITA VIONIC product family.
- For bonding denture teeth whose outer contour does not match the contour of the cavity.
- For patients with allergies/sensitivities to MMA.

#### Product application/control

Please adhere to the following workflow when working with this product:

- Check the fit of the prepared VITA VIONIC denture teeth within the cavities of the denture base (free of tension, but without a visible gap).
- Roughen the bonding surfaces of the denture base and the tooth elements (Al<sub>2</sub>O<sub>3</sub>, 50 µm, 2 bar / 29 psi). Remove residues with oil- and water-free compressed air.
- Measure VIONIC BOND I and II using the VIONIC BOND multidose measuring cup (empty, clean, dry, not deformed). See dosing instructions and dosing table.
- Stir mixture for 30 sec. Avoid creating bubbles / foam.
- Apply mixture to the adhesive surface of teeth and moisten cavities generously.
- Insert denture teeth into cavities and push them gently into place.
- Let rest for 30 minutes.
- Cure in a pressure pot (55°C / 131 °F, 2 bar / 29 psi, 20 min.) or by leaving at room temperature for a min. of 12 hours without pressure load.
- Remove any excess by blast-polishing (1 - 2 bar).

#### Dosing table:

Number of teeth to be bonded	Total quantity of adhesive required	Comprised of	
		Dosage VIONIC BOND I multiDose	Dosage VIONIC BOND II multiDose
1 tooth arch (max. 14 teeth)	1.0 ml = recommended minimum mixing quantity	2 graduation marks on measuring cup	1 graduation mark on measuring cup
2 tooth arches (max. 28 teeth)	2.0 ml	4 graduation marks on measuring cup	2 graduation marks on measuring cup

If more than 28 teeth are to be bonded, we recommend mixing VIONIC BOND multiDose again. The VIONIC BOND multidose measuring cup can be reused and is available separately. Detailed information on working with this product can be found in the document Instructions for Use | full version at [www.vita-zahnfabrik.com/vionibond](http://www.vita-zahnfabrik.com/vionibond).

#### Recommended product combinations

For recommended product combinations with the VITA system components and corresponding processing recommendations and restrictions regarding the combination possibilities of the product, please refer to the Instructions for Use | full version.

#### Storage/disposal

Store at a refrigerated temperature of 5°C (41°F) to 10°C (50°F) before and after opening. Store in a dry location. Protect from direct sunlight. Always keep the threads of the bottles and lids clean to ensure tight resealing. Once opened, use within 90 days.

The products labeled with a pictogram for hazardous substances are to be disposed of as hazardous waste. Recyclable waste (such as attachments, paper and plastics) must be disposed of using appropriate recycling systems. If necessary, contaminated product residues should be pretreated in accordance with regional regulations and disposed of separately.

#### Technical data

Physical properties	Value
Bond strength with denture acrylic according to ISO 22112	Complies with standard

#### Chemical composition

VIONIC BOND I multiDose	Methyl methacrylate, acrylic polymer based on methyl methacrylate, triethylene glycol dimethacrylate, dibenzoyl peroxide, other.
VIONIC BOND II multiDose	Methyl methacrylate, N,N-dimethyl-p-toluidine.

#### Safety information

When handling uncovered, use equipment with local exhaust ventilation.

#### Product reliability

Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks and (if applicable) short clinical safety and performance reports (SSCPs) can be found at [https://www.vita-zahnfabrik.com/product\\_safety](https://www.vita-zahnfabrik.com/product_safety).

#### Hazardous substances

##### IMPORTANT!

- H225 Highly flammable liquid and vapor.
- H315 Causes skin irritation.
- H317 May cause an allergic skin reaction.
- H335 May cause respiratory irritation.
- P210 Keep away from heat, hot surfaces, sparks, open flames and other sources of ignition. No smoking.
- P233 Keep the container tightly closed.
- P261 Avoid inhalation of dust / smoke / gas / fog / vapor / aerosol.
- P280 Wear protective gloves/protective clothing/eye and face protection.
- P403 Store well-sealed in an adequately ventilated location.

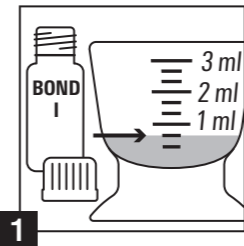
The safety data sheets can be downloaded at [www.vita-zahnfabrik.com/vionibond](http://www.vita-zahnfabrik.com/vionibond) or requested by fax at (+49) 7761-562-233.

#### Disclaimer


Please note: Our products must be used in accordance with the instructions for use. We do not accept liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not necessarily a component of the product. Date of issue of these instructions for use: 2024-10.

After the publication of this information for use, any previous versions become obsolete. The current version can be found at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com).


## Dosieranleitung / Dosing instruction / Instruction de dosage / Instrucciones de dosificación / Istruzioni per il dosaggio:

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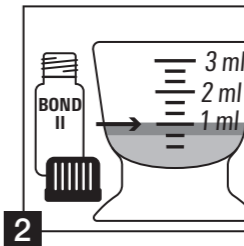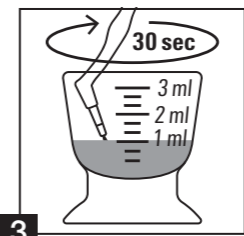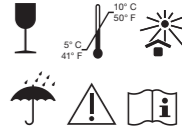
## VITA VIONIC® BOND multiDose

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### Instructions for use | short version

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