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### VITA ADIVA® OXY-PREVENT

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## Instructions for use

### VITA ADIVA® OXY-PREVENT

Please read the instructions for use before using the product.

### Product description

VITA ADIVA OXY-PREVENT is an oxygen-impermeable, translucent glycerine gel designed to prevent the oxygen inhibition layer from forming during the processing of composites. It can be removed easily with water spray after application and is also well-suited to use as a try-in paste.

### Patient target group

Persons who are treated during a dental procedure

### Indication/ Intended use

- For application to the cementation joint of all-ceramic, hybrid ceramic or composite restorations *before* the polymerization of the composite cement.
- For masking composite fillings before polymerization.
- As a try-in paste for trying in ceramic or composite restorations to evaluate the shade effect.

### Contraindication

None known.

### Performance features

The performance features of the product meet the requirements of the intended use.

### Intended users

This medical device should only be used by a professionally trained dental practitioner.

### Product application/control

Preparation of the VITA ADIVA OXY-PREVENT syringe:

Screw off the cap of the syringe and substitute it by one of the supplied single-use application cannulas (by turning clockwise).

Store the screw cap!

After use discard the application cannula and attach the cap to the syringe again. For hygienic reasons application cannulas are for single use only.

1. Application during the luting of all-ceramic, hybrid ceramic or composite restorations: Use restorations with composite cement (e.g., VITA ADIVA F-CEM). Remove excess. Apply VITA ADIVA OXY-PREVENT 1 -2 mm thick to the composite joint using an application cannula. Light-cure the entire composite cement with VITA ADIVA OXY-PREVENT. Remove VITA ADIVA OXY-PREVENT with water spray under suction. Finish and polish the restoration.
2. Use with direct composite fillings: Apply VITA ADIVA OXY-PREVENT 1 -2 mm thick to the modeled composite filling using an application cannula. Light-cure the entire composite using VITA OXY-PREVENT. Remove VITA ADIVA OXY-PREVENT with water spray under suction. Finish and polish the restoration.
3. Use as a try-in paste for trying in ceramic and composite restorations: Apply VITA ADIVA OXY-PREVENT in the cavity or on the restoration using the application cannula. Try in to evaluate the shade and fit. Remove VITA ADIVA OXY-PREVENT with water spray. Once this is done, dry the restoration with an air blower.

### Recommended product combinations

VITA ADIVA OXY-PREVENT is applied using single-use application cannulas for VITA ADIVA OXY-PREVENT.

### Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority

### Storage/disposal

Do not store above 25°C (77°F). Do not use after the expiration date.

Disposal of the product according to local authority regulations.

### Guarantee

Harvard Dental International GmbH guarantees that this product is free from material and manufacturing faults. Harvard Dental International GmbH assumes no further responsibility, as well as no implicit guarantee, relating to marketability or suitability for a specific purpose. The user is responsible for the use and proper application of the product. If defects occur on the product within the warranty period, the customer's only claim and the only obligation of Harvard Dental is in the repair or replacement of the Harvard Dental International GmbH product.

### Limitation of liability

Insofar as exclusion of liability is legally allowed, Harvard Dental International GmbH bears no liability for losses or damage from this product, regardless of whether direct, indirect, concomitant, or consequent damage is concerned, regardless of the legal basis, including guarantee, contract, negligence, or premeditation.

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VITA Zahnfabrik H. Rauter GmbH & Co.KG