

# Important!

Please follow the Sinterparameters of **VITA YZ®**:

	Pre-Heating	Holding Time	Cooling off
<b>VITA YZ®</b> MULTI TRANSLUCENT	17°C/min. to 1450°C	30 min. at 1450°C	at 100% to 200°C
<b>VITA YZ®</b> HT	17°C/min. to 1450°C	120 min. at 1450°C	at 100% to 200°C
<b>VITA YZ®</b> ST	8°C/min. to 1530°C	120 min. at 1530°C	at 100% to 200°C
<b>VITA YZ®</b> XT	4°C/min. to 1450°C	120 min. at 1450°C	at 100% to 200°C




For further information please read  
the instruction for use or go to  
[www.vita-zahnfabrik.com/cadcam](http://www.vita-zahnfabrik.com/cadcam)

# VITA

# Nesting recommendation

## VITA YZ® MULTI TRANSLUCENT

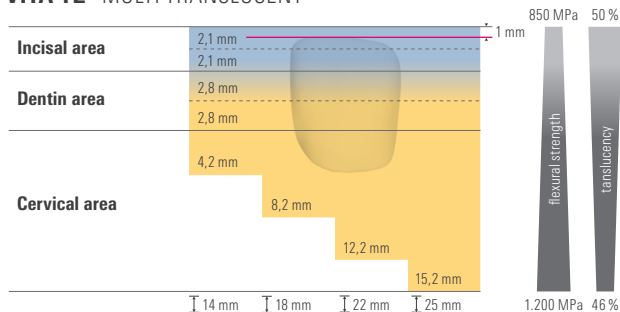

## VITA YZ® XT Multicolor / VITA YZ® ST Multicolor

