

Important!

Please follow the sintering parameters of **VITA YZ®**:

	Pre-Heating	Holding Time	Cooling off
VITA YZ® MULTI TRANSLUCENT	17°C/min. to 1450°C	30 min. at 1450°C	at 100% to 200°C
VITA YZ® HT	17°C/min. to 1450°C	120 min. at 1450°C	at 100% to 200°C
VITA YZ® ST	8°C/min. to 1530°C	120 min. at 1530°C	at 100% to 200°C
VITA YZ® XT	4°C/min. to 1450°C	120 min. at 1450°C	at 100% to 200°C




For further information please read
the instruction for use or go to
www.vita-zahnfabrik.com/cadcam

VITA

Nesting recommendation

VITA YZ® MULTI TRANSLUCENT

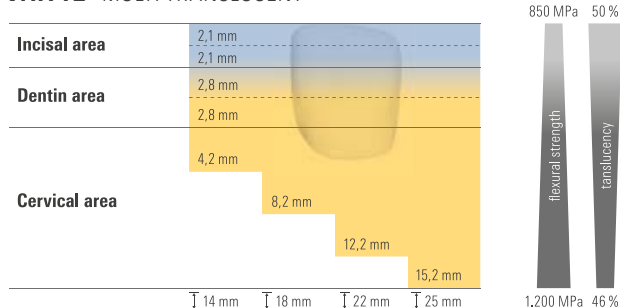

VITA YZ® XT Multicolor / VITA YZ® ST Multicolor

