

VITA

Accurate shade determination.  
**Maximum reliability.**



**VITA Easyshade® V**

Reliable shade determination on  
natural teeth, ceramic restorations  
and for bleaching treatments.





Digital tooth shade determination

# VITA Easyshade® V

Highly precise shade determination – reliable shade reproduction

One essential quality criterion for esthetic restorations is the shade match between the natural teeth and the restoration. Tooth shade determination and an exact reproduction are prerequisites for this.

**VITA Easyshade V** supports dentists and dental technicians in determining and reproducing tooth shades. With its wide range of features, it offers numerous advantages that contribute to process reliability:

- No dependence on light conditions, thanks to spectrometric shade measurement
- Efficient shade determination through a single measurement instead of incremental approximations of the correct shade
- Thanks to objective metric procedures, it can also be carried out by inexperienced dental staff or by those less adept at shade matching
- Measurement results displayed using four different shade systems:  
VITA classical A1–D4®, VITA SYSTEM 3D-MASTER®, VITA Bleachedguide 3D-MASTER® – for bleached shades, for bleaching therapy, shade coordinates (LCh or Lab values)
- Various measuring methods such as determining the mean shade and determining the tooth shade as a three-point measurement (cervical, central and incisal) for local allocation of the tooth shade
- Simplified tooth shade communication and documentation in combination with VITA Assist and VITA mobileAssist +

## Accurate

Easy and accurate shade determination in seconds. Reliable process and simple communication between practices and dental laboratories.

## Versatile

Can be used on natural teeth, as well as ceramic restorations. Efficient solution for consulting and for monitoring bleaching treatments.

## Cost effective

The time and cost savings achieved with VITA PERFECT MATCH tooth shade determination reduces reworking and dissatisfaction.



“VITA Easyshade V is an ideal tool for accurate tooth shade determination and for follow-up visits, providing confidence for us and our patients. We get clear results – for example, before and after bleaching treatments. This offers true added value in our everyday work.”

**Dr. med. dent. Iris Kraljevic**  
Prophylaxe Zentrum Zürich  
Switzerland



# Versatile.

## Which Easyshade is right for you?

While Easyshade V offers a wide range of measurement options that also support communication with the laboratory, Easyshade LITE focuses on straightforward use exclusively on natural teeth.



I would like ...	VITA Easyshade® V	VITA Easyshade® LITE
... to precisely determine the shade of <b>natural teeth</b> using VITA classical A1 – D4 or the VITA SYSTEM 3D-MASTER shade guide.	✓	✓
... to get information on shade deviations between the measured tooth and the nearest shade value.	✓	✓
<b>... a reliable tool for monitoring and for consulting with bleaching treatments, including the simple transfer of image and patient data.</b>	✓	-
... concrete recommendations as to how, if necessary, an even more accurate match to the measured tooth shade can be achieved, for example by mixing shades (in VITA SYSTEM 3D-MASTER).	✓	-
... to precisely determine <b>restorations</b> using VITA classical A1 – D4 and the VITA SYSTEM 3D-MASTER shade guides.	✓	-
<b>... to use VITA Assist Software and the VITA mobileAssist + app for tooth shade communication and documentation.</b>	✓	-
... Lab and LCH values to be displayed.	✓	-
Operation function	Measurement key, touch display	Measurement key
Use	Battery operation (plug)	2x AA batteries



## VITA Assist and VITA mobileAssist +

Professional tooth shade communication and documentation for dental practices and laboratories

The Windows-based PC software VITA Assist professionally documents and manages dental findings, as well as tooth shade information. VITA Assist allows integration of the VITA Easyshade V measurement results into digital patient photos and offers additional image editing tools, as well as a wide range of comment functions. The database allows for efficient patient case management and information exchange between practice and laboratory.



With the practical app VITA mobileAssist +, data from VITA Easyshade V is wirelessly transmitted to the app and can be combined with a patient photo and sent through email. All of the relevant tooth shade information and patient photos can be sent conveniently from a smartphone or tablet.



# Suitable solutions for processes.



## Tooth shade determination

VITA Easyshade V, VITA Easyshade LITE, VITA classical A1–D4 shade-guide with VITA Bleachedshades, VITA Bleachedguide 3D-MASTER, VITA Toothguide 3D-MASTER with Bleached Shades



## CAD/CAM

VITABLOCS, VITA YZ, VITA ENAMIC, VITA SUPRINITY PC



## Characterization

VITA AKZENT Plus stains / glaze materials, VITA AKZENT LC



## Firing

VITA VACUMAT 6100 M, VITA SMART.FIRE ADVANCED



## Polishing

VITA CERAMICS Polishing Set / VITA Polish Cera Paste



## Luting

VITA ADIVA Solutions



# We are happy to help.



## Hotline Sales Support

Phone +49 7761 562-884  
Fax +49 7761 562-299  
8 a.m. to 5 p.m. CET  
info@vita-zahnfabrik.com

## Technical Hotline

Phone +49 7761 562-222  
Fax +49 7761 562-446  
8 a.m. to 5 p.m. CET  
info@vita-zahnfabrik.com

## Hotline VITA North America

USA: +1 800-828-3839  
US Technical Support: +1 888-249-1640  
info@vitanorthamerica.com

## Hotline VITA Canada

Canada: +1 800-263-4778  
Canada Technical Support: +1 800-324-6224

## Please note:

Our products must be used according to the product brochures. We accept no liability for any damage resulting from incorrect handling or usage. The user is also obliged to check the product before use with regard to its suitability for the intended area of application. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. The VITA Modulbox is not a required component of the product. Publication of this product brochure: 2025-03. After publication of this product brochure, any previous versions become obsolete. The current version can be found under [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com)

VITA Zahnfabrik is certified, and the following products bear the mark: **VITA Easyshade® V** bears the CE mark in accordance with the EU guidelines 2014/53/EU, 2014/30/EU and 2011/65/EU. **VITA Easyshade® LITE** bears the CE mark in accordance with the EU guidelines 2014/30/EU and 2011/65/EU. The products/systems of other manufacturers mentioned in this document are registered trademarks of the respective manufacturers.

## References

### Internal studies, VITA R&D:

VITA Zahnfabrik H. Rauter GmbH & Co. KG  
Research and Development Division  
Spitalgasse 3, 79713 Bad Säckingen, Germany  
Dr. Berit Gödiker, Project Manager for VITA R&D,  
VITA Zahnfabrik, Bad Säckingen

### Detailed test data:

See technical and scientific documentation  
**VITA Easyshade® V** and **VITA Easyshade® LITE**  
Download at [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com)



For more information on **VITA Easyshade® V**  
simply scan the QR code.  
[https://www.vita-zahnfabrik.com/VITA\\_EASYSHADE\\_V](https://www.vita-zahnfabrik.com/VITA_EASYSHADE_V)





## VITA Easyshade® V

### VITA Zahnfabrik H. Rauter GmbH & Co. KG

Spitalgasse 3  
79713 Bad Säckingen  
Germany

Phone: +49 7761 562-0  
Hotline: +49 7761 562-222

info@vita-zahnfabrik.com  
www.vita-zahnfabrik.com

Follow us on  
Social Media!

