VITA LUMEX® UNIQUE

Liquid Ceramic System

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VITA LUMEX® UNIQUE

Liquid Ceramic System

Please read the instructions for use before using the product.
VITA LUMEX UNIQUE are ready-to-use dental ceramic materials in paste form for the esthetic finalization of crown and bridge frameworks made of yttrium-stabilized zirconium dioxide, lithium disilicate and feldspar ceramic. Small changes in shape and surface structure and the addition of contact points are possible.

Intended purpose
The VITA LUMEX UNIQUE pastes are ceramic materials for dental treatments.
Patient target group
No restrictions.
Intended user
Dental professionals only: dentists and dental technicians (Rx only).

Indication for use
Esthetic finalization of monolithic restorations made of (CTE-range approx. 9.0 to 10.5 x 10⁻⁶ K⁻¹):
• Zirconium dioxide
• Lithium disilicate
• Feldspar ceramics
Characterization of partial and full ceramic veneering with suitable dental veneering ceramics** on the following framework materials (CTE-range approx. 9.0 to 10.5 x 10⁻⁶ K⁻¹):
• Zirconium dioxide (partially and fully veneered e.g., with VITA LUMEX AC)
• Lithium disilicate (fully veneered e.g., with VITA LUMEX AC)
• Titanium (Grad 1-5) (fully veneered e.g., with VITA LUMEX AC)
• Feldspar ceramics (partially veneered e.g., with VITA LUMEX AC)
* According to ISO 6872, the CTE range is subject to a possible deviation of up to ± 0.5 x 10⁻⁶ K⁻¹
** to be found in document: 10887E VITA LUMEX UNIQUE - approved veneering ceramics

Contraindication
• Frameworks with unsuitable CTE values and material properties
• In patients with allergies/sensitivities to the ingredients
• In case of insufficient space

Workflow
Please observe the following workflow during processing:
Preparation:
• Lithium disilicate: Prepare according to the manufacturer’s instructions.
• Zirconium dioxide: Prepare according to the manufacturer’s instructions.
• The surface must be clean and free of grease before applying the materials.
• Clean the surface in an ultrasonic bath with distilled water and/or steam clean with a steam cleaner.
• Do not pour excess liquid out of the containers.
• After removing the paste, ensure that the respective containers are always completely closed again.
• Mix the VITA LUMEX UNIQUE pastes thoroughly with a metal-free spatula before each application.
• Always use a clean brush.

Application:
• Apply the pastes to the pre-treated surface according to their intended purpose and the desired effect
• Characterization/glaze firing (see firing table)
• Optional: Further application of pastes for characterization possible
• Characterization/glaze firing (see firing table)
• Optional: form correction and surface structure application with TEXTURE PASTE & TEXTURE PASTE GINGIVA
• Characterization/glaze firing (see firing table)

Firing Table*			
VITA LUMEX UNIQUE		Single-tooth restorations & 3-unit bridges	Wide-span bridges**
Pre-drying temperature	[°C]	400	400
Pre-drying time	[min]	08:00	08:00
Heating rate	[°C/min]	50	40
Firing temperature	[°C]	750	750
Holding time	[min]	01:00	01:00
Vacuum	[min]	07:00	08:45
Long-term cooling	[°C]	500	500

* The firing temperatures stated are only guide values. The firing results are influenced by several variables such as the brand, performance efficiency and age of the firing unit used.
** Depending on the weight of the restoration, it may be advantageous to extend the pre-drying time and/or reduce the heating rate.
Detailed information on processing and firing can be found in document 10873 ‘Instructions for use I full version’

Recommended product combinations
For recommended product combinations with the VITA system components, as well as corresponding processing recommendations and restrictions regarding the combinability of the product, please refer to the Instructions for Use I full version.

Storage/disposal
Recyclable waste (such as attachments, paper, plastics) must be disposed of via appropriate recycling systems. If necessary, contaminated product residues must be pretreated in accordance with regional regulations and disposed of separately.

Technical data		
Physical properties	Unit of measure	Value
3-point flexural strength	MPa	approx. 100
Solubility in acids	µg/cm ²	approx. 10
Coefficient of thermal expansion (CTE)	10 ⁻⁶ K ⁻¹ (25-400°C)	approx. 8,7

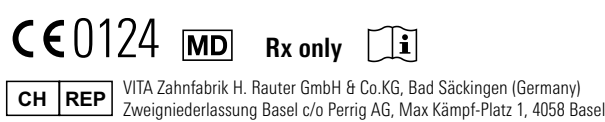
Ceramic Powder	
Chemical composition	Wt%
SiO ₂	58 – 66
Al ₂ O ₃	7 – 11
K ₂ O	6 – 9
B ₂ O ₃	5 – 9
Na ₂ O	5 – 9
CaO	1 – 5
ZrO ₂	< 5
other components	< 10

Chemical composition	Wt%	
	Ceramic	Powder
Ceramic powder	60 – 70	100
1,3-butandiol	25 – 35	–
other components	3 – 7	–

Safety Instructions
Wear suitable safety goggles/face protection, gloves and protective clothing when working with the product.

Product Safety
Information on reporting serious incidents in connection with medical devices, general risks associated with dental treatments, residual risks, and (if applicable) short clinical safety and performance reports (SSCPs) can be found at www.vita-zahnfabrik.com/product_safety. The safety data sheets can be downloaded at www.vita-zahnfabrik.com or requested by fax at (+49) 7761-562-233.

Disclaimer
Please note: Our products must be used in accordance with the instructions for use. We do not accept any liability for damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials or equipment from other manufacturers that are not compatible or not authorized for use with our product and this results in damage. Date of issue of this information: 2025-09
VITA Zahnfabrik is certified and the following products bear the mark:
VITA LUMEX® UNIQUE

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