

VITA

Quickstart Guide



VITA Easyshade® V

Perfect Shade Determination.
Maximum Reliability.





Content overview

- 1 Installation**
 - 1.1 Installing the VITA mobileAssist+ app 4
- 2 Permissions**
 - 2.1 Permissions. 5
 - 2.2 Allowing access to Bluetooth 5
 - 2.3 Allowing access to camera 6
- 3 Bluetooth**
 - 3.1 Activating Bluetooth on VITA Easyshade V 8
 - 3.2 Activating Bluetooth on an iPhone 8
- 4 Connection**
 - 4.1 Connecting VITA Easyshade V to the app 9
- 5 Transferring measurement data**
 - 5.1 Transferring measurement data. 10

1. Installation

1.1 Installing the VITA mobileAssist+ app

- Open the App Store on your iPhone or iPad.
- Search of **"VITA mobileAssist+."**
- Download and install the app.

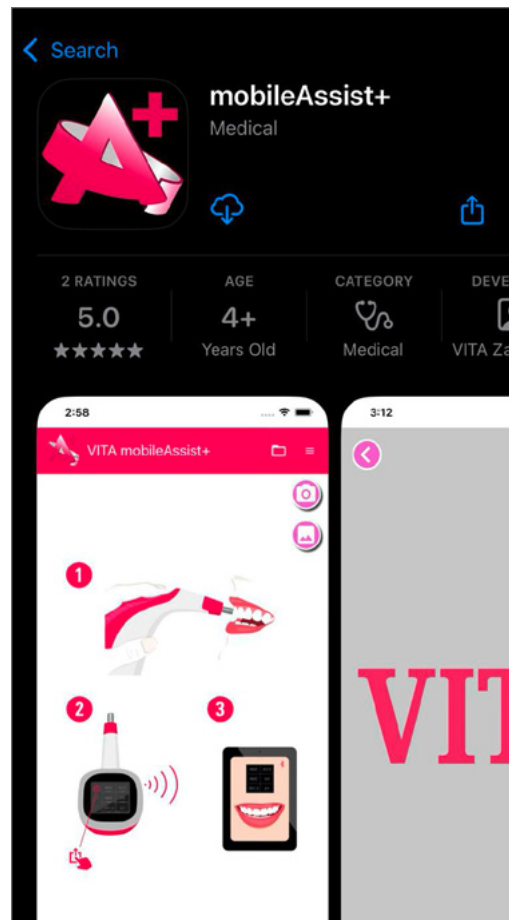


Figure 1: Installing the VITA mobileAssist+ app

2. Permissions

2.1. Permissions

The first time you start the VITA mobileAssist+ app, you will be asked for access to permissions. These permissions are required for the app to detect your VITA Easyshade V via Bluetooth and to use camera shots for shade measurements.

2.2. Allowing access to Bluetooth

- A window with the following question will only appear the first time you start the app:
"Can mobileAssist + search for Bluetooth devices?"
- **Tap on "Allow"** so that the app can find your VITA Easyshade V via Bluetooth and connect to it.

2. Permissions

2.3. Allowing access to camera

- As soon as you select the camera icon in the app, a window will appear with the question: **“mobileAssist+” wants to access your camera.”**
- Tap **“Allow”** so the app can access the camera to take photos

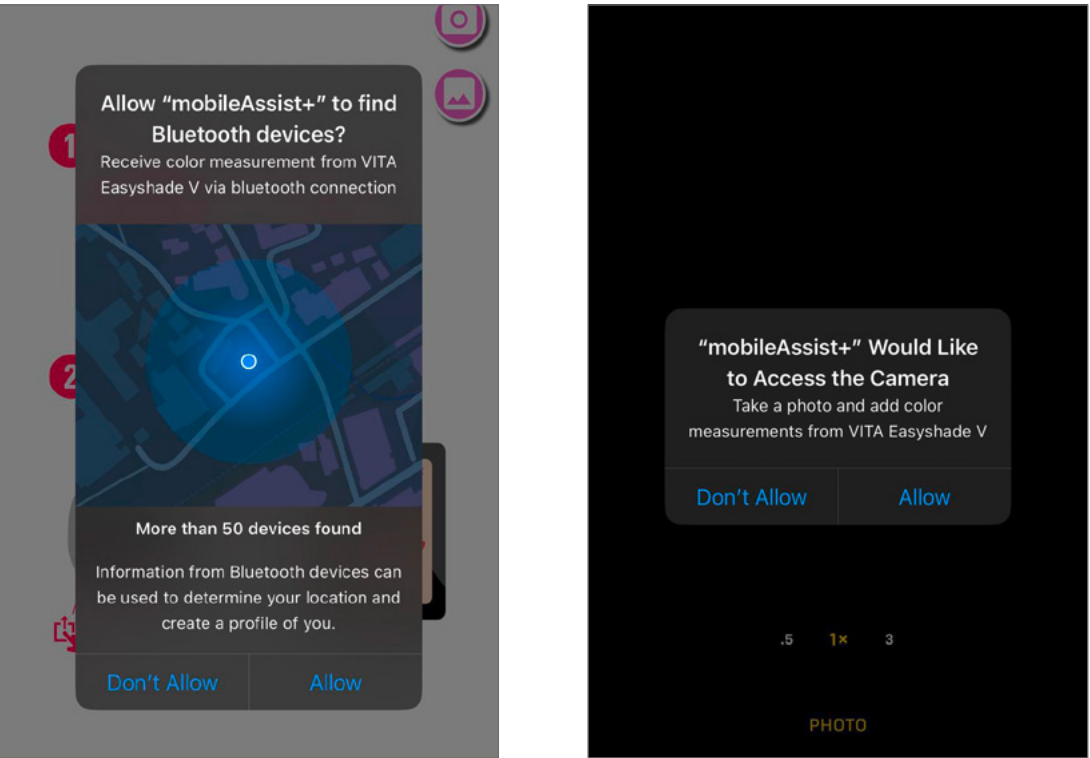


Abbildung 2: Allowing access to Bluetooth, location and camera

Note

- If you did not allow the permissions when you first started or used the camera, you can manually activate them later in the settings:
- Settings ⇒ Apps ⇒ mobileAssist+ ⇒ Activate Bluetooth.
 - Settings ⇒ Apps ⇒ mobileAssist+ ⇒ Activate camera.
- OR**
- Settings ⇒ Data protection & security ⇒ Bluetooth ⇒ Activate mobileAssist+.
 - Settings ⇒ Data protection & security ⇒ Camera ⇒ Activate mobileAssist+.

Make sure that the Bluetooth and camera permissions are activated.
Without these permissions, the app cannot communicate with your VITA Easyshade V or take pictures.

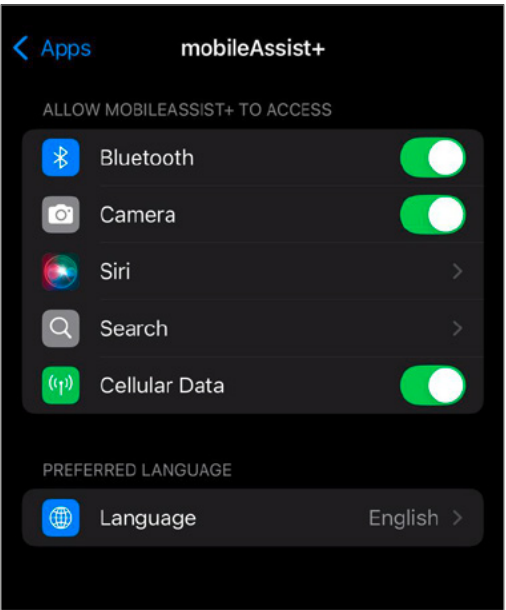


Figure 3: Manual activation

3. Bluetooth

3.1. Activating Bluetooth on VITA Easyshade V

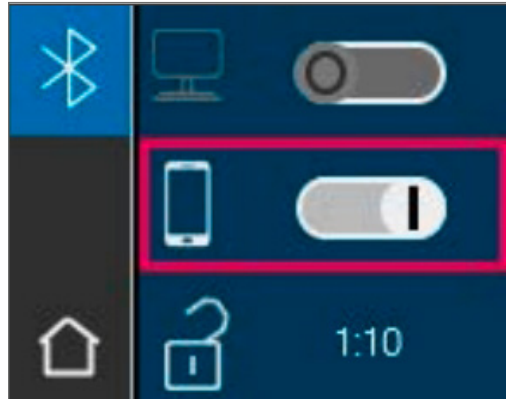


Figure 4: Activating Bluetooth on VITA Easyshade V

3.2. Activating Bluetooth on an iPhone



Figure 5: Activating Bluetooth on an iPhone

- Open the settings on your iPhone.
- Go to Bluetooth and make sure that it is switched on.

4. Connection

4.1. Connecting VITA Easyshade V to the app

- Start the VITA mobileAssist+ app.
- Tap on the VITA Easyshade V image at the bottom.
- Select your device from the list. ⇒ You can find the serial number on the underside of the charging station.
- Wait until the connection is established.


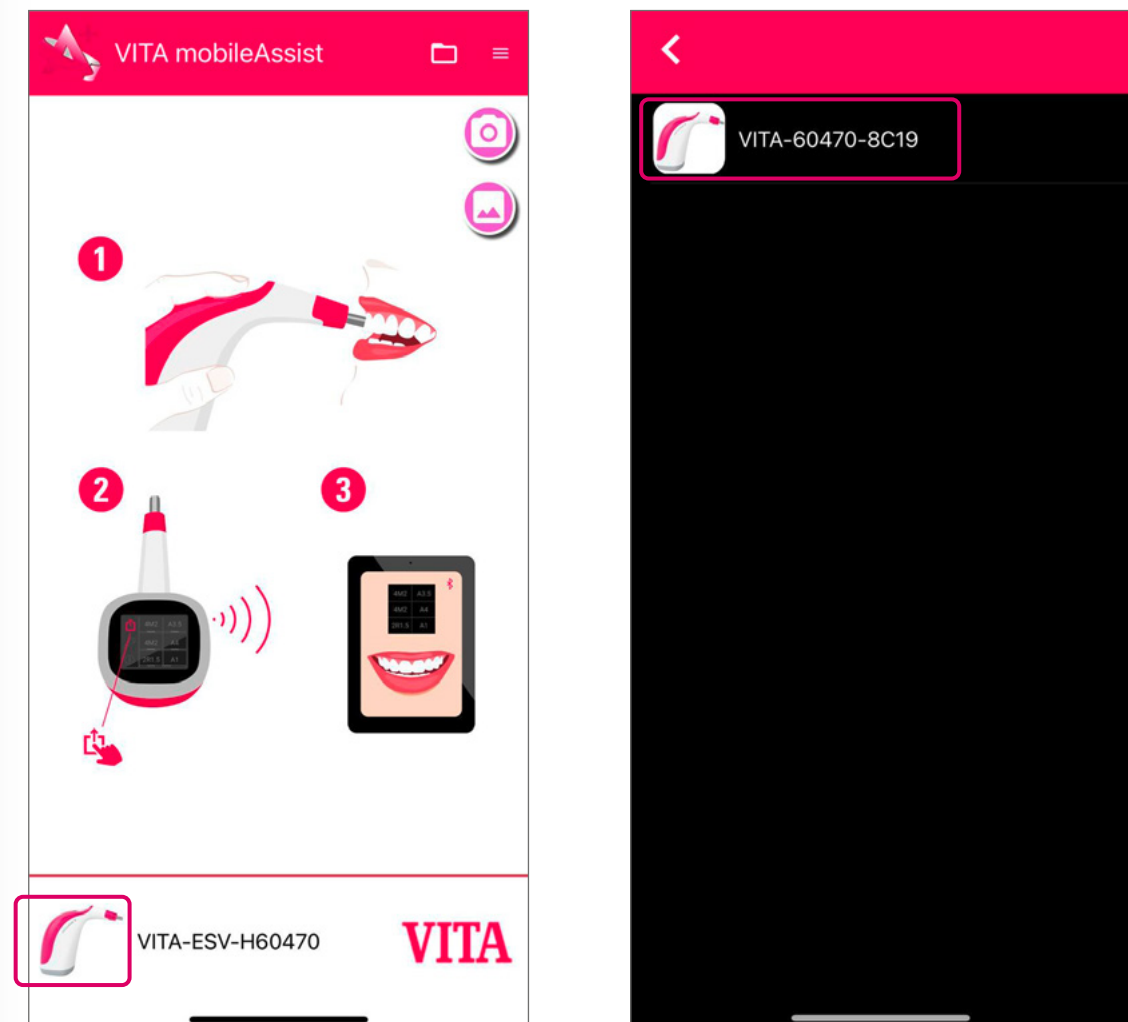


Figure 6 : Connecting VITA Easyshade V to the app

5. Transferring measurement data

5.1. Transferring measurement data

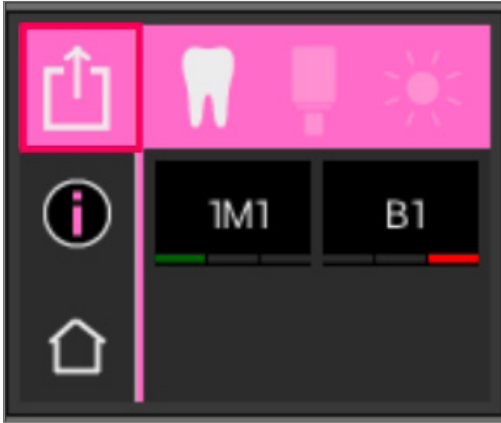

- Take a shade measurement or tap the measurement result symbol
- The share icon indicates an active connection.
- **Tap** the share icon to transfer measurement data

Figure 7: Transferring measurement data

Note

If an iOS or Android device that has been previously paired to VITA Easyshade V is still within Bluetooth range, this may affect the connection with a new device.

iOS: Settings ⇒ Bluetooth ⇒ Find VITA Easyshade in the “My devices” list ⇒ Tap the (i) icon ⇒ Select “Ignore this device” ⇒ Deactivate Bluetooth briefly, then activate it again ⇒ Try pairing with the new device again

Android: Settings ⇒ Bluetooth ⇒ Find VITA Easyshade in the “Paired devices” list ⇒ Remove device or select “Forget” ⇒ Deactivate Bluetooth briefly, then activate it again or restart the device ⇒ Try pairing with the new device again

VITA Zahnfabrik H. Rauter GmbH & Co. KG

Spitalgasse 3
79713 Bad Säckingen
Germany

Phone: +49 7761 562-0
Hotline: +49 7761 562-222

info@vita-zahnfabrik.com
www.vita-zahnfabrik.com

Follow us on
Social Media!

