VITA classical A1 – D4®

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Instructions for Use

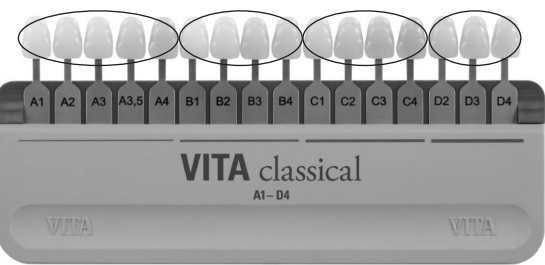
VITA classical A1 – D4®

Please read the instructions for use before using the product.

Structure

The VITA classical shade family is divided into the following categories:

A1 . A2 . A3 . A3,5 . A4	B1 . B2 . B3 . B4	C1 . C2 . C3 . C4	D2 . D3 . D4
Reddish-brownish	Reddish-yellowish	Grayish	Reddish-gray

Procedure

- The shade guide should be held directly against the patient's tooth with your arm bent (at a distance of approx. 25-30 cm).
- Decide quickly on the shade by comparing the shade samples of groups A – D with the natural tooth.
- Disinfect the shade guide after each shade determination.

Cleaning and disinfection

For hygienic care, we recommend using **mild spray disinfectants or disinfectant wipes that are commercially available**. According to the manufacturer, these products are effective against HIV and HBV pathogens and are available from most dental depots. Please follow the **manufacturer's instructions for proper disinfection**.

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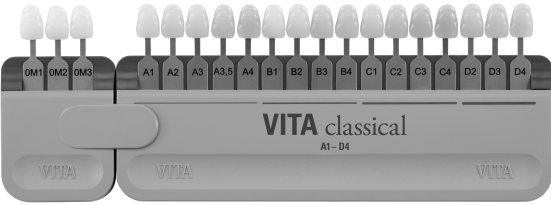
Important information on product safety

Only use **mild, non-aggressive disinfectants**.

Cleaning agents containing the following ingredients are **not suitable**:

Chlorine or chlorinated substances
Phenol or phenylphenol-containing compounds
Methyl ethyl ketone

Sterilization

The entire shade guide can be sterilized in an autoclave if required – up to a temperature of **132 °C / 270 °F**.

Storage

Protect the shade guide from intense sunlight to prevent material changes and colour distortion. After cleaning and disinfecting, ensure that the shade guide and all its components are completely dry. Please ensure that no moisture remains in cavities or areas that are difficult to access. If necessary, use suitable drying methods to ensure that the instruments are completely dry before storing them. Residual moisture can impair functionality and increase the risk of microbial contamination.

VITA BLEACHED SHADES

With the additional VITA Bleached Shades, you can correctly determine the shades of bleached teeth. The bleached shades OM1, OM2, OM3 are derived from the VITA 3D-Master system. If you have opted for the VITA Bleached Shades as a retrofit set, attach the adapter to the left side of the VITA classical A1–D4 shade guide.

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H. Rauter GmbH & Co.KG