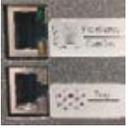


## Single furnace operation

Image	Firing units	Serial number	 <b>VITA vPad excellence</b> serial number > 2820170051 <b>VITA vPad PRO</b> serial number > 3020180051	 <b>VITA vPad comfort</b> serial number > 3020180051 <b>VITA vPad LITE</b> serial number > 3720250049
	<b>VITA VACUMAT 6000 M</b> (DV6000Mxxx)	< 1320178853	✓ Connect vPad power injector cable (D63100) to VITA vPad excellence and to the upper network socket of the furnace.	✓ Connect vPad power injector cable (D63100) to VITA vPad comfort and to the upper network socket of the furnace.
	<b>VITA VACUMAT 6000 MP</b> (DV6000MPxxx)	< 1820172400		
	<b>VITA ZYRCOMAT 6000 MS</b> (DZY6000MSxxx)	< 2120170987		
	<b>VITA VACUMAT 6000 M</b> (DV6000Mxxx)	> 1320178854	✓ Connect VITA vPad excellence to the lower network socket of the furnace.	✓ Connect VITA vPad comfort to the lower network socket of the furnace.
	<b>VITA VACUMAT 6000 MP</b> (DV6000MPxxx)	> 1820172401		
	<b>VITA ZYRCOMAT 6100 MS</b> (DZY6100MSxxx)	> 3120190049	✓	✓
	<b>VITA VACUMAT 6100 M</b> (DV6100Mxxx)	> 3520240049		
	<b>VITA SMART.FIRE</b> (DSFxxx)	> 272022xxxx		
	<b>VITA SMART.FIRE ADVANCED</b> (DSFAxxx)	> 3420240049	✓	✓ Only possible with VITA vPad LITE

## Multiple furnace operation

(Devices manufactured before 2017 and after 2017 not combined)

Image	Firing units	Serial number	 <b>VITA SWITCHBOX I</b> (D47009)	 <b>VITA SWITCHBOX II*</b> (D65000)	 <b>VITA SWITCHBOX I</b> (D47009)	 <b>VITA SWITCHBOX II*</b> (D65000)
	<b>VITA VACUMAT 6000 M</b> (DV6000Mxxx)	< 1320178853	✓ Connect VITA vPad excellence to VITA SWITCHBOX I. Connect VITA SWITCHBOX I to the upper network socket of the furnace. Connect power injector cable (D63100) to VITA SWITCHBOX I.	✓ Connect VITA vPad excellence to VITA SWITCHBOX II. Connect VITA SWITCHBOX II to the upper network socket of the furnace. Connect power injector cable (D63100) to VITA SWITCHBOX II.	✗	✗
	<b>VITA VACUMAT 6000 MP</b> (DV6000MPxxx)	< 1820172400				
	<b>VITA ZYRCOMAT 6000 MS</b> (DZY6000MSxxx)	< 2120170987				
	<b>VITA VACUMAT 6000 M</b> (DV6000Mxxx)	> 1320178854	✓ Connect VITA vPad excellence to VITA SWITCHBOX I. Connect VITA SWITCHBOX I to the upper network socket of the furnace. Connect power injector cable (D63100) to VITA SWITCHBOX I.	✓ Connect VITA vPad excellence to VITA SWITCHBOX II. Connect VITA SWITCHBOX II to the lower network socket of the furnace.	✗	✗
	<b>VITA VACUMAT 6000 MP</b> (DV6000MPxxx)	> 1820172401				
	<b>VITA VACUMAT 6000 M</b> (DV6000Mxxx)	> 1320178854	✗	✓	✗	✗
	<b>VITA VACUMAT 6000 MP</b> (DV6000MPxxx)	> 1820172401				
	<b>VITA ZYRCOMAT 6100 MS</b> (DZY6100MSxxx)	> 3120190049				
	<b>VITA VACUMAT 6100 M</b> (DV6100Mxxx)	> 3520240049				
	<b>VITA ZYRCOMAT 6100 MS</b> (DZY6100MSxxx)	> 3120190049	✗	✓	✗	✗
	<b>VITA SMART.FIRE</b> (DSFxxx)	> 272022xxxx				
	<b>VITA SMART.FIRE ADVANCED</b> (DSFAxxx)	> 3420240049				

  
**VITA vPad excellence**  
 serial number > 2820170051  
**VITA vPad PRO**  
 serial number > 3720250049

  
**VITA vPad comfort**  
 serial number > 3020180051  
**VITA vPad LITE**  
 serial number > 3620250049

\* The device at port 1 always has to be switched on.

# Mixed multi furnace operation

(Devices manufactured before and after 2017 combined)



**VITA vPad excellence**  
serial number > 2820170051



**VITA vPad comfort**  
serial number > 3020180051



**VITA vPad excellence**  
serial number > 2820170051  
**VITA vPad PRO**  
serial number > 3720250049



**VITA vPad comfort**  
serial number > 3020180051  
**VITA vPad LITE**  
serial number > 3620250049

Image	Firing units	Serial number	VITA SWITCHBOX I (D47009)	VITA SWITCHBOX II* (D65000)	VITA SWITCHBOX I (D47009)	VITA SWITCHBOX II* (D65000)	VITA SWITCHBOX I (D47009)	VITA SWITCHBOX II* (D65000)	VITA SWITCHBOX I (D47009)	VITA SWITCHBOX II* (D65000)
	<b>VITA VACUMAT 6000 M</b> (DV6000Mxxx)	< 1320178853	✓ Connect VITA vPad excellence to VITA SWITCHBOX I.	✓ Connect VITA vPad comfort to VITA SWITCHBOX II.	✓ Connect VITA vPad comfort to VITA SWITCHBOX I.	✓ Connect VITA vPad comfort to VITA SWITCHBOX II.	✓ Connect VITA vPad excellence to VITA SWITCHBOX I. Connect VITA SWITCHBOX I to the upper network socket of the furnace.	✓ Connect VITA vPad excellence to VITA SWITCHBOX II. Connect the upper network socket of the furnace to ports 2-4 of the VITA SWITCHBOX II.	✗	✗
	<b>VITA VACUMAT 6000 MP</b> (DV6000MPxxx)	< 1820172400								
	<b>VITA ZYRCOMAT 6000 MS</b> (DZY6000MSxxx)	< 2120170987	Connect VITA SWITCHBOX I to the upper network socket of the furnace.	Connect VITA SWITCHBOX II to the upper network socket of the furnace.	Connect VITA SWITCHBOX I to the upper network socket of the furnace.	Connect VITA SWITCHBOX II to the upper network socket of the furnace.	Connect power injector cable (D63100) to VITA SWITCHBOX I.	Connect the lower network socket of the furnace to port 1 of the VITA SWITCHBOX II.	✗	✗
	<b>VITA VACUMAT 6000 M/MP</b> (DV6000Mxxx/DV6000MPxxx)	> 1320178854 > 1820172401								
	<b>VITA VACUMAT 6000 M</b> (DV6000Mxxx)	< 1320178853	✗	✗	✗	✗	✗	✓ Connect VITA vPad excellence to VITA SWITCHBOX II. Connect the upper network socket of the furnace to ports 2-4 of the VITA SWITCHBOX II.	✗	✗
	<b>VITA VACUMAT 6000 MP</b> (DV6000MPxxx)	< 1820172400								
	<b>VITA ZYRCOMAT 6000 MS</b> (DZY6000MSxxx)	< 2120170987	✗	✗	✗	✗	✗	Connect the furnace to port 1 of the VITA SWITCHBOX II.	✗	✗
	<b>VITA ZYRCOMAT 6100 MS</b> (DZY61000MSxxx)	> 3120190049								
	<b>VITA VACUMAT 6100 M</b> (DV6100Mxxx)	> 3520240049	✗	✗	✗	✗	✗	Connect VITA vPad excellence to VITA SWITCHBOX II. Connect the upper network socket of the furnace to ports 2-4 of the VITA SWITCHBOX II.	✗	✗
	<b>VITA ZYRCOMAT 6000 MS</b> (DZY6000MSxxx)	< 2120170987								
	<b>VITA SMART.FIRE</b> (DSFxxx)	> 272022xxxx	✗	✗	✗	✗	✗	Connect the lower network socket of the furnace to port 1 of the VITA SWITCHBOX II.	✗	✗
	<b>VITA ZYRCOMAT 6100 MS</b> (DZY6100MSxxx)	> 3120190049								
	<b>VITA SMART.FIRE ADVANCED</b> (DSFAxxx)	> 3420240049	✗	✗	✗	✗	✗	Connect VITA vPad excellence to VITA SWITCHBOX II. Connect the upper network socket of the furnace to ports 2-4 of the VITA SWITCHBOX II. Connect the lower network socket of the furnace to port 1 of the VITA SWITCHBOX II.	✗	✗
	<b>VITA ZYRCOMAT 6000 MS</b> (DZY6000MSxxx)	< 2120170987								

\* The device at port 1 always has to be switched on.