VITA ZETA HLC BOND rosa

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D

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Instructions for use

GB

Before using, please read the instructions for use.
VITA ZETA HLC BOND pink works as a bonding agent and opaquer as part of a metal-acrylic bond. VITA ZETA HLC BOND pink is a system based on the principle of adhesive bonding.
An additional primer is not required.

Indication:

VITA ZETA HLC BOND pink is used for covering retentions or denture saddles made from metal, e.g. using the investment casting technique.

Preparation of the metal surface:

- Casting objects are prepared using cross-cut milling tools in accordance with the specifications of the alloy manufacturer.
- Surfaces that are not to be covered are rubber-coated and polished.
- Depending on the type of alloy, the surfaces that are to be covered must be sand-blasted using 110-250 µm aluminum oxide at a pressure of 3-3.5 bar.
- Insufficient sanding results in inadequate bonding.
- Clean the casting object using dry compressed air (with a water separator) or a clean “dust brush”.
- It is essential to avoid contact with water!
- If the surface becomes contaminated through contact with the skin, acetone can be used to remove oil.

Mixing:

- HLC BOND pink must be mixed and applied within 10 minutes following sand-blasting of the metal surface.
- Place a large measuring spoon (0.3 ml) of HLC BOND pink powder on a mixing pad and crush it.
- Mix one graduation line of a corresponding quantity of ZETA HLC BOND LIQUID using a plastic spatula.
- The consistency resulting from mixing is considered recommended. When mixed to a thicker or thinner consistency, ZETA HLC BOND pink requires different drying times.
- Do not use metal instruments for mixing!

Application:

- Cover the metal surface by applying ZETA HLC BOND pink with a brush and then dry.

VITA VM LC CLEANER (prod. no. CVMLC50) can be used to clean the instruments.

Drying:

HLC BOND pink with an infrared lamp (100-150 W): 10-15 minutes, Distance object – device: 5-10 cm.

Note:

Excessive temperatures may cause bubbles to form in HLC BOND.

Delivery form:

ZETA HLC BOND LIQUID, disposable syringe, 5 ml
ZETA HLC BOND COLOR C012 pink, bottle, 4 g

Note:

The safety data sheet can be downloaded from www.vita-zahnfabrik.com or requested by fax from (+49) 7761-562-233.

Warning

VITA ZETA HLC BOND LIQUID contains ethylene glycol dimethacrylate

- May cause an allergic skin reaction.
- May cause respiratory irritation.
- When working with the product, wear suitable safety goggles/gloves and safety clothing.
- Work under an extraction unit.

Storage:

Tightly closed, do not store above 25 °C.
Do not expose to direct sunlight.

Disposal considerations:

Recommendation: Disposal must be made according to official regulations.

Please note:

We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product.

As a reliable alternative that saves you time, we recommend the use of light-curing VITA VM LC GINGIVA OPAQUE PASTE. This is ideally suited to covering retentions in partial prosthetics – without smearing! Further information: www.vita-zahnfabrik.com

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VITA Zahnfabrik H. Rauter GmbH & Co.KG