VITA ENAMIC®

Kurzinformation

Stand: 11.13

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VITA

Software requirements

- For the **CEREC software > 4.0 and < 4.2**, you can visit the Sirona website at www.sirona.com/en/CERECdownloads to download a **VITA Material Service Pack CEREC SW 4.0.x**
- For the **inLab software > 4.0 and < 4.2**, you can visit the Sirona website at www.sirona.com/en/inLabdownloads to download a **VITA Material Service Pack CEREC SW 4.0.x**

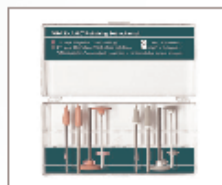
Polishing

To allow VITA ENAMIC to be effectively polished, special 2-step polishing products were developed.

Perform the high-gloss final polish with the lowest possible rotational speed and without using water for cooling.

Product no. EENPSETC/VITA ENAMIC
Polishing Set clinical/for the angle
section

Product no. EENPSETT/VITA ENAMIC
Polishing Set technical/for the
handpiece

Each material service pack adds the VITA ENAMIC hybrid ceramic to the material menu and implements the block size and special milling parameters for this material.

In order to also mill the material with the small CEREC / inLab milling system and with older software versions < 4.0, select VITABLOCS Mark II, I 14 resp. I 10.

Dentatec milling additive, in relation to a full tank

1. CEREC and inLab/50 ml
2. CEREC and inLab MC XL/75 ml

Shade characterization

The specially-developed VITA ENAMIC STAINS KIT with 6 shades and accessories is provided for shade characterization (staining) of VITA ENAMIC restorations. The corresponding working instructions should be followed.

Note:

Under no circumstances should restorations made of VITA ENAMIC be fired during processing. A polymerization process is used for characterizing and individualizing the shade.

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facebook.com/vita.zahnfabrik

Further detailed instructions for use (working instructions, etc.) are available from www.vita-enamic.com.

Thank you for your purchase – and for trusting VITA ENAMIC!

VITA Zahnfabrik is certified according to the Medical Device Directive and the following product bears the CE mark **CE 0124: VITA ENAMIC®**

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VITA Zahnfabrik H. Rauter GmbH & Co. KG