## VITA Firing Paste

D

### Indikation

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GB

### Indications

VITA Firing Paste is a ready-to-use, refractory paste for the manufacture of individual firing trays for the all-ceramic and metal-ceramic technique. The material enables objects to be attached securely on the firing trays, stabilizes substructure-free press ceramic restorations and at the same time optimizes heat distribution during firing. VITA Firing Paste can be easily removed again on completion of the firing procedure.

### Hazard Warnings / Safety Advice

- Classification of the fiber according to the EC Directive 97/69/EEC: Carc. Cat 2
- Can cause cancer when inhaling
- Prevent dust release, do not blow on with compressed air
- When mechanically processing the hardened paste, use a local extraction unit or wear a particle-filtering respiratory protective half mask
- Irritating to the skin
- Avoid contact with eyes and skin

- Do not eat, drink, smoke when working
- Keep out of the reach of children

The corresponding safety data sheet can be downloaded from [www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) or requested via fax from +49(0)7761/562-233.

### Use

#### Crowns and bridges:

Apply VITA Firing Paste completely, directly from the syringe into the interior of the restoration with some excess and place carefully onto the firing tray.

#### Inlays, Onlays, Veneers:

Apply VITA Firing Paste directly onto fibrous pad or a firing tray. Then apply to the restoration and place the latter onto the firing tray.

Close syringe immediately after use in order to prevent drying out.

After the firing procedure remove the hardened paste from the object with an instrument. **Do not sandblast!**

**Important:** VITA Firing Paste contains aluminium silicate fibers. When removing the hardened material, wear a face mask and work under an extraction unit.

### Storage

Store syringe closed at room temperature (12 - 28 °C). Protect from direct sunlight.

### Marking

The batch no. (LOT) is displayed visibly on the container.

### Shelf life

The expiry date (M) is displayed visibly on the container.

### Supply schedule

VITA Firing Paste, normal package with 1 syringe of 12 ml, large package with 3 syringes of 12 ml.

The product was developed for dental use and must be processed according to the instructions for use. Information and illustrations contained therein are not an assurance of properties and are not binding. Date of issue of this information 10.2014.

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VITA Zahnfabrik H. Rauter GmbH & Co. KG