



# Manual for downloading, copying to memorstick and installing at furnace


## 1.) Download of Software

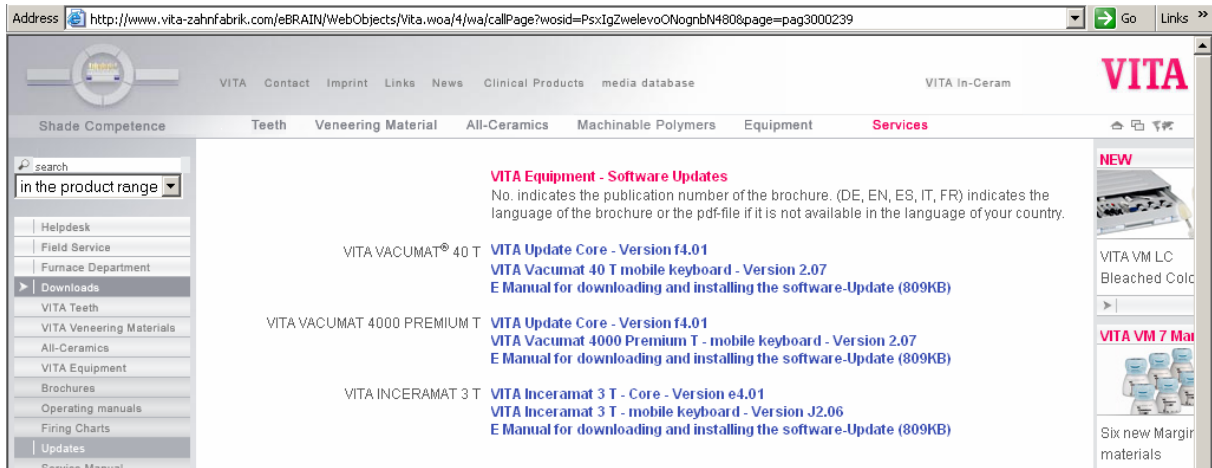


- select needed software

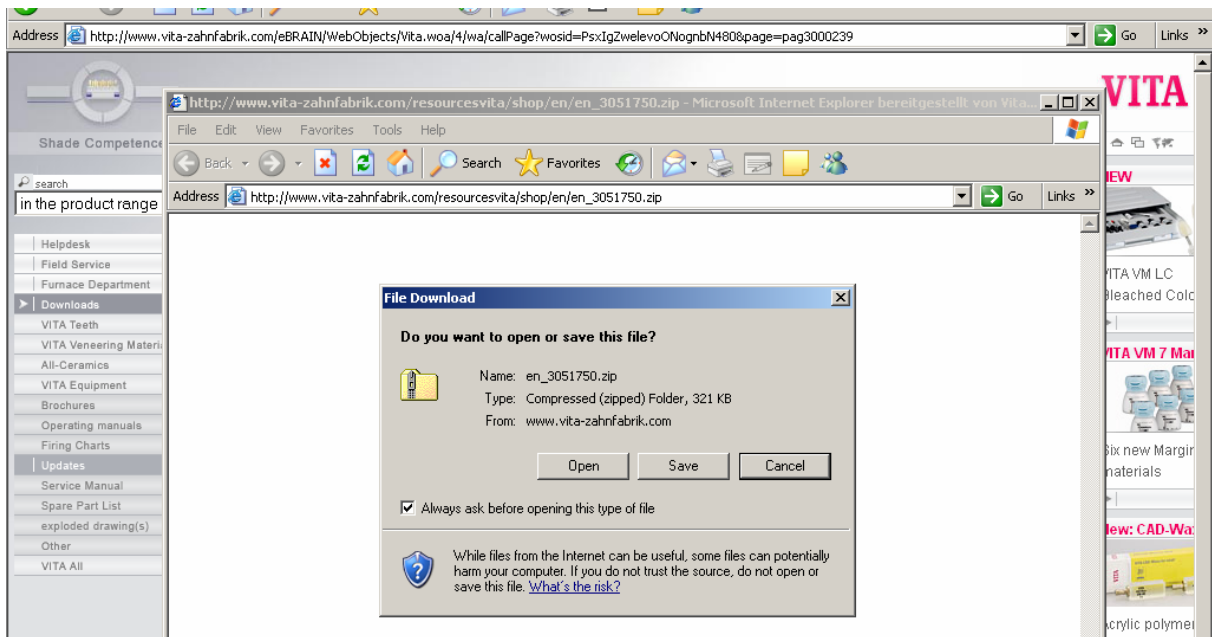


- choose „save“

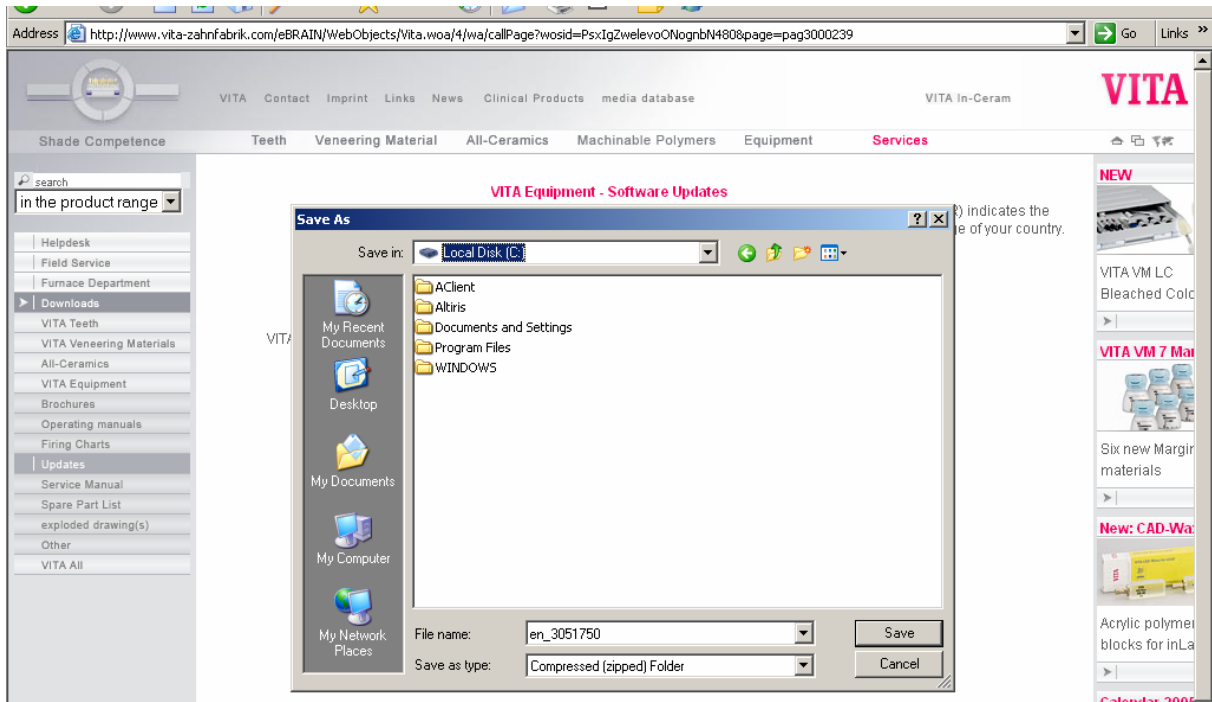


- select location and confirm „save“

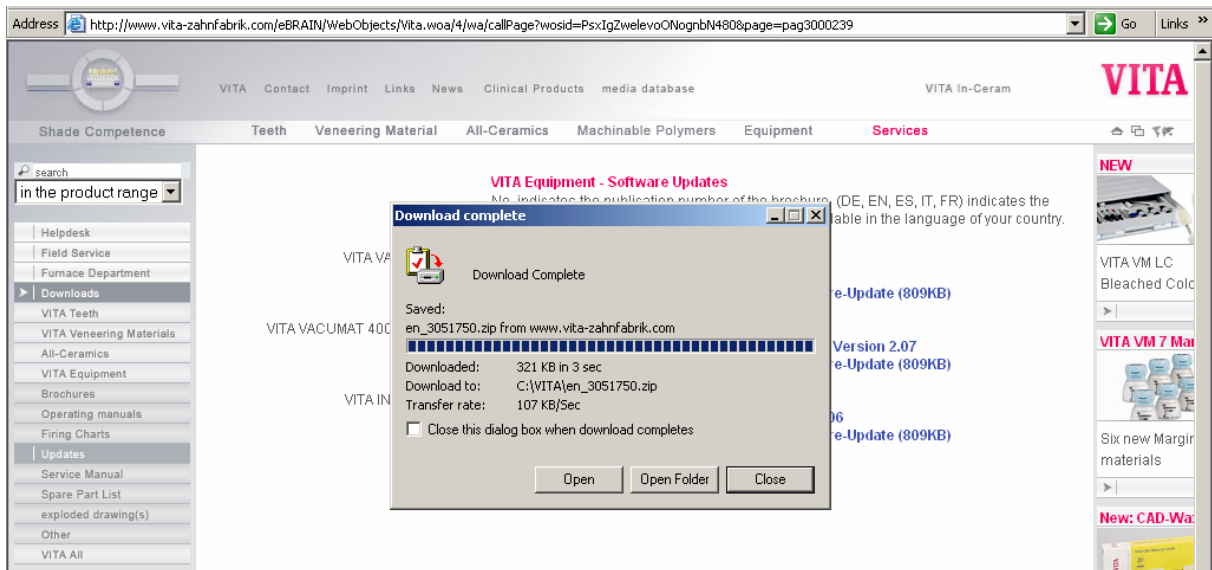


- choose „open“

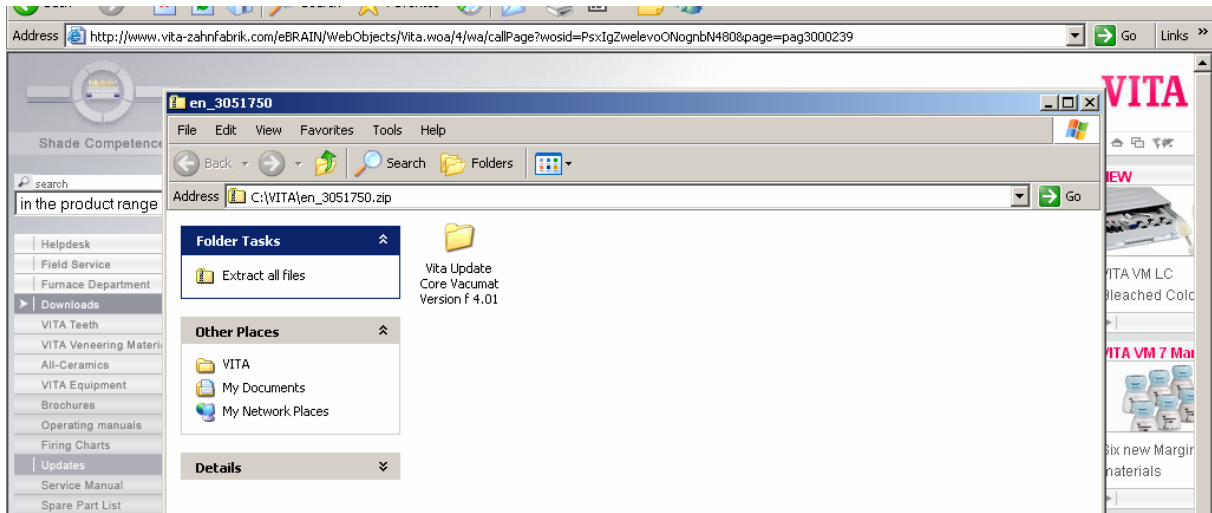


- double-click to folder

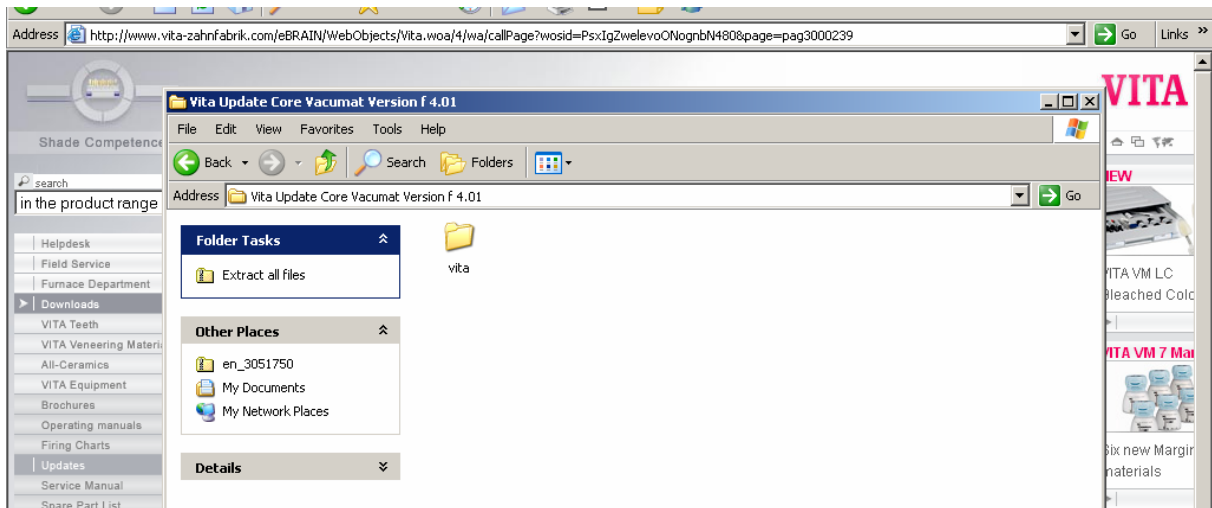


- assign folder „vita“, press right mouse button and choose „copy“

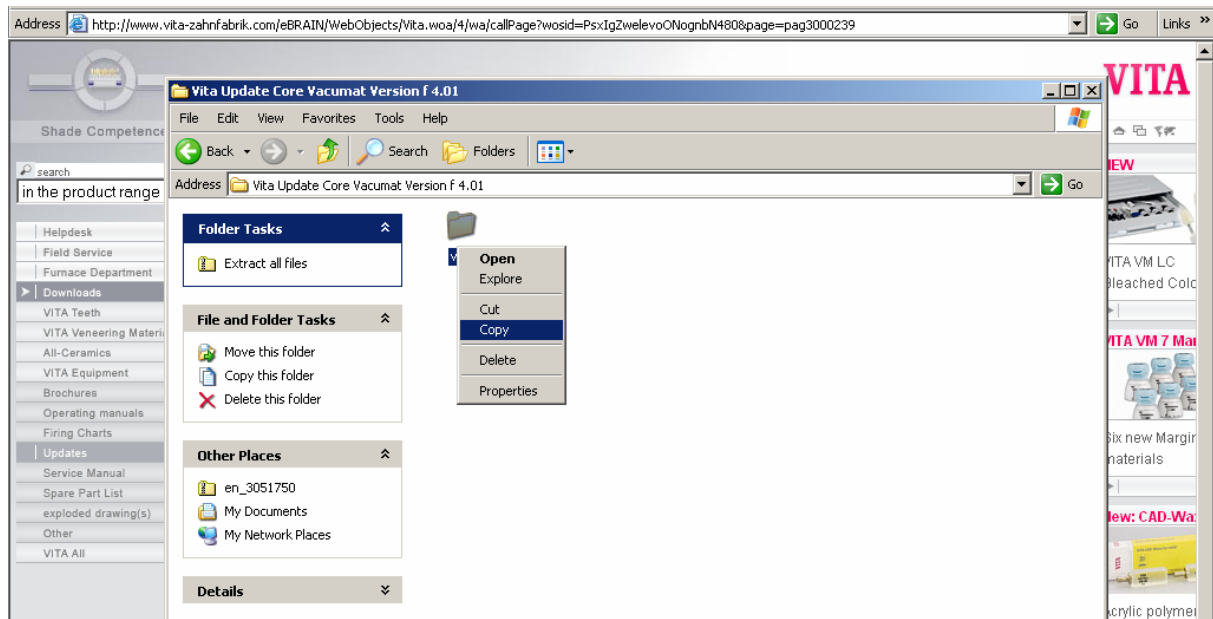


- open explorer

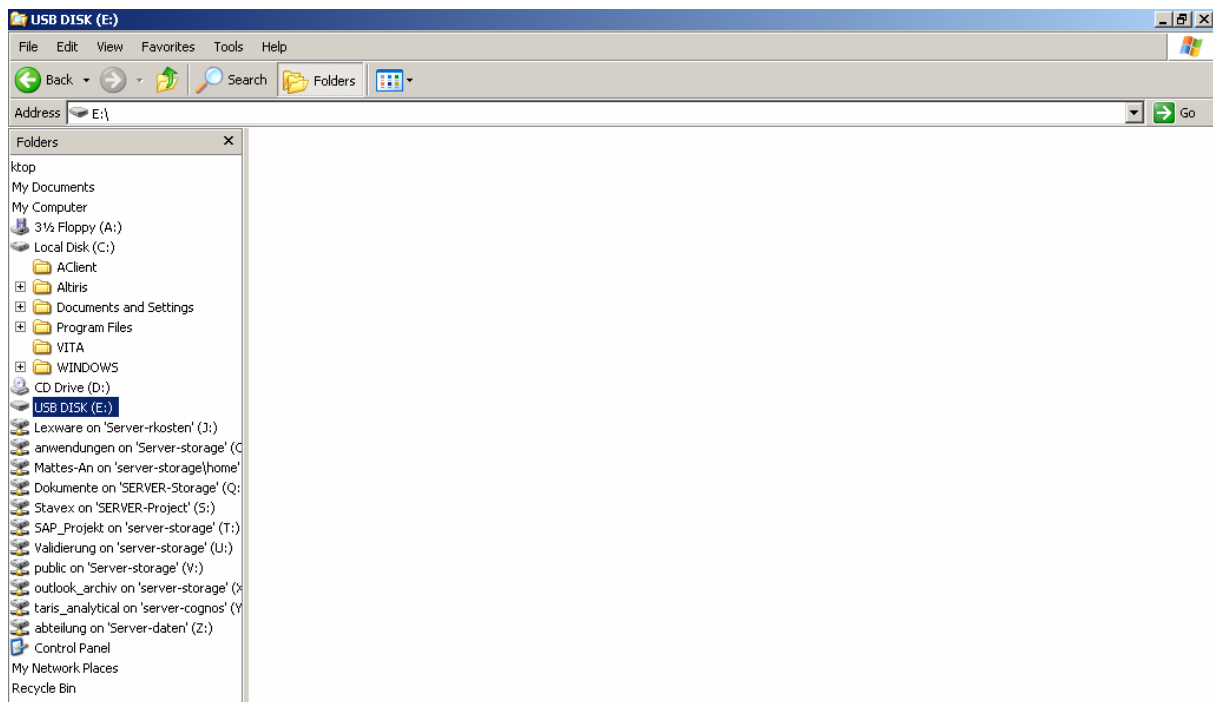


- select „removable medium“ (USB-Stick)

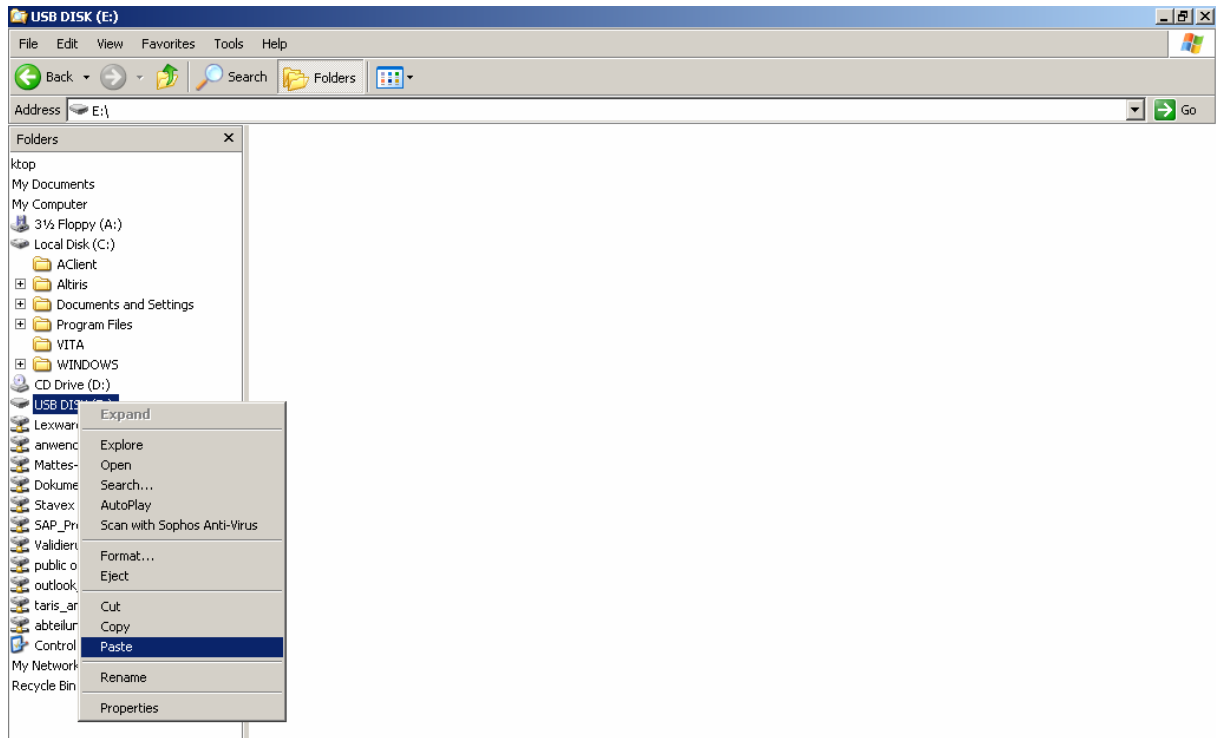

- press right mouse button and choose „paste“
- transaction of copying will be shown

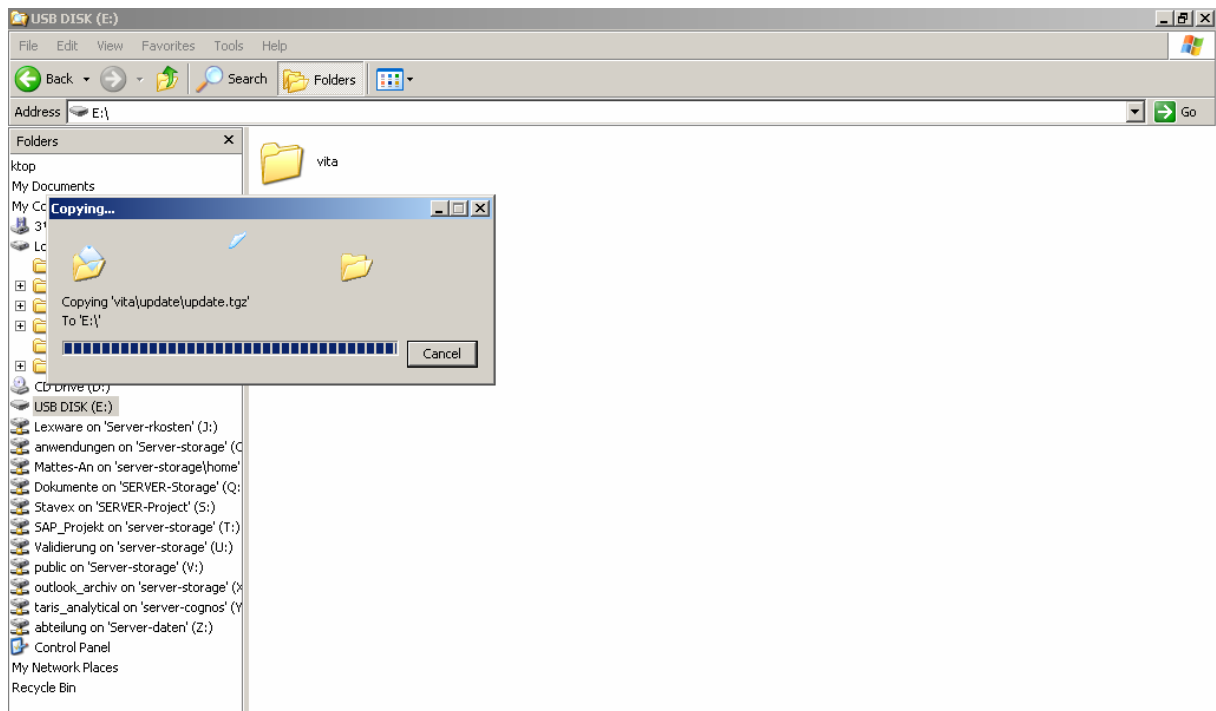

- folder „vita“ was copied to memorstick (removable medium)

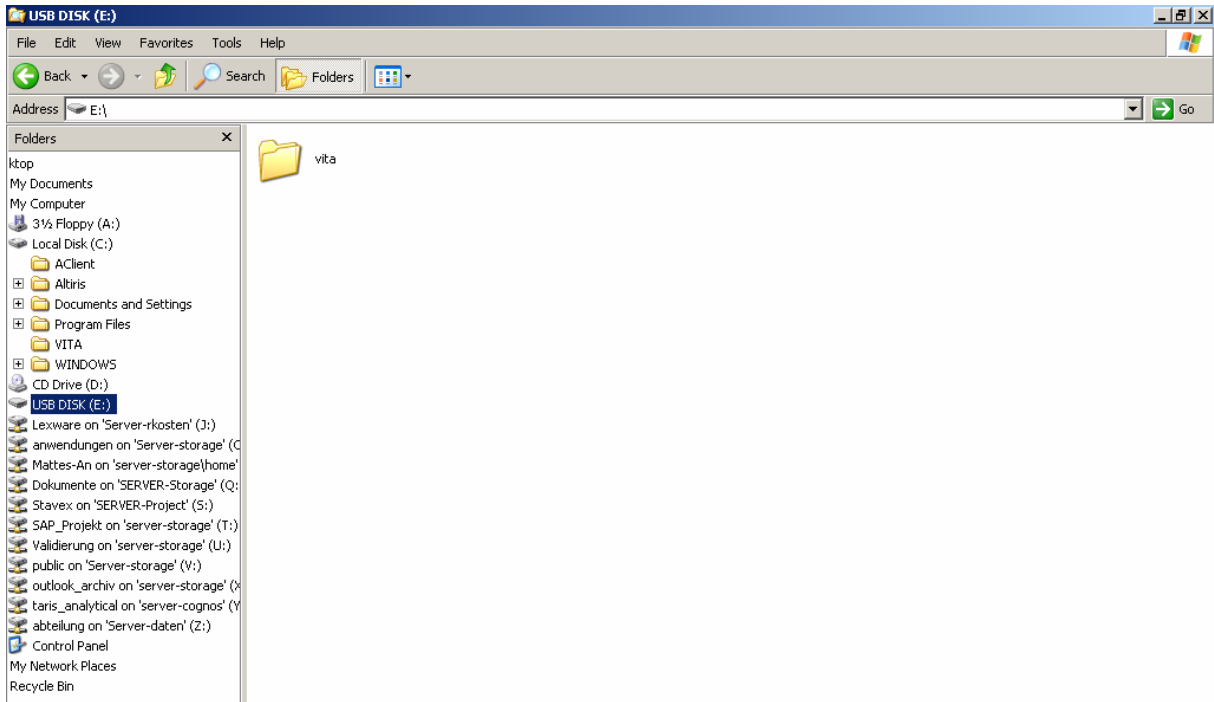

## 2.) Installation of the new software

- Switch furnace „off“ at main switch
- Plug-in memorstick
- Switch furnace „on“ at main switch
- Press button "Service"
- Press button "Info"
- Press button „Update“ , software will be transfered
- After transfer has finished confirm with button "OK"

