# VITA In-Ceram® VITASONIC II

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D

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GB

### Please read carefully prior to use!

Check your ultrasonic unit for shipping damage. Ensure that your mains voltage corresponds to that quoted on the type plate on the rear of the unit.

### Caution:

In the interests of safety, the In-Ceram Vitasonic II unit must only be plugged into a properly earthed socket.

### Filling the unit with liquid

The tank should be 2/3 filled with water. Switch the unit on with the timer.

- Use the automatic timer for mixing times of up to 15 minutes. Set the desired time by turning the knob to the right. The unit will switch off automatically once the set mixing time has elapsed.
- Should the desired mixing time exceed 15 minutes, or if the unit is to be used continuously, turn the knob to the left until it clicks into position, indicating the clock symbol. In this setting, the unit must be switched off manually.
- When using the unit as an ultrasonic cleaner, the cleaning solutions may discolour the stainless steel tank. These can be removed with any household cleaning material.

### Important:

Inflammable solutions must not be used in ultrasonic units due to the increased risk of explosion.

Do not put your hand into the tank whilst the unit is switched on. Always unplug the unit prior to cleaning it.

Keep all working surfaces clean and dry, in order to prevent accidents such as electric shocks.

The product was developed for dental use and must be operated according to the operating instructions.

The information/illustrations contained herein do not constitute an assurance of properties and are not binding.

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VITA Zahnfabrik H. Rauter GmbH & Co. KG