

VITA ENAMIC® STAINS KIT: For more natural shades

Reproducing natural shades in a simple and fast manner



VITA shade taking

VITA shade communication

VITA shade reproduction

VITA shade control

Date of issue: 01.13

VITA shade, VITA made.

VITA

Product description

The VITA ENAMIC STAINS KIT is comprised of six stains and accessories for the reproduction of natural shade nuances of hybrid ceramic restorations. The stains are bonded to the restoration in a polymerization process. VITA ENAMIC GLAZE, available for surface sealing, increases the durability and brilliance

of the shades in the oral environment. Processing is based on a simple step-by-step principle: condition the surface, mix and apply the shades, carry out intermediate polymerization, apply chemical glaze material and perform final polymerization.

Product advantages

Fast processing:

Simply apply the VITA ENAMIC stains to the restoration, polymerize and that's it! This way, the shade of VITA ENAMIC restorations can be quickly characterized.

High level of individuality:

The anomalies and shade nuances of natural teeth can be individually characterized with the six VITA ENAMIC stains.

Simple processing:

The shade intensity of the VITA ENAMIC stains can be perfectly controlled by the ratio of liquid to shade powder. This way homogeneous application of shades is ensured.

Indication range

VITA ENAMIC stains are suitable for the characterization (occlusal/cervical) of restorations made of VITA ENAMIC hybrid ceramic.

Product data

Range of shades: VITA ENAMIC STAINS are available in six shade nuances: white, corn, brown, khaki, blue and black.

Accessories: liquid, glaze, cleaner, microbrush, mixing plate, spatula.

VITA ENAMIC® STAINS

VITA ENAMIC® STAINS KIT – Prod. No. EENSTKIT

Designation	Description	Content / shade
VITA ENAMIC STAINS 1-6	6 x bottle cont. 2 g	white, corn, brown, khaki, blue und black
VITA ENAMIC STAINS LIQUID	bottle	20 ml
VITA ENAMIC GLAZE	bottle	20 ml
VITA VM LC CLEANER	bottle	50 ml
Microbrush disposable applicators	pack	20 pieces each
Brush No. 1	–	2 pieces
Porcelain mixing plate, black	8,5 x 11 mm	1 piece
Mixing spatula, double-ended	–	1 piece

VITA ENAMIC® STAINS KIT – single products

Designation	Description	Content / shade	Prod. No.
VITA ENAMIC STAIN 1	bottle cont. 2 g	white	EENST012
VITA ENAMIC STAIN 2	bottle cont. 2 g	corn	EENST022
VITA ENAMIC STAIN 3	bottle cont. 2 g	brown	EENST032
VITA ENAMIC STAIN 4	bottle cont. 2 g	khaki	EENST042
VITA ENAMIC STAIN 5	bottle cont. 2 g	blue	EENST052
VITA ENAMIC STAIN 6	bottle cont. 2 g	black	EENST062
VITA ENAMIC STAINS LIQUID	bottle	20 ml	EENSTL20
VITA ENAMIC GLAZE incl. microbrush disposable applicators	bottle / pack	20 ml / 20 pieces each	EENSTG20
VITA VM LC CLEANER	bottle	50 ml	CVMLC50
Microbrush disposable applicators	pack	20 pieces each	EENSTMB20
Brush No. 1	–	1 piece	B085
Porcelain mixing plate, black	8,5 x 11 mm	1 piece	C014
Mixing spatula, double-ended	–	1 piece	C013

System compatibility

The following dental and dental-technical polymerization units are recommended for intermediate and final polymerization: Bluephase C8 (Ivoclar Vivadent), Astralis 10 (Ivoclar Vivadent), Valo (Ultradent), Speed Labolight (Hager & Werken), Heraflash (Heraeus Kulzer) and Polyflux PT (Dreve). Tradenames shown are property of their respective owners.

More information about VITA ENAMIC is available at www.vita-enamic.de / www.vita-enamic.com

VITA