

Brenntabellen VITABLOCS im VACUMAT

VITABLOCS

VITABLOCS

	Vt. °C	→ min.	↗ min.	↗ °C/min	Temp. ca. °C	→ min.	VAC min.
Malfarben-Fixierbrand	500	4.00	5.00	76	880	0.30	0.00
Malfarbenbrand	600	6.00	4.00	83	930	1.00	0.00
Glasurbrand	600	6.00	6.00	58	950	1.00	0.00

VITA shade, VITA made.

VITA