


Brenntabelle VITA VM 15 im Austromat M

NAME	START		↑	→	VAC LEVEL	°C ↗ Min.	END	→ min. sec.	(V)	↘ 1	2 ↘
Washopaque Pulver	400	2	1	0	9	70	820	1:00		0	0
Washopaque Paste	400	3	3	0	9	70	820	1:00		0	0
Opaque Pulver	400	2	1	0	9	70	820	1:00		0	0
Opaque Paste	400	3	3	0	9	70	820	1:00		0	0
Margin	400	3	3	0	9	50	810	1:00		0	0
Effect Liner	400	3	3	0	9	50	810	1:00		0	0
1.Dentinbrand	400	3	3	0	9	50	800	1:00		0	0
2.Dentinbrand	400	3	3	0	9	50	790	1:00		0	0
Glanzbrand mit Glasurmasse	400	2	2	0	0	80	790	1:00		0	0
Korrektubrand Corrective	400	2	2	0	9	50	750	1:00		0	0

Dies sind Brennwerte die uns seitens Kunden überlassen wurden eine Gewähr kann nicht übernommen werden.