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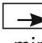
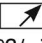
Silver Test Set Temperature test set for dental porcelain furnaces

GB

Contents: 6 ceramic firing trays, white
3 silver rods, 70 mm long
1 set of instructions

The furnace must be kept for approx. 30 minutes at standby temperature before carrying out the silver sample test. The melting point of silver is 961°C. Cut off a piece of silver wire about 10 mm long and place it into one of the ceramic firing trays. Place the ceramic firing tray with the silver wire, using no further firing tray, onto the middle of the firing platform and fire as follows:

	pre-drying	 min.	 °C/min.	 min.	approx. Temp. °C	 min.	VAC min.
1st temperature test	600	1.00	59	6.00	955	4.00	–
2nd temperature test	600	1.00	61	6.00	965	4.00	–

At the end of the first temperature test the silver wire should **not** have melted. The same sample can now be subjected to a second temperature test. The temperature is adjusted to 965°C. The other firing parameters are identical to those of the first temperature test. After firing, the silver wire must have melted.

If a temperature variation occurs during testing, the firing temperature can be altered at VITA VACUMAT furnaces using the temperature adjustment function (see the Operating Instructions for VACUMAT furnaces).

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