VITA NP BOND Paste

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D

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Instructions for use

GB

Before using, please read the instructions for use.

VITA NP BOND Paste is a ceramic material that balances the differing coefficients of thermal expansion between non-precious-metal alloys and ceramics (such as VITA VMK MASTER or VITA VM 13, for example). It also protects the ceramic from metal oxides emitted from the metal.

Indication:

VITA NP BOND Paste can be used with all non-precious metal alloys such as cast, CAD/CAM or laser-sintered alloys.

Framework preparation:

- Prepare the framework in accordance with the criteria specified by the alloy manufacturer.
- Depending on the manufacturer specifications, oxidation firing must be carried out.
- Once oxidation firing has been completed, the metal surface must be sandblasted with 110µm and approx. 2-3 bar pressure.
- Afterwards, the metal surface may no longer be touched.
- Clean with pressurized air.

Mixing:

- Before using for the first time, VITA NP BOND Paste should be mixed thoroughly using the enclosed spatula.
- If VITA NP BOND Paste has dried out, it can be mixed to its original consistency again using VITA PASTE FLUID. Other liquids are not suitable.

Application:

- Using a brush, apply an opaque layer of VITA NP BOND Paste to the framework (similar to a washbake) and then perform firing in the ceramic furnace.

Firing chart:

Vt. in °C	→ min.	↗ min.	↘ °C/min.	Temp. approx. °C	→ min.	VAC min.
600	6.00	6.00	60	960	1.00	6.00

The values specified are guide values and are intended solely as a general guide. The firing results depend on the performance, make and age of the furnace.

A washbake is no longer necessary once VITA NP BOND Paste has been used.

Once firing of the NP BOND Paste has been completed, apply OPAQUER as usual and then perform secondary firing of the ceramic.

More information on processing as well as an application film is available on our Internet page <https://www.vita-zahnfabrik.com/npbond>

Storage:

VITA NP BOND Paste can be stored in a dry, airtight container at room temperature (10-25°C) for 48 months.

Supplementary information:

We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers that are not compatible or not authorized for use with our product.

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VITA Zahnfabrik H. Rauter GmbH & Co.KG