

VITA INTERNO®

Creating in-depth natural appearance

Creating natural and individual characteristics, mamelons and calcifications effortlessly



VITA shade determination

VITA shade communication

VITA shade reproduction

VITA shade control

Date of issue: 03.15

VITA shade, VITA made.

VITA

Product description

VITA INTERNO are fine-grained and strongly fluorescent stains with high shade intensity for in-depth characterization of ceramic restorations. They offer an almost unlimited range of applications to demanding dental technicians: natural reproduction of all individual characteristics in the interior of the tooth is enabled.

The materials can be mixed with one another independent of the individual CTE. Users can benefit from maximum creative freedom with an unlimited variety of shades. Applied on their own or mixed into the ceramic materials, VITA INTERNO stains can be used with **all** VITA veneering ceramics.

VITA INTERNO®

creating in-depth natural appearance

Application area

VITA INTERNO stains are shaded according to the anomalies occurring in natural teeth. These materials enable the user to reproduce natural shade effects and obtain a perfect match of ceramic restorations to natural teeth.

Above all VITA INTERNO enables natural reproduction of shade effects occurring in the incisal area of natural teeth.

They can be used either on their own or mixed into base dentine, opaque dentine, dentine, enamel and translucent materials.

VITA INTERNO materials can be universally used for the following products: VITA VM 7, VM 9, VM 11, VM 13, VM 15, VITA VMK Master and VITA TITANKERAMIK.

VITA INTERNO materials are distinguished by...

- **Fine-grained structure:** VITA INTERNO materials are very suitable for washing into the ceramic materials thanks to their special structure
- **Intensive shades:** the shade intensity allows VITA INTERNO to achieve shade effects visible through the overlying layers and to create a chroma from the depth of the veneer
- **Strong fluorescence:** the fluorescence results in high luminosity of the shades and increased light transmission within the ceramic
- **Can be mixed with one another:** INTERNO materials can be shaded with other INTERNO materials and the shade can also be individualized
- **Can be mixed with all VITA ceramic materials:** when mixed with the ceramic materials, the fluorescence and the natural luminosity of the ceramic materials in varying light conditions are increased



VITA INTERNO® – the product line




VITA INTERNO KIT (BISETV2)

Quantity	Contents	Material
14	3 g	VITA INTERNO Int01 - Int14
1	20 ml	VITA INTERNO FLUID
1		Glass spatula for stains
1		Brush No. 3/0 a & e
1		VITA INTERNO mixing plate, acrylic (Ø 144 mm, with indentations)
1		VITA INTERNO shade sample disc

“ For me VITA INTERNO stains are the key to veneers with a perfectly natural appearance. ”

Ceramic expert Renato Carretti,
dental technician from Zurich

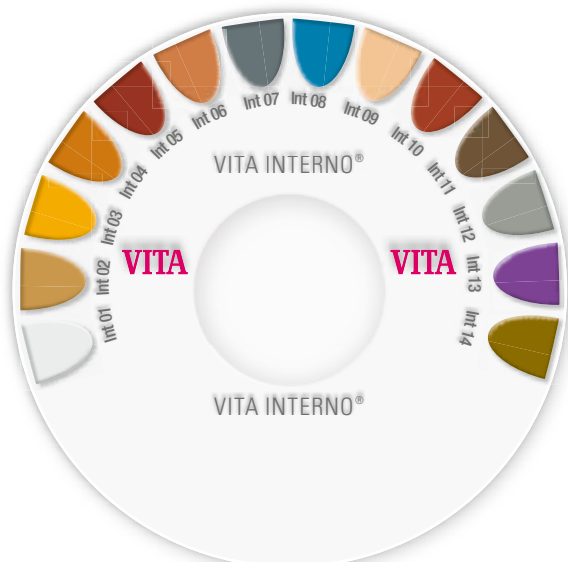

VITA INTERNO stains

The VITA INTERNO product line includes 14 stains which form the basis for an unlimited number of shades. They offer the user the benefit of unlimited freedom of reproducing truly natural tooth effects.

	Int 01 white		Int 08 blue
	Int 02 sand		Int 09 beige
	Int 03 yellow		Int 10 brown
	Int 04 orange		Int 11 grey-brown
	Int 05 terracotta		Int 12 grey
	Int 06 light brown		Int 13 lilac
	Int 07 anthracite		Int 14 khaki

Individual materials:**VITA INTERNO, 3 g**

Designation	Shade	Prod. No. 3 g
Int01	white	BI013
Int02	sand	BI023
Int03	yellow	BI033
Int04	orange	BI043
Int05	terracotta	BI053
Int06	light brown	BI063
Int07	anthracite	BI073
Int08	blue	BI083
Int09	beige	BI093
Int10	brown	BI103
Int11	grey-brown	BI113
Int12	grey	BI123
Int13	lilac	BI133
Int14	khaki	BI143

VITA INTERNO shade sample device:

Designation	Prod. No.
VITA INTERNO shade sample disc	B444

VITA INTERNO liquids:

Designation	Prod. No.
VITA INTERNO FLUID, 20 ml	BIF20

VITA