VITACOLL®

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Warnings

Hole retentions must be avoided, as the denture base material could partially flow into the hole retentions, causing hollow spaces to form in the tooth. These hollow spaces could weaken the tooth, with the result that the tooth could break even at minimum occlusal load.

b. Using VITACOLL®

- VITACOLL® liquid bonding agent is applied evenly using a brush to the roughened surfaces, and using circular movements, to the roughened edges of the acrylic teeth.

- VITACOLL® bonding agent must then be left to act for **5 minutes**. During this time, the evenly coated base surface should have a silk-matte appearance - otherwise, the areas of the tooth that are not glossy must be coated again with VITACOLL® liquid.

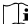
- Once the working time has passed, the denture base material must be applied within **10 minutes**. If 10 minutes are exceeded, VITACOLL® no longer offers sufficient adhesion and you must repeat the process from step b. "Using VITACOLL®".

Storage and storage information

Close the bottle immediately after use with the original cap.

Store properly closed in a cool, dark and dry location!

Expiry date: see the inscription on the label.

 Please read the working instructions.

Safety information / storage

Danger

Important

Active ingredients

Methacrylate, butanone

Safety information

H225 Highly flammable liquid and vapor.

H315 Causes skin irritation.

H319 Causes severe eye irritation.

H317 May cause allergic skin reactions.

H335-H336 May irritate the respiratory tract. May cause drowsiness and dizziness.

Labeling

The batch no. [LOT] is shown on the container.

Delivery form

VITACOLL®, 1 bottle, 100 ml bonding agent liquid, prod. no. A9C100

This product has been developed for dental applications and must be processed in accordance with the instructions for use. The information and illustrations provided do not constitute a guarantee of features and are not binding. The manufacturer does not assume any liability. Date of issue of these instructions for use: 11.2014.

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Working instructions

Product description

VITACOLL® is a liquid bonding agent that creates a reinforced bond between PMMA-based acrylic teeth and denture base materials.

Indication

The use of VITACOLL® liquid bonding agent is recommended for heat-curing polymer resins, and is a mandatory requirement in the case of self-curing / cold-curing polymer resins. Today's self-curing polymer resins can generally be used with both the casting and packing techniques. In order to always achieve a reinforced bond between acrylic teeth and the base material, the teeth should be treated with VITACOLL®.

Contraindication

VITACOLL® may only be used as a bonding agent for PMMA-based acrylic teeth.

Composition

VITACOLL® contains methyl methacrylate and butanone.

Further information is provided in the safety data sheet available from the download center at www.vita-zahnfabrik.com or via fax from (+49) 7761-562-281.

Instructions for use

a. Preparation of the acrylic teeth

Surface treatment of acrylic teeth increases their adhesive force when bonding with acrylic denture base material. The base surface of acrylic teeth should therefore be pre-treated as described below before transferring the dentures to acrylic, i.e. prior to packing, casting, injecting or injection molding the denture base material:

- Cleaning:** First remove wax or oily residue and other contaminants from the acrylic teeth by cleaning them in boiling water.

Warning

Wax and contaminants must be thoroughly removed from the base surfaces of the teeth. Otherwise there may be a loss of adhesive force between the acrylic teeth and the denture base material.

- Abrasion:** The base surfaces of the acrylic teeth are roughened using a powder jet with corundum particles (grain size 100-150 µm) at an air pressure of 2-3 bar; alternatively, a milling tool can be used.

TIP:

To improve bonding, the teeth should also be roughened using circular movements in the area that will subsequently be surrounded by the denture base material. As a rule, sharp edges are to be avoided in the base area, otherwise they must be rounded off. This is because edges that are too thin can fracture easily.

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VITA Zahnfabrik H. Rauter GmbH & Co.KG