
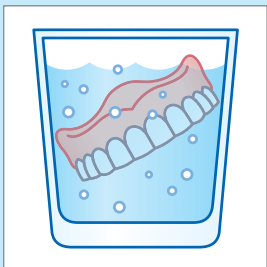
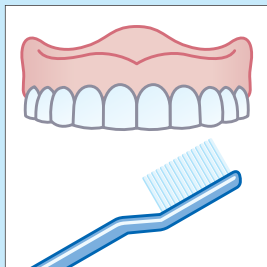


## The safe way to clean dentures

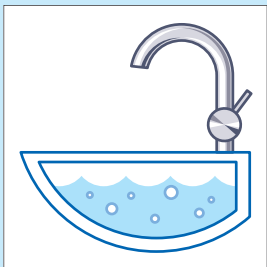

Date of issue 09.15

VITA shade, VITA made.

**VITA**



## Dear Patient.

Today you received your new dentures from us – many thanks for your trust!

A **highly qualified team of dentists, dental assistants and dental technicians** have made your customized dentures with the **best materials** and **according to the latest findings**.

Thanks to the high esthetics of VITA prosthetic teeth, your dentures will support your natural smile for years to come. You will enjoy **outstanding functionality** and **extraordinary oral comfort**.

To ensure that you enjoy your dentures for as long as possible, as well as to protect your health, we would like to give you a few instructions on caring for your dentures. If you have any further questions, please don't hesitate to contact us – you will find our contact information on the reverse.


## First cleaning step: rinsing



To **protect against deposits** and to **prevent infection**, please rinse your dentures thoroughly under **running water in the morning and the evening**.

In order to protect your dentures, we recommend cleaning them in a sink filled with water. Alternatively you can place a towel at the bottom of the sink.

## Second cleaning step: cleaning and brushing

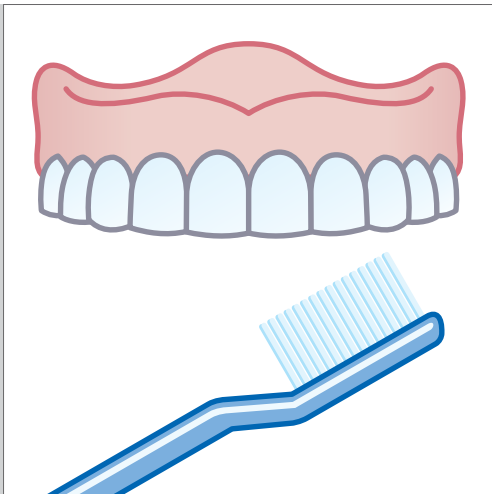


Next, clean your dentures in your hand **with a soft brush**. Manual tooth brushes or tooth brushes especially made for dentures with **nylon bristles** are ideal.

**Important:** please take care to brush **each and every tooth carefully**, and pay particular attention to the **gaps between the teeth** and all places in which there is contact to your gums or natural teeth.

Please use **warm water** for cleaning and a **toothpaste WITHOUT abrasive particles**.

## Third cleaning step: soaking

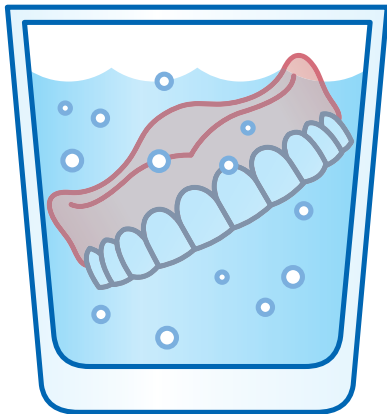

**Rinse** your dentures again thoroughly after cleaning **with cold water**, and for deep cleaning put them in a sufficiently large **container** filled with water.

If deposits begin to appear despite regular cleaning, you can make use of an ultrasonic cleaning instrument. Please speak with your dentist about this. Additionally a ten-minute soak in a 5–10% vinegar solution is also possible. In order to guarantee that your dentures with VITA teeth have a long life, we recommend that you do not use cleaning tablets or solutions, which can be quite aggressive, and observe the cleaning instructions provided here.

## Fourth cleaning step: rinsing off



**Before you wear your dentures**, rinse them off thoroughly under running water.

Regular cleaning and check-ups by your dentist will ensure that your dentures retain their high quality for a long time. In addition, by following these instructions you protect your health and avoid possible repair costs.

**We hope you always have a good bite!**



A friendly recommendation from your dentist / dental technician

**We are happy to support you!**

**VITA**

---

VITA Zahnfabrik H. Rauter GmbH & Co. KG

Spitalgasse 3 · D-79713 Bad Säckingen · Germany

Tel. +49 (0) 7761 / 562-0 · Fax +49 (0) 7761 / 562-299

Hotline: Tel. +49 (0) 7761 / 562-222 · Fax +49 (0) 7761 / 562-446

[www.vita-zahnfabrik.com](http://www.vita-zahnfabrik.com) · [info@vita-zahnfabrik.com](mailto:info@vita-zahnfabrik.com)

 [facebook.com/vita.zahnfabrik](https://facebook.com/vita.zahnfabrik)