

VITA SUPRINITY PC[®]

Recommmodation for CNC machine

Machining mode : Grinding - Block

- Information and tips
 - Tools
 - Machining strategy
 - Parameters
-

VITA

Version: 12.12.17

Information

The information presented here, are intended as a recommendation.
Depending on the available CNC machines, CAM software, tools, etc. the information have to be adapted to your own production situation.
As a result, different results may obtained.

The development of the strategies and parameter was done with following system:

- imes-icore CORiTEC350i
- CAM Software: Hyperdent 8.2 Beta

According to this recommendation, a fully anatomical posterior tooth crown (tooth 26) can be finished in 00:25:30 min, with a good surface and fit.

We recommend Tools from:

FRANKEN GmbH & Co. KG, Fabrik für Präzisionswerkzeuge
Tools for the Dental Industry
www.franken-dental.com

Tips for VITA SUPRINITY PC®

Avoid vertically or fast plunge movements. It is important that the tool always plunge slow and soft into the material.

- Plunge into the material with a ramp (5 degree) or helically and use a reduced plunge feed (feed Z)
- We recommend to grind VITA SUPRINITY wet
- The diameter of the restoration holding pin should be 3,0 - 3,5 mm

Strategy

- A two side machining and 3+2 strategies are sufficient in most cases.
- In order to maintain a good fit, even by restoration with undercuts, the last finishing of the cavity should be done with a 5 axis strategy.
- In order to maintain a good occlusal fit, the complete occlusal side should be finished with max. a $\varnothing 1.2\text{mm}$ tool (or less). In that way, a special finishing of the fissures isn't necessary.
- If a smaller tool is used after a bigger one, it can be necessary to use a roughing strategy to remove remaining material.
Tool life and process reliability are increased this way.
- To process cavities or pockets, the tool should be tilted 4-7 degrees (5 axis strategy). This will decrease the wear of the tool tip.
- When using grinding tools, the whole grinding body should be used.



Recommended Tools

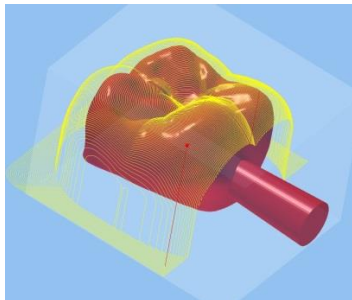
Diameter	Grain size	Description	Manufacturer	Order-Code
Ø 3 mm	D126	Diamond ball nose grinding burr	Emuge-Franken	1716.300613 (6mm shaft)
Ø 2 mm	D126	Diamond ball nose grinding burr	Emuge-Franken	1716.200611 (6mm shaft)
Ø 1 mm	D76	Diamond ball nose grinding burr	Emuge-Franken	1716.100609 (6mm shaft)

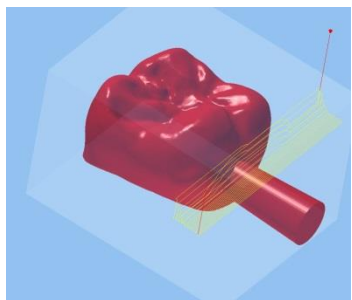
Tool Life

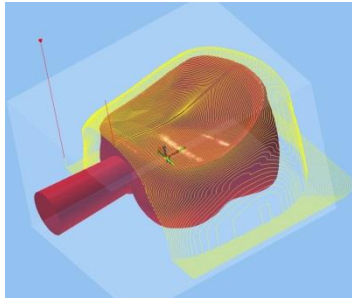
Tool	Units	Restoration
Ø 3 mm Diamond ball nose grinding burr	90	Fully anatomical crown tooth 26
Ø 2 mm Diamond ball nose grinding burr	150+	Fully anatomical crown tooth 26
Ø 1 mm Diamond ball nose grinding burr	65	Fully anatomical crown tooth 26

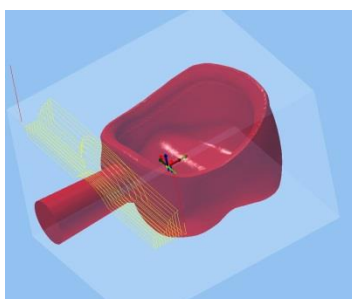
Bearbeitungsreihenfolge

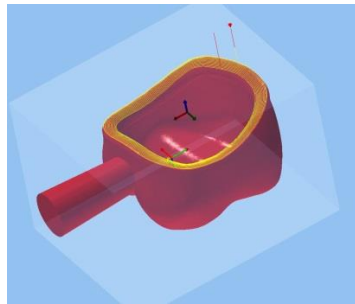
Schritt	Bearbeitungsseite	Bearbeitung	Werkzeug
1	Occlusal side	Roughing 1	Ø 3mm
2	Occlusal side	Roughing 2	Ø 3mm
3	Cavity side	Roughing 1	Ø 3mm
4	Cavity side	Roughing 2	Ø 3mm
5	Preparation margin inside	Pre-Finishing	Ø 2mm
6	Preparation margin outside	Pre-Finishing	Ø 2mm
7	Cavity, inside	Roughing	Ø 2mm
8	Cavity, outside	Finishing	Ø 2mm
9	Occlusal side	Pre-Finishing	Ø 2mm
10	Preparation margin inside	Finishing	Ø 1mm
11	Preparation margin outside	Finishing	Ø 1mm
12	Cavity, inside	Finishing	Ø 1mm
13	Cavity, inside	Remaining material	Ø 1mm
14	Occlusal side, fissures	Finishing	Ø 1mm

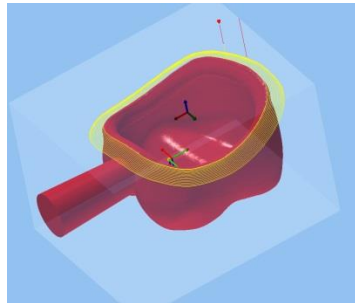
Step 1	Occlusal side - roughing 1				3 axis
	Tool	Ø 3mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	1800	mm/min	
	Feed speed Z	[Vf]	500	mm/min	
	Width of cut XY	[ae]	0,15	mm	
	Depth of cut Z	[ap]	Full Tool	mm	
	Override		0,15	mm	

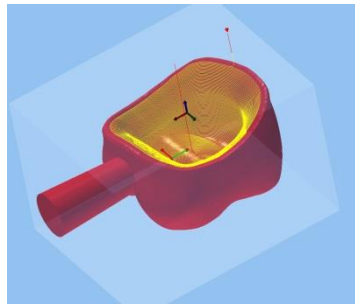
Step 2	Occlusal side - roughing 2				3 axis
	Tool	Ø 3mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	1500	mm/min	
	Feed speed Z	[Vf]	500	mm/min	
	Width of cut XY	[ae]	0,12	mm	
	Depth of cut Z	[ap]	1/3 Block height	mm	
	Override		0,15	mm	

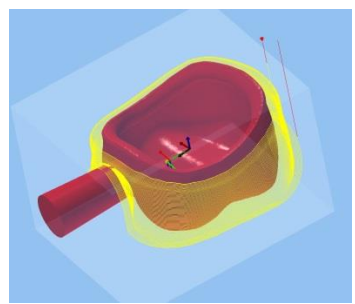
Step 3	Cavity side - roughing 1				3 axis
	Tool	Ø 3mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	1800	mm/min	
	Feed speed Z	[Vf]	500	mm/min	
	Width of cut XY	[ae]	0,15	mm	
	Depth of cut Z	[ap]	Full Tool	mm	
	Override		0,15	mm	

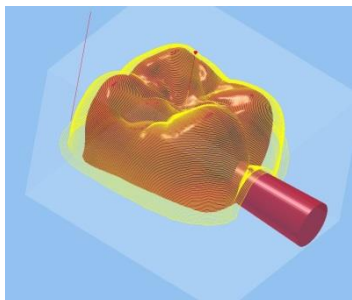
Step 4	Cavity side - roughing 2				3 axis
	Tool	Ø 3mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	1500	mm/min	
	Feed speed Z	[Vf]	500	mm/min	
	Width of cut XY	[ae]	0,12	mm	
	Depth of cut Z	[ap]	1/3 Block height	mm	
	Override		0,07	mm	

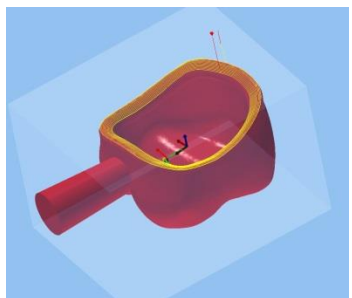
Step 5	Preperation margin inside - Pre-Finishing			3+2 axis
	Tool	Ø 2mm		
	Tolerance	0,01		
	Spindel speed	[n]	50000	rpm
	Feed speed XY	[Vf]	1300	mm/min
	Feed speed Z	[Vf]	1000	mm/min
	Width of cut XY	[ae]	0,1	mm
	Depth of cut Z	[ap]	-	mm
	Oversize		0,07	mm

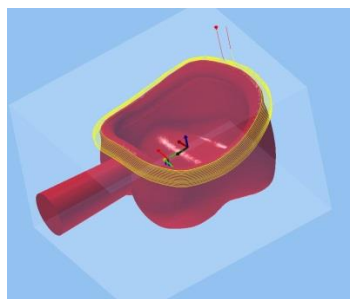
Step 6	Preperation margin outside - pre-finishing			3+2 axis
	Tool	Ø 2mm		
	Tolerance	0,01		
	Spindel speed	[n]	50000	rpm
	Feed speed XY	[Vf]	1300	mm/min
	Feed speed Z	[Vf]	1000	mm/min
	Width of cut XY	[ae]	0,1	mm
	Depth of cut Z	[ap]	-	mm
	Oversize		0.07	mm

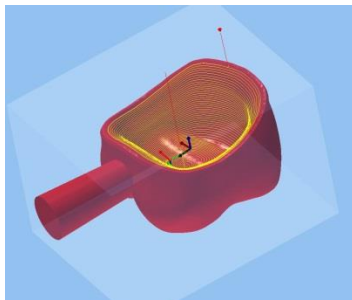
Step 7	Cavity, inside - roughing			3+2 axis
	Tool	Ø 2mm		
	Tolerance	0,01		
	Spindel speed	[n]	50000	rpm
	Feed speed XY	[Vf]	1200	mm/min
	Feed speed Z	[Vf]	1000	mm/min
	Width of cut XY	[ae]	0,1	mm
	Depth of cut Z	[ap]	-	mm
	Oversize		0	mm

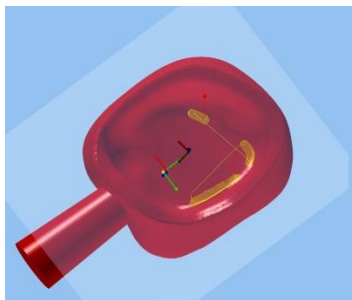
Step 8	Cavity, outside - finishing			3+2 axis
	Tool	Ø 2mm		
	Tolerance	0,01		
	Spindel speed	[n]	50000	rpm
	Feed speed XY	[Vf]	1300	mm/min
	Feed speed Z	[Vf]	1000	mm/min
	Width of cut XY	[ae]	0,1	mm
	Depth of cut Z	[ap]	-	mm
	Oversize		0	mm

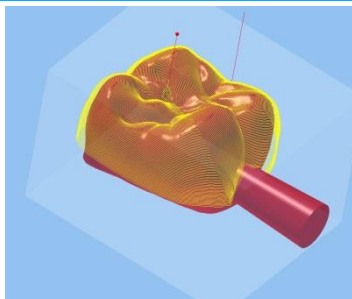
Step 9	Occlusal side - pre-finishing				3+2 axis
	Tool	Ø 2mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	1300	mm/min	
	Feed speed Z	[Vf]	1000	mm/min	
	Width of cut XY	[ae]	0,12	mm	
	Depth of cut Z	[ap]	-	mm	
	Oversize		0	mm	

Step 10	Preperation margin inside - finishing				5 axis
	Tool	Ø 1mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	1000	mm/min	
	Feed speed Z	[Vf]	1000	mm/min	
	Width of cut XY	[ae]	0,1	mm	
	Depth of cut Z	[ap]	-	mm	
	Oversize		0	mm	

Step 11	Preperation margin outside - finishing				5 axis
	Tool	Ø 1mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	1000	mm/min	
	Feed speed Z	[Vf]	1000	mm/min	
	Width of cut XY	[ae]	0,1	mm	
	Depth of cut Z	[ap]	-	mm	
	Oversize		0	mm	

Step 12	Cavity inside - finishing				5 axis
	Tool		Ø 1mm		
	Tolerance		0,01		
	Spindel speed		[n]	50000 rpm	
	Feed speed XY		[Vf]	1200 mm/min	
	Feed speed Z		[Vf]	1000 mm/min	
	Width of cut XY		[ae]	0,2 mm	
	Depth of cut Z		[ap]	- mm	
	Oversize			0 mm	

Step 13	Cavity - remaining material				5 axis
	Tool	Ø 1mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	500	mm/min	
	Feed speed Z	[Vf]	250	mm/min	
	Width of cut XY	[ae]	0,1	mm	
	Depth of cut Z	[ap]	0,05	mm	
	Oversize		0	mm	

Step 14	Occlusalside, fissures - finishing				3+2 axis
	Tool	Ø 1mm			
	Tolerance	0,01			
	Spindel speed	[n]	50000	rpm	
	Feed speed XY	[Vf]	1500	mm/min	
	Feed speed Z	[Vf]	1000	mm/min	
	Width of cut XY	[ae]	0,1	mm	
	Depth of cut Z	[ap]	-	mm	
	Oversize		0	mm	

Formulas for cutting data calculation

Expression used in text	Term	Symbol	Formula
<i>Feed speed XY</i> <i>Feed speed Z</i>	<i>Feed speed</i>	Vf [mm/min]	$Vf = fz * z * n$
<i>Spindle speed</i>	<i>Spindle speed</i>	n [U/min]	$n = \frac{Vc * 1000}{\pi * d}$
<i>Width of cut XY</i>	<i>Width of cut</i>	ae [mm]	
<i>Depth of cut Z</i>	<i>Depth of cut</i>	ap [mm]	
		fz [mm]	$fz = \frac{Vf}{n * z}$
		Vc [m/min]	$Vc = \frac{\pi * d * n}{1000}$

Secrecy

We point out that, according to the declaration of intent and secrecy, both parties committed themselves to treat business and company secrets and any other confidential information of the other party confidentially - whether they are of a technical or commercial nature and regardless of the way they have been disclosed - including all documents, drawings and records of data, and not to disclose them to any third parties.

The obligation to maintain secrecy shall not apply if confidential information was already known to the recipient of the information or the public or if such information was disclosed to him or the public through no fault of the recipient of the information; the recipient of the information shall be obliged to furnish proof.

The obligation to maintain secrecy shall remain in force after termination of the cooperation as long as the information has not been disclosed by a third party.

Exclusion of liability

VITA Zahnfabrik H. Rauter GmbH & CO KG reserves the right not to be responsible for the topicality, correctness, completeness or quality of the information provided. Liability claims regarding damage caused by the use of any information provided, including any kind of information which is incomplete or incorrect, will therefore be rejected.

Our products should be used according to the working instructions. We cannot be held liable for damages resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of application.