

VITA FiringAssist i-Line

Program for processing VITA V60 i-Line® firing programs

Quick user guide – Adding new firing programs to an existing program list



VITA shade determination

VITA shade communication

VITA shade reproduction

VITA shade control

Issued 11.16

VITA shade, VITA made.

VITA

Table of contents

1	Installation of the VITA FiringAssist i-Line	4
2	Important information	4
3	Transferring a program list from VITA V60 i-Line to VITA FiringAssist i-Line	5
3.1	Preparation of the USB flash drive	5
3.2	Exporting a program list from the VITA V60 i-Line	6
3.3	Importing a program list into VITA FiringAssist i-Line	6
4	Processing the program list	7
5	Adding programs to the program list	8
6	Transferring a new program list into VITA V60 i-Line	9
6.1	Exporting a program list to USB flash drive	9
6.2	Importing a program list into the VITA V60 i-Line	10
6.3	Preparing a USB flash drive for further use	10

1 Installation of the VITA FiringAssist i-Line

To install the VITA FiringAssist i-Line, carry out the following steps:

- Install the file **VITA_FiringAssist_i_Line_installer_1_0_0.exe** on the PC
Administrator rights may be needed for the installation.
- Indicate the installation path during the installation (e.g., *C:\Programmew60tool*).

After the program is successfully installed, there will be a new menu item with the name *VITA FiringAssist i-Line* in the Start menu.

There, in the *VITA FiringAssist i-Line program*, these instructions and a deinstallation routine can be called up.

2 Important information


If a program list is transferred to a VITA V60 i-Line, the entire firing program list already present in the VITA V60 i-Line will be overwritten.

To avoid an unwanted overwrite of the existing program list in a VITA V60 i-Line, a backup copy of the VITA V60 i-Line firing program list must always be exported with an empty, prepared USB flash drive.

3 Transferring a program list from VITA V60 i-Line to VITA FiringAssist i-Line

To be able to import the program list saved in the VITA V60 i-Line into the VITA FiringAssist i-Line, a USB flash drive must first be prepared. Then the program list can be exported from the VITA V60 i-Line and imported into the VITA FiringAssist i-Line.

3.1 Preparing the USB flash drive

- Insert a completely empty USB flash drive into the PC.
- Open the VITA FiringAssist i-Line.
- Click on the symbol  (Prepare) in the upper area of the program.

After that, all existing drives (= possible USB flash drives) are displayed in a new window. In the example illustrated below (Fig. 1), for instance, the drive "N:." has the name "KING".

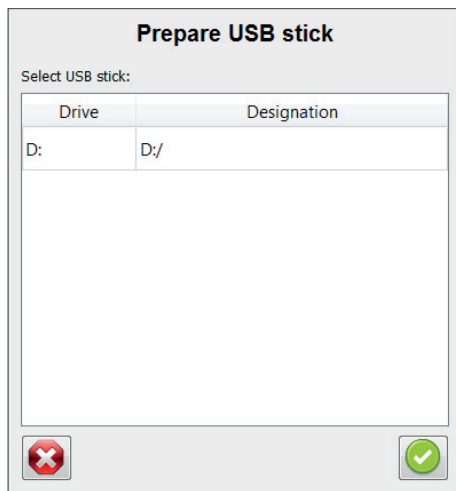


Fig. 1: Selection of the drive (= USB flash drive)

- Select the drive (= USB flash drive) and confirm (or double click on the drive).

A window appears with the information "Preparation successful! The USB flash drive can be ejected and then removed."

3.2 Exporting a program list from the VITA V60 i-Line


- Insert the prepared USB flash drive into the switched off VITA V60 i-Line.
- Turn the VITA V60 i-Line on at the main switch.

After switching on the VITA V60 i-Line, the entire program list of the VITA V60 i-Line is exported to the USB flash drive.

- Wait until the VITA V60 i-Line is ready for operation.
- Then remove the USB flash drive again.

A copy of the program list ("programs.vup") is now located on the USB flash drive in the "programs" folder.

3.3 Importing a program list into VITA FiringAssist i-Line

- Insert the USB flash drive into the PC again.
- In the VITA FiringAssist i-Line, click on the symbol  (Import).
- In the new window, select the drive (= USB flash drive) with a double click.

After that, the imported program list is displayed in the left-hand table of the program window (Fig. 2)

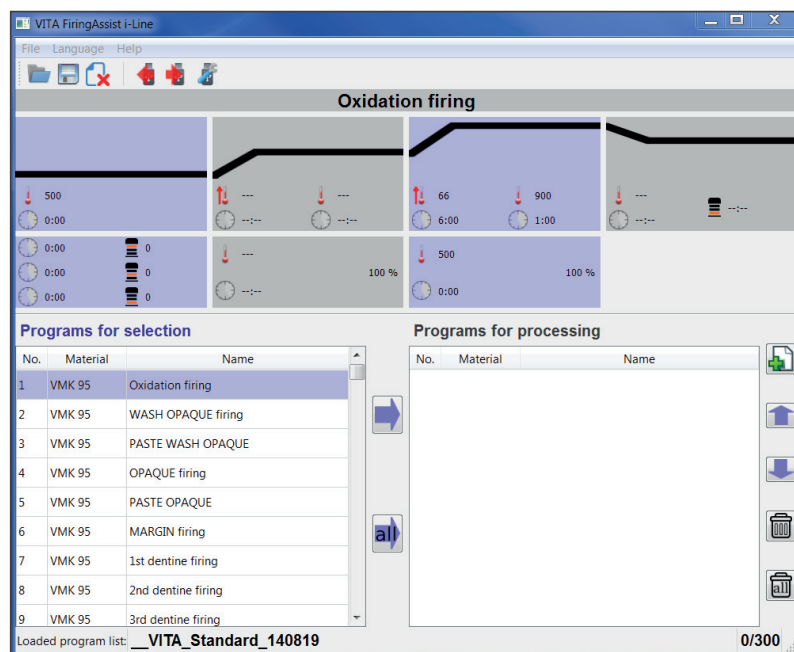



Fig. 2: VITA FiringAssist i-Line program window

4 Processing the program list


- Click on the symbol .

All programs of the program list in the left-hand table are copied into the right-hand table.

Table 1: Left-hand and right-hand table with function options

Left-hand table	Right-hand table
Display of the opened program list	Processing of the opened program list
Program list cannot be processed	Program list can be processed, saved + exported
One or all programs of the opened program list can be moved to the right	Programs can not be moved to the left

Now save a backup copy of the program list on the PC.

- Click on the symbol  (Save).
A new program window for entering a name for the program list opens.


- Enter a name for the program list and confirm.

Tip: If you write the current date with the name, you will know when the last update was.

Then the imported program list can be processed in the right-hand table if desired.

5 Adding programs to the program list

To add new programs from another program list in the right-hand program list, the new program list must be opened in the left-hand table.

- Click the symbol  (Open).
A list with all saved program lists appears.
- Select the desired program list and confirm.

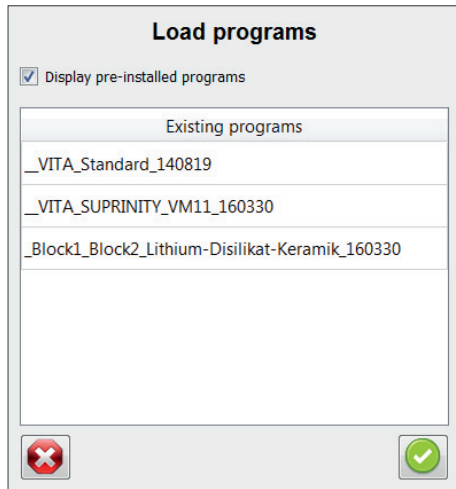





Fig. 3: Load a program selection.

- In the right-hand table, click the program under which the programs to be copied are to be placed.
- Click one or more (keep mouse button pressed) desired programs in the left-hand table.
- Click on the Symbol .

The copied programs are placed in the right-hand table under the marked program.

The position of one or more marked programs in the right-hand table can be changed by clicking the symbols   .
Continuous pressing leads to a continuous movement.

6 Transferring a new program list into VITA V60 i-Line

The expanded program list in the right-hand table can be exported to the USB flash drive and imported into the VITA V60 i-Line.

6.1 Exporting a program list to the USB flash drive

- Click on the symbol  (Export).
- Select the drive (= USB flash drive) and confirm the selection (or double click on the drive).

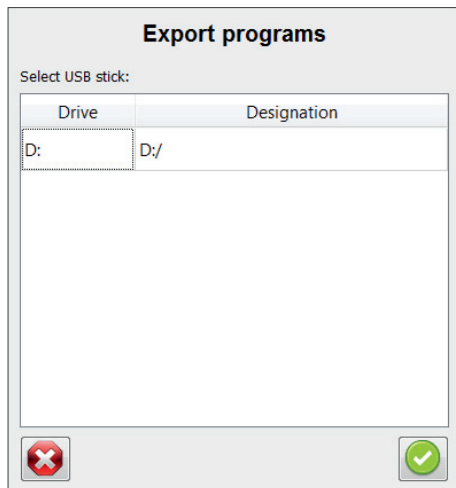


Fig. 4: Selecting the drive (= USB flash drive)

The message "Export successful! The USB flash drive can now be ejected and then removed." confirms the successful export of the data.

If the message "Overwrite existing file?" appears, the file must be overwritten in order to be able to transfer the new program list to the USB flash drive. The new program list is now found on the USB flash drive and can be imported into the VITA V60 i-Line.

6.2 Importing a program list to the VITA V60 i-Line

- Insert the prepared USB flash drive into the **switched off** VITA V60 i-Line.
- Turn the VITA V60 i-Line on at the main switch.


After switching the VITA V60 i-Line on, the entire new program list is imported from the USB flash drive to the VITA V60 i-Line.

⚠ The program list pre-installed in the VITA V60 i-Line will be overwritten!

- Wait until the VITA V60 i-Line is ready for operation.
- Remove the USB flash drive again from the VITA V60 i-Line.

6.3 Preparing a USB flash drive for further use

To prevent any future **unwanted** overwrite of a program list in a VITA V60 i-Line, delete the program list from the USB flash drive after the successful transfer of the new program list.

- Insert the USB flash drive into the PC.
- Open the VITA FiringAssist i-Line.
- Click on the symbol  (Prepare) in the upper area of the program.

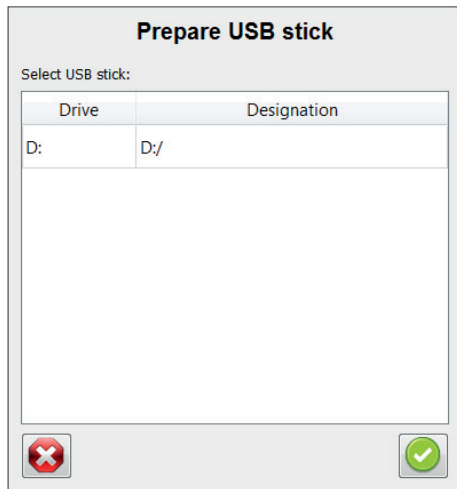


Fig. 5: Selecting the drive (= USB flash drive)


- In the new window, select the drive (= USB flash drive) and confirm (or double click on the drive).

The following query appears: "Delete existing file?"

- Confirm the deletion of the file.

A window opens with the information "Preparation successful! The USB flash drive can now be ejected and then removed." Now the USB flash drive is empty again and prepared for the next use.

With the unique VITA SYSTEM 3D-MASTER, all natural tooth shades can be systematically determined and perfectly reproduced.

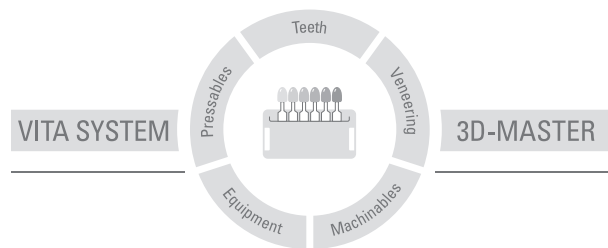

Please note: Our products should be used according to the working instructions. We accept no liability for any damage resulting from incorrect handling or usage. The user is furthermore obliged to check the product before use with regard to its suitability for the intended area of applications. We cannot accept any liability if the product is used in conjunction with materials and equipment from other manufacturers which are not compatible or not authorized for use with our product. Furthermore, our liability for the correctness of this information is independent of the legal ground and, insofar as legally permissible, is limited to the invoiced value of the goods supplied excluding turnover tax. In particular, as far as legally permissible, we do not assume any liability for profit loss, for indirect damages, for consequential damages or for claims of third parties against the purchaser. Claims for damages based on fault liability (fault in making the contract, breach of contract, unlawful acts, etc.) can only be made in the case of intent or gross negligence. The VITA Modulbox is not necessarily a component of the product.

Date of issue of this product information: 11/16

On publication of these instructions for use, previous versions will no longer be valid. The current version can be found under www.vita-zahnfabrik.com

VITA

VITA Zahnfabrik H. Rauter GmbH & Co.KG
Spitalgasse 3 · D-79713 Bad Säckingen · Germany
Tel. +49 (0) 7761 / 562-0 · Fax +49 (0) 7761 / 562-299
Hotline: Tel. +49 (0) 7761 / 562-222 · Fax +49 (0) 7761 / 562-446
www.vita-zahnfabrik.com · info@vita-zahnfabrik.com
 facebook.com/vita.zahnfabrik